

AQUAPURE II

SMART GENUINE FACIAL CARE SYSTEM



Cluederm™
Entire Beauty Solution

Cryomed
Aesthetics

AQUAPURE II

The AQUAPURE II is a multi-functional dermal hydro facial and body care device combined with wet & dry dermabrasion, micro-focused needling and LED hand-pieces into one single system.

AQUAPURE II is equipped with various proprietary solution delivery to infuse exfoliating and antioxidant ingredients into the skin, resulting in improved hydration and skin elasticity with collagen regenerating effects.



AD (AQUA+DIAMOND) PEEL

AD Peel Technology creates channels in the skin to enhance delivery of solutions, further maximising the effects of microdermabrasion. While removing dead skin cells, it cleanses pores and delivers moisturising agents and nutrients to the deeper layers of the skin.



New Upgraded AD Peel Technology


Advanced concept of AD(Aqua+Diamond) peel technology improves skin exfoliation and extraction in a shorter time, thus reducing treatment time.

Precise Solution Control Technology

Utilizes a stable adjustment valve to deliver equal distribution of all solutions being applied during all treatment circumstances.

VACUUM ASSISTED MF (MICRO-FOCUSED) NEEDLE

New transdermal delivery technology with vacuum maximises treatment comfort, efficiency and results. By using a thin needle, nutrients are supplied to the epidermis through microscopic channels and elasticity is improved through epidermal remodelling.

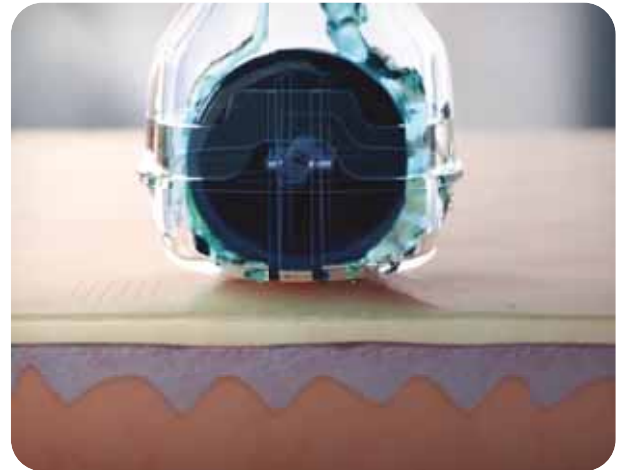

User-friendly Smart Auto Reaction

The Smart Auto Reaction Interface detects and enables automatic recognition of all activated hand pieces.

Higher Suction Power

AQUAPURE II is equipped with dual motor heads to produce better treatment results.


2.5 times stronger suction power than the previous version.

LIGHTPURE LED COLOR THERAPY

The optional LightPure LED helps to boost blood circulation of skin cells, improves the complexion, controls sebum production, and improves inflammation response.

625nm Red

Cell activity; Skin firming, anti-ageing efficacy

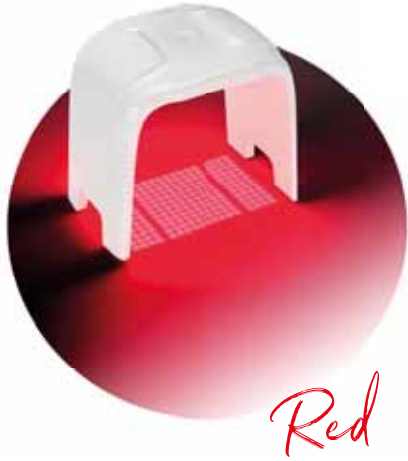


520nm Green

Balance the water and oil



465nm Blue

Bactericidal and anti-inflammatory

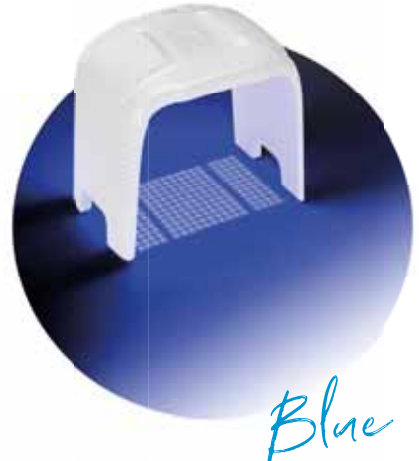

AQUASOLUTION



PEEL+

The first step of treatment is the process of non-irritating skin scaling which facilitates accelerated exfoliation, the removal of dead skin cells and other impurities.



SEBO+

The second step of treatment is for sebum dissolution, treatment of blackheads, and effective pore control. This assists the skin to balance between oily and moisture textures.



REJUVE+

The final step of treatment promotes skin elasticity by inducing collagen and elastin regeneration and improving the inhibition of moisture loss by improving water retention.



CLEANSYS

To provide a convenient cleaning agent specifically made for the needs of the system, CLEANSYS assists in easy maintenance procedures for the AQUAPURE.

CLINICAL CASES



OPTIMIZED APPLICATORS



Micro
Needle



Diamond
Peel



Aqua
Peel



LED Light
Therapy

FREQUENTLY ASKED QUESTIONS

How does the AQUAPURE II WORK?

AQUAPURE II is a multifunctional facial and body care device combining microdermabrasion system with solution delivery, microneedling and LED therapy into one single system for the treatment of various skin concerns.

What concerns are suitable for AQUAPURE II treatments?

Due to the non-invasive nature of AQUAPURE II treatments, almost anyone who desires glowing and healthy skin is an appropriate candidate. Some of the most common skin concerns treated by AQUAPURE II include fine lines, UV-induced skin damage, problematic skin and dehydration by enhancing skin tone and texture on both face and body.

What preparation or downtime is required for AQUAPURE II treatments?

There are no special requirements prior to an AQUAPURE II treatment other than removing makeup and cleansing the skin. Post-treatment, patients should avoid wearing makeup for 24 hours and ensure the application of sunscreen. Saunas and strenuous exercise should be avoided for approximately 1 week.

When do results become apparent?

Immediate benefits are noticed after treatment, with gradual improvements also seen as new collagen is produced. A course of six to eight treatments is recommended, followed by quarterly maintenance treatments.

What advantage does the AQUAPURE II have over other cosmetic procedures?

The multi-functional AQUAPURE II device is equipped with unique hand-pieces which allow operators to provide impressive skin rejuvenation results with individualised treatment plans based on each patient's specific needs. Thus, AQUAPURE II is able to improve and address a wide range of skin types and concerns on both face, neck and body.

AQUAPURE II FUNCTIONS INCLUDE

Skin Rejuvenation
Deep Exfoliation
& Extraction

Skin Texture
Skin Boost
Transdermal Delivery

How long do treatments with AQUAPURE II take?

A complete AQUAPURE II treatment can take as little as 30 to 45 minutes with no downtime.

How can the AQUAPURE II benefit my practice?

The multi-functional AQUAPURE II programs allows practitioners to provide customised treatment plans for a wide demographic of patients without any limitation on skin tone and type. Treatments are reliable with minimal downtime resulting in a high level of consumer satisfaction. With a course of six to eight treatments recommended, practices benefit from frequent patient visits.



SYSTEM SPECIFICATIONS

Input Voltage	100~240V, 50/60Hz
Panel	Touch panel
Dimension	365 x 523 x 460mm (WxDxH)
Weight	14kg