


primelase[®]

feel the new generation of
laser hair removal

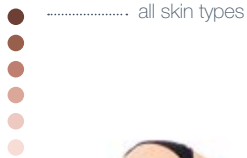

Cryomed
Aesthetics



HAIR REMOVAL TREATMENT

cocoon medical treatments have been designed to make your dream of smooth hairless skin a reality.

Our excellence in innovation and design offers you the best medical aesthetic treatment and results available.

We invite you to discover our exclusive range of treatments and to become one of our thousands of satisfied clients.


www.cocoonmedical.com

Premium hair removal for all different skin types all year round.

primelase HR uses diode laser technology with the greatest peak power available today (4800w). It is characterised by its use of very short pulses with high doses of energy, ideal for residual hair in the final sessions of treatment.

The dynamic or pain free mode is a very attractive characteristic of primelase HR. This system allows us to work without any kind of discomfort, making laser hair removal highly tolerable for all types of patients during the first sessions, from women with fine hair to men with thick hair.

primelase HR uses the "crystal freeze" system in its applicators: this is an exclusive cooling system created by **cocoon medical** which automatically adjusts the temperature, making hair removal much more comfortable.

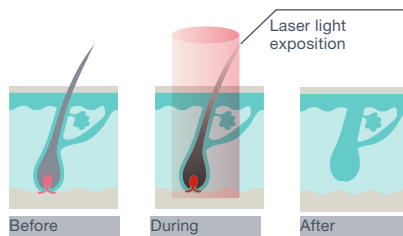

FAQ

What are the benefits of primelase HR laser hair removal?

Permanent hair removal is achieved in a short time, improving the quality of the skin. It also helps to remove cystic hair and improves problems with folliculitis, carbuncles, hypertrichosis and hidradenitis.

How many sessions are required?

The number of treatments depends on your specific skin type and colour, the thickness and density of hair, and the size of the treatment area. On average, between 6 and 10 treatments will result in 85-90% permanent hair reduction.

Is hair removal permanent with primelase HR?

primelase HR progressively eliminates active follicles permanently (hair in anagen growth phase) through successive sessions.

What does the treatment feel like?

primelase treatment is practically painless and easily tolerated. Some patients may feel a short burst of heat as the laser reaches the hair follicles, rather like a tingling or pinching sensation. There may also be some cold sensation from the **primelase** "crystal freeze".

How long does the treatment take?


The duration depends on the size of the area to be treated. An upper lip may take up to 5 minutes, 15 minutes each for underarms, and for legs between 30 and 60 minutes.

Home care advice: before and after treatment

The patient should avoid sunbathing, using self-tanning lotions and colouring the hair in the days prior to treatment. This is to avoid any discolouration of the skin. From the first treatment onwards, avoid the use of tweezers, waxing, electrolysis or any hair removal systems which remove hair by the root.

Do primelase HR treatments have any side effects?

The risks are minimal, and it is unusual for patients to experience side effects. Some patients may experience temporary redness.

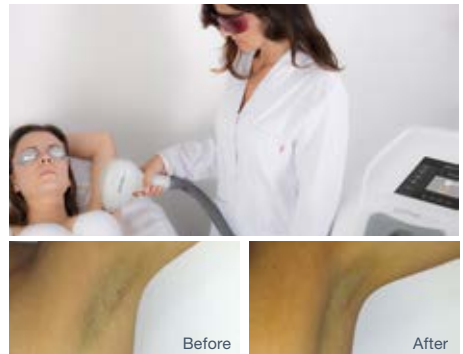

primelase effect



755
nm

Patient data 1:

Phototype II
Applicator: Spot 20x9
755nm
16J/ 3Hz/ 20ms
Mode: Static
Date: 12/02/2016



810
nm

Patient data 2:

Phototype IV
Applicator: Spot 30x9
810nm
8J/ 10Hz/ 3ms
Mode: Dynamic
Date: 19/01/2016



1060
nm

Patient data 3:

Phototype VI
Applicator: Spot 10x9
1060nm
12J/ 7Hz/ 4ms
Mode: Dynamic
Date: 20/11/2015

Cryomed
Aesthetics

www.cryomed.com.au | 1300 346 448

Please consult your medical practitioner to see if you are suitable for treatment. Potential risks and side effects need to be discussed with your medical practitioner before undergoing treatment with primelase.

© Copyright Cryomed 2016

Clinic Details:

057UV1