

A woman with long blonde hair, wearing a bright neon green sleeveless dress and large, colorful sunglasses, stands in a city street. She is holding a large, tan leather handbag. The background features a yellow building with white architectural details and a green car. The overall scene is brightly lit, suggesting a sunny day.

CLATUUTM

Non-invasive Fat Reduction and Body Shaping

Consultation Guide



How does the CLATUU work for you?

Freezing Fat Away is completely non-invasive. You can schedule your CLATUU procedure time into your normal routine. Its sophisticated cooling energy is delivered directly into the targeted fat deposits without disturbing other body tissues.

The CLATUU procedure crystallizes fat cells, which are then eliminated by the body's natural process. Patients experience noticeable and measurable changes in the targeted area.

Natural fat reduction by the CLATUU procedure

No surgery, no anesthesia, no downtime with minimal discomfort.



1 No Surgery No Anesthesia

The CLATUU procedure is complete non-surgical and anesthesia is non necessary and works by applying suction and chilling applicators on stubborn fat deposits.



2 Effective Result!


You will begin to notice a difference approximately 4-6 weeks after your first procedure. The result will continuously be improved for the next 3 month. You may make sure to follow healthy lifestyle with a good exercise program.

3 No Downtime


You can expect to return to your normal routine immediately after your procedure.




Mechanism of Freezing fat away.

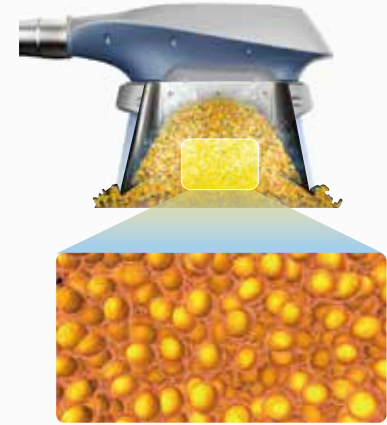


1 Applied target area is suctioned into the handpiece.

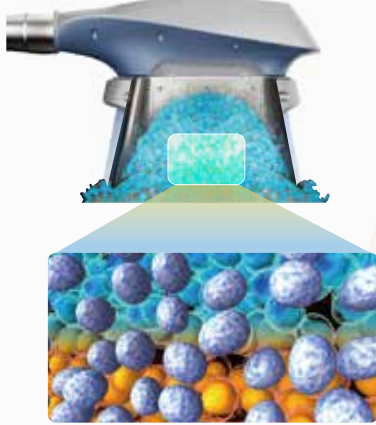


2 Cooling energy treats the fat layer without damaging surrounding tissues.

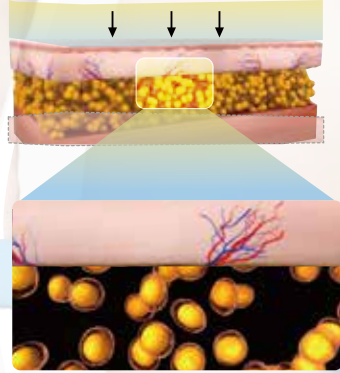


3 Damaged fat cells are broken down and eliminated by natural body process (apoptosis).

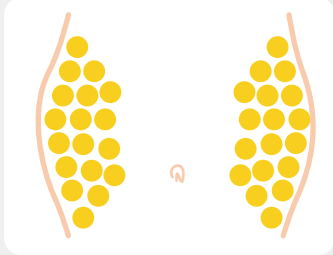


Fat bulges in the abdomen.

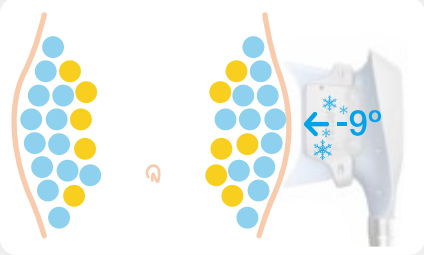

Delivering cool energy

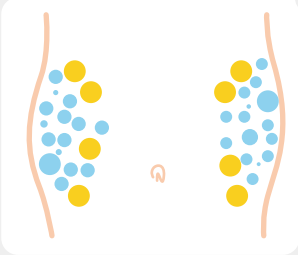


Fat cells destroyed

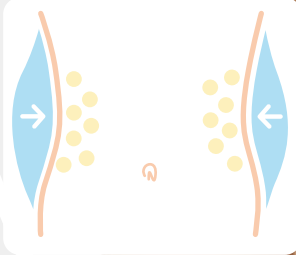

Number of fat cells are decreased in abdomen

Where the CLATUU can be applied?

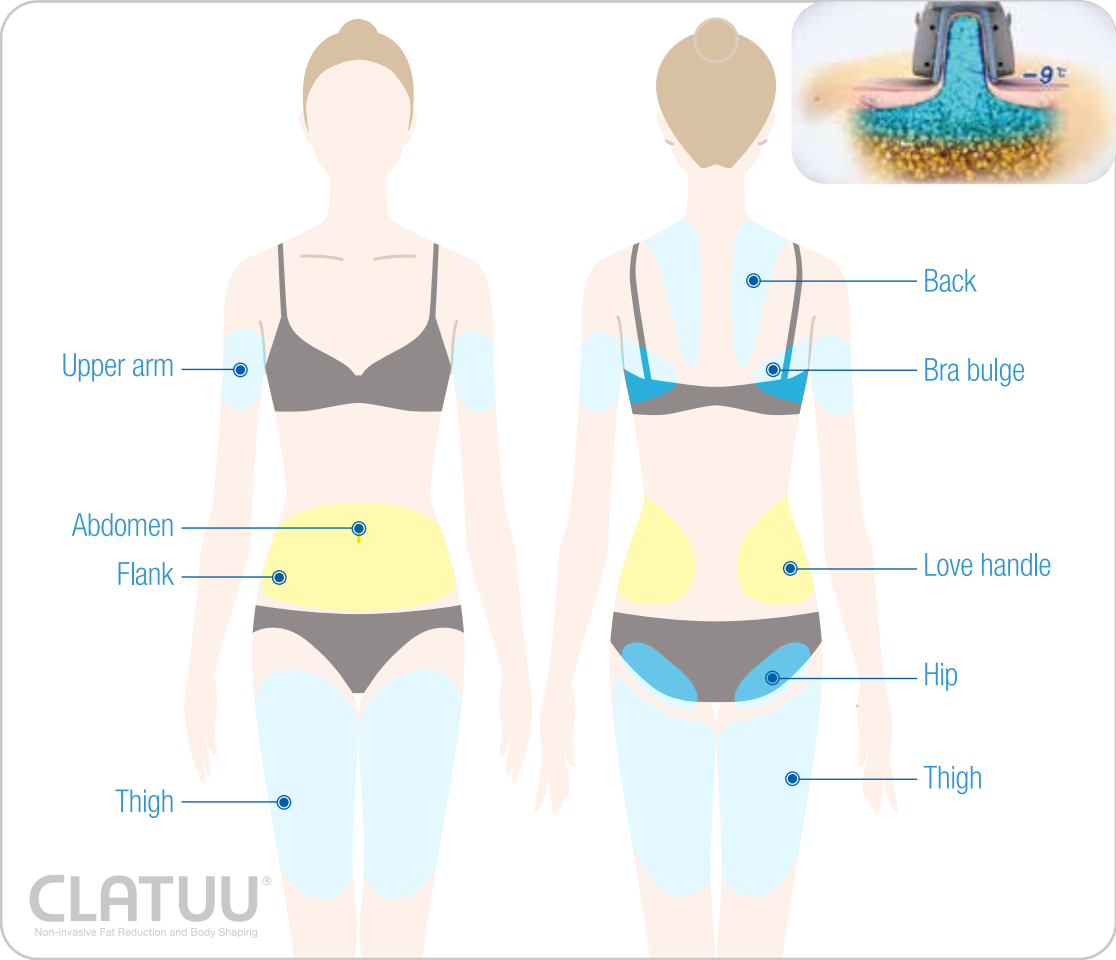

Flat type applicator :

Small areas such as upper arms



Flat type

Wing type applicator :

Big bulges such as belly, back, flanks




Wing type

Prior to the CLATUU procedure



Your practitioner will help you to determine whether the CLATUU is the right treatment for you. The CLATUU could be the right solution for you if you are

in good physical shape but have some problem areas of fat bulges such as muffin tops, upper arm, belly, love handles, back bulges, bra flaps, fat deposits on the thighs and flanks. You may experience a sensation of deep pulling, pinching, tugging or discomfort early in the procedure. This will be replaced by a feeling of cold and possibly numbness as the procedure progresses. Following the procedure, you may experience redness, bruising, swelling, firmness, tingling, some pain or numbness.

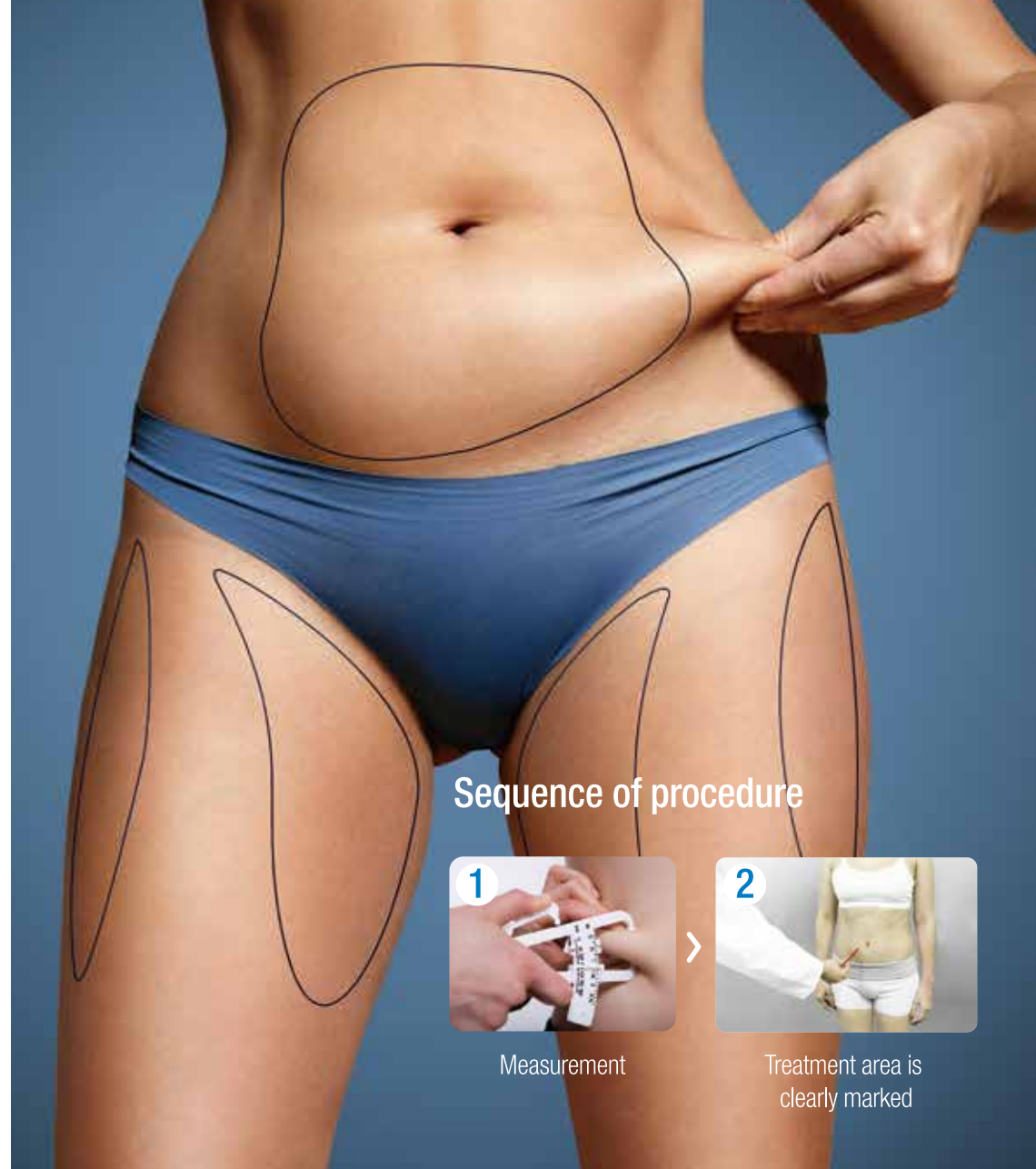

Sequence of procedure



Measurement



Treatment area is clearly marked

During the CLATUU procedure

Your practitioner will measure the area and design the exact position for the procedure. A special matrix gel pad will be applied to protect your skin surface. The handpieces will be placed on the gel pad and then activated. During the treatment, you will be in a comfortable position, so you can rest, read, or use a portable electronic device as you wish. You will probably feel a cold sensation for the first 5-10 minutes. Then, the chill has an anesthetic effect and you will be feeling comfortable.

Sequence of procedure



Placing matrix gel pad to the target area



Applying the applicators



The procedure takes approximately
40-60 minutes




Massaging the area after the procedure



After the CLATUU procedure

The treated area will feel numb when the applicators are removed. Your practitioner will be massaging the area for 5-10 minutes immediately after the procedure. The fatty deposits in the treated area will gradually be reduced over the next few months as shown in pictures below. Exercise on a regular basis would be required to achieve a better and fast result.

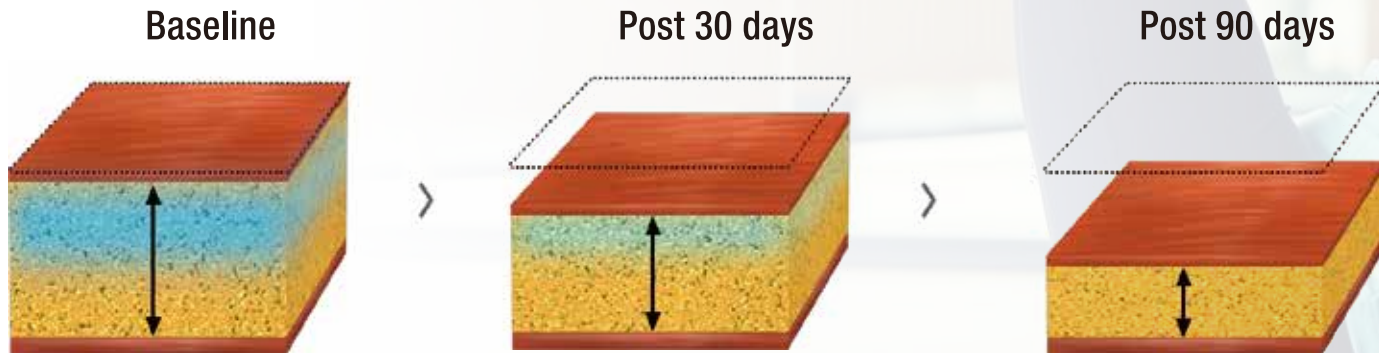


Clinical Cases

Abdoman & Love Handles



Post 90 days **34/Female / -2.5 inches**

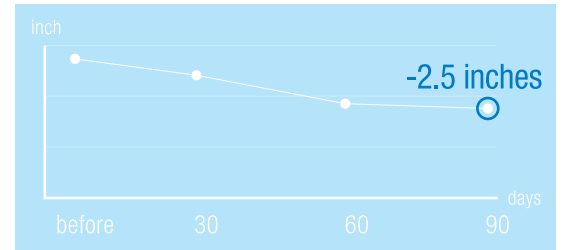

Baseline



Post 45 days




Post 90 days

Clinical Cases

Love Handles



Post 90 days **31/Female / -3.5 inches**

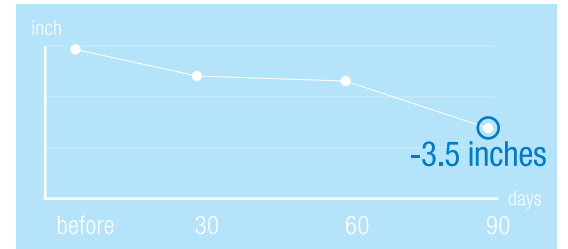

Baseline



Post 45 days




Post 90 days

Clinical Cases

Arms



Post 90 days **26/ Female / -1 inch**

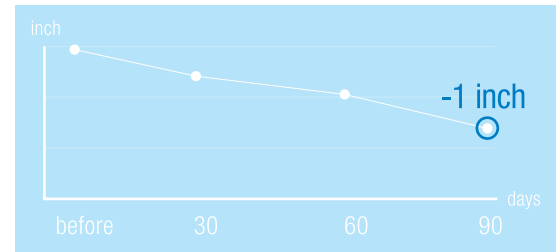


Clinical Cases

Thighs



Post 90 days **26/ Female / -3 inches**

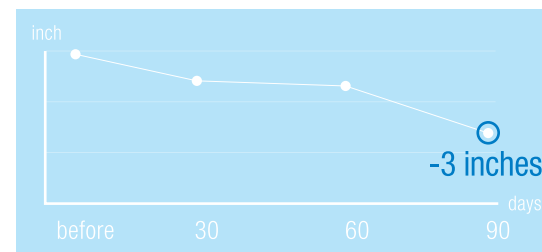

Baseline



Post 45 days



Post 90 days

FAQ

Q. Is CLATUU treatment painful?

A. When the treatment starts, you will feel your skin being pulled into the handpiece. For the first 5-10 minutes, as the cooling process takes effect, you will feel a cold sensation localized in the area. The area being treated will feel numb for the rest of the treatment. You may feel some soreness about 10 minutes after the end of the procedure.

Q. Is CLATUU treatment safe?

A. The CLATUU treatment has been through countless hours of testing. It demonstrates the cooling damages affects to the fat cells. Other tissues at the treatment site are very rarely affected.

Q. Who is a good candidate for the CLATUU treatment?

A. Both men and women are suitable for the CLATUU treatment. The ideal candidate is someone who is in good physical shape but has some fat bulges such as muffin tops, upper arm, belly, back, bra bulges, thigh and flank fat deposits and other isolated fatty areas. The CLATUU is not a weight-loss treatment and is a treatment for shaping and contouring of your body.

Q. How long does the CLATUU treatment take?

A. The CLATUU treatment takes 40-60 minutes. The length of treatment depends on the thickness of the fat in the targeted area. You can relax, read or use your mobile devices during the treatment.

Q. What happens after the treatment?

A. You may notice some redness on the treated area, but this will disappear within less than an hour or so. After the treatment, the cooled fat cells begin the process of apoptosis and start to swell and die. These damaged fat cells are slowly metabolized by the body over a period of 6 to 12 weeks.



FAQ

Q. **When will I see the results of the CLATUU procedure?**

A. The CLATUU procedure works by freezing fat cells to the point and induce apoptosis to reduce the stubborn fat cells gradually. So, it will take about a month to see the early results and progressive improvement may continue for up to three months.

Q. **What reduction in size can I expect?**

A. According to our clinical studies, one session with one applicator reduces the size of inch in the area. The CLATUU has 2 applicators, so you can expect to reduce 2 inches of stubborn fat in 3month from the first CLATUU procedure.

Q. **How long will it take to recover from the CLATUU procedure?**

A. The CLATUU procedure is a complete non-invasive and non-surgical procedure, so in most cases patients can resume to their daily routine after the CLATUU procedure.

Q. **How often do I need to take the procedure?**

A. The number of treatments required depends on the thickness of the fat. The CLATUU procedure can be repeated on the same area at 90days of interval. It is important to stress that a proper diet and some exercise after the procedure is always helpful in achieving the best results.



News & Magazines

The CLATUU procedure is CE and KFDA cleared as well as the other registrations required in the countries. The CLATUU is available more than 50 countries worldwide.



Discover CLATUU Freezing Fat Away

The non-surgical body contouring treatment that uses controlled cooling to eliminate stubborn fat deposits. Find out how it works. See the results.

The CLATUU is available worldwide. For more information, please visit our website or email to info@classys.com www.classys.com | www.clatuu.com | www.ultraformer.com | www.facebook.com/classysglobal
Clatuu name and logo are registered trade mark of Classys Inc. Copyright © 2015. All rights reserved.

CLATUU™
Non-invasive Fat Reduction and Body Shaping