

FILMED
LABORATOIRES

The Art of Regeneration

HYALURONIC ACID DERMAL FILLER



This brochure is designed for those patients who have been prescribed
ART FILLER® by their healthcare practitioner.


THE ART OF *Ageing*

The aging process is natural, and as we age, our skin undergoes visible changes in texture, volume, and overall health.

Elastin and collagen fibres provide the skin with its suppleness and strength. However, as we get older, the production of these fibres and natural dermal moisturisers like Hyaluronic Acid (HA) diminishes.

Consequently, fine lines and wrinkles begin to appear, replacing the youthful, smooth, and radiant complexion we once had.




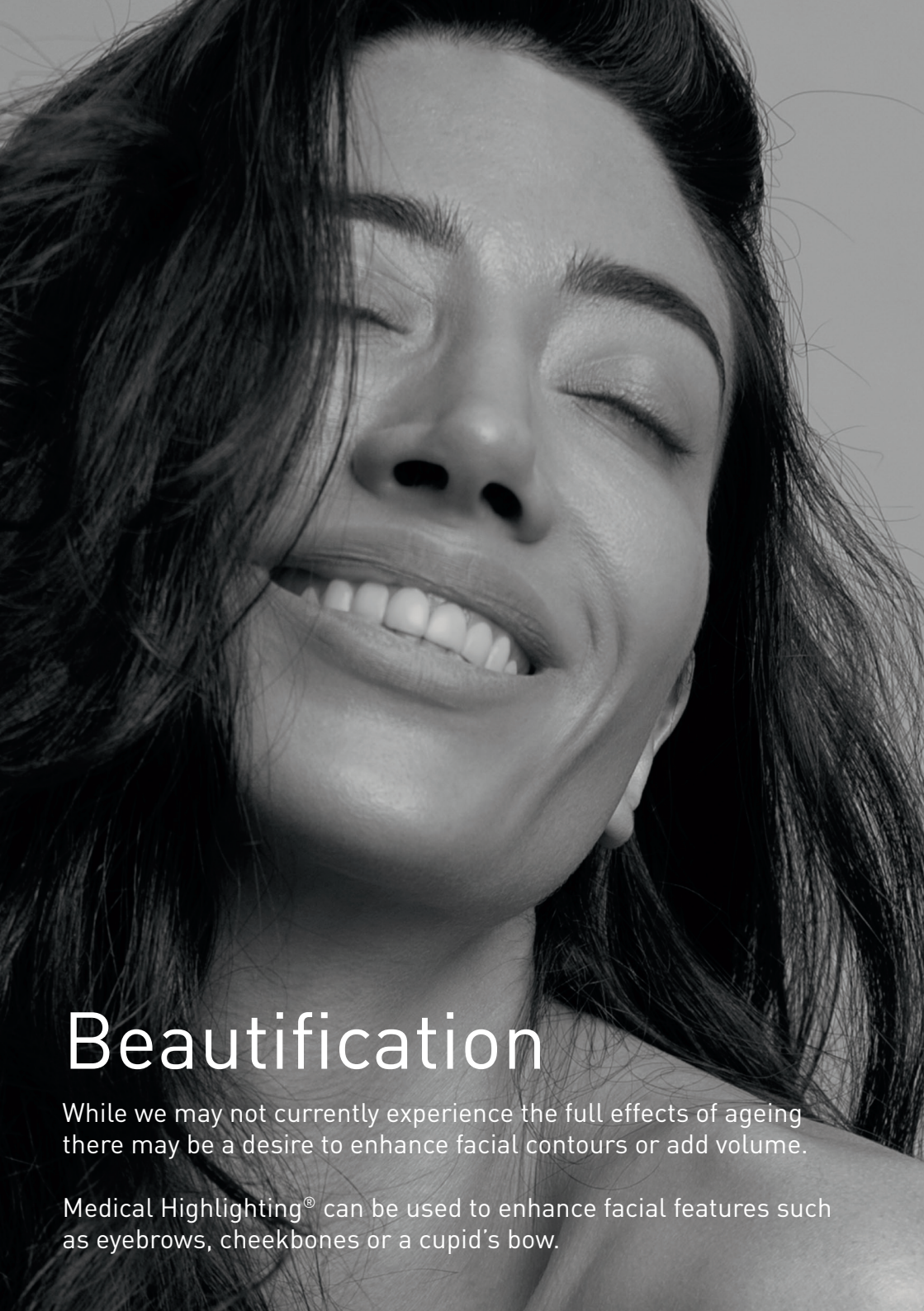

Beautification

While we may not currently experience the full effects of ageing there may be a desire to enhance facial contours or add volume.

Medical Highlighting® can be used to enhance facial features such as eyebrows, cheekbones or a cupid's bow.

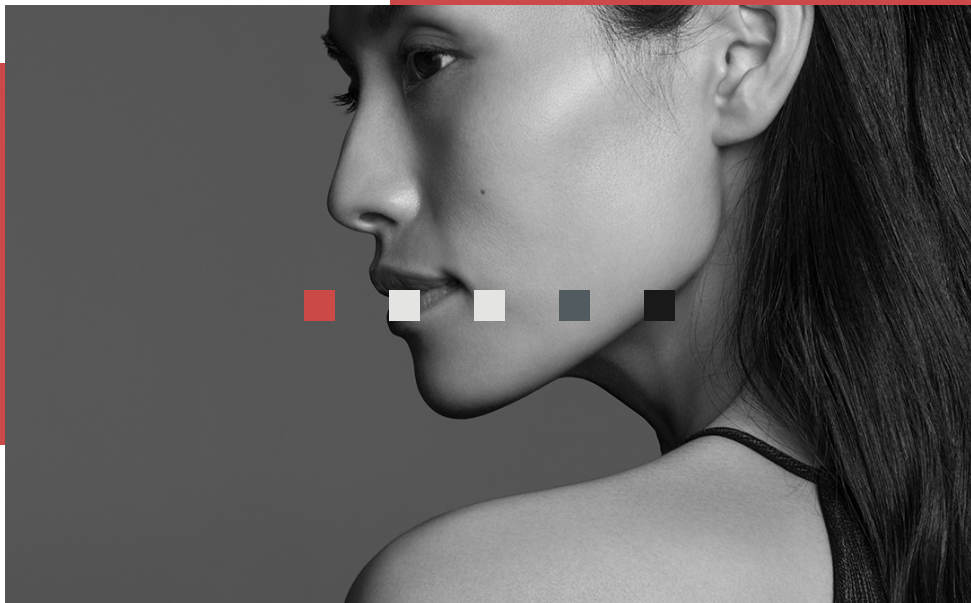

the art of REGENERATION

You may have already started to notice the effects of ageing globally which involves a combination of three factors:

- **Skin dehydration** – loss of elasticity, firmness and radiance
- **Loss of facial fat** – resulting in sagging of the skin and volume loss
- **Changes in bone structure** – changes to facial contours

FILLMED recognises the desire for a rejuvenated and natural look that restores those youthful features.

Hyaluronic Acid (HA) is a natural component of the skin that functions like a sponge, retaining water to maintain softness and firmness. This helps enhance skin volume and elasticity.

Products containing Hyaluronic Acid can effectively address various signs of facial aging by replenishing the natural Hyaluronic Acid that has been depleted over time.

Full Face *Rejuvenation*

During a full-face rejuvenation consultation your concerns are addressed simultaneously to create a customised plan.

This type of holistic treatment plan often involves the use of various products, ranging from softer dermal fillers to smooth fine lines, to more structural dermal fillers that restore volume loss and improve facial contours.



Wrinkle Management

Hyaluronic Acid (HA) can be used to address and soften wrinkles, creases, and other areas that may benefit from enhancement, like the nasolabial folds and lips.

Area *Specific*



LIPS

Restore volume, hydration, structure, and balance to the lip and lip area.



CHEEKS

Restore volume to the midface and define the natural cheekbone structure.



EYES

Smoothing the eye area between the lower eyelid and the cheeks.



JAWLINE

Create a strong jaw angle, augment a weak chin, or smooth jowls.

ART FILLER®

ART FILLER® is a Hyaluronic Acid Dermal Filler brought to you by FILLMED Laboratories, experts in anti-ageing since 1978. The French manufacturing company specialises in developing aesthetic and cosmetic treatment solutions using Hyaluronic Acid and high-performance active ingredients.

ART FILLER® contains an innovative Tri-Hyal® technology that is designed for smoothing, sculpting and adding volume.



FINE LINES

SOFT & SMOOTHING

Fine and superficial wrinkles



UNIVERSAL

FILLING & SCULPTING

Medium to deep wrinkles



VOLUME

VOLUMISING & SHAPING

Volume creation & restoration



LIPS SOFT

SCULPTING & SMOOTHING


Perioral wrinkles and soft volume



LIPS

PLUMPING & DEFINING

Lip volume

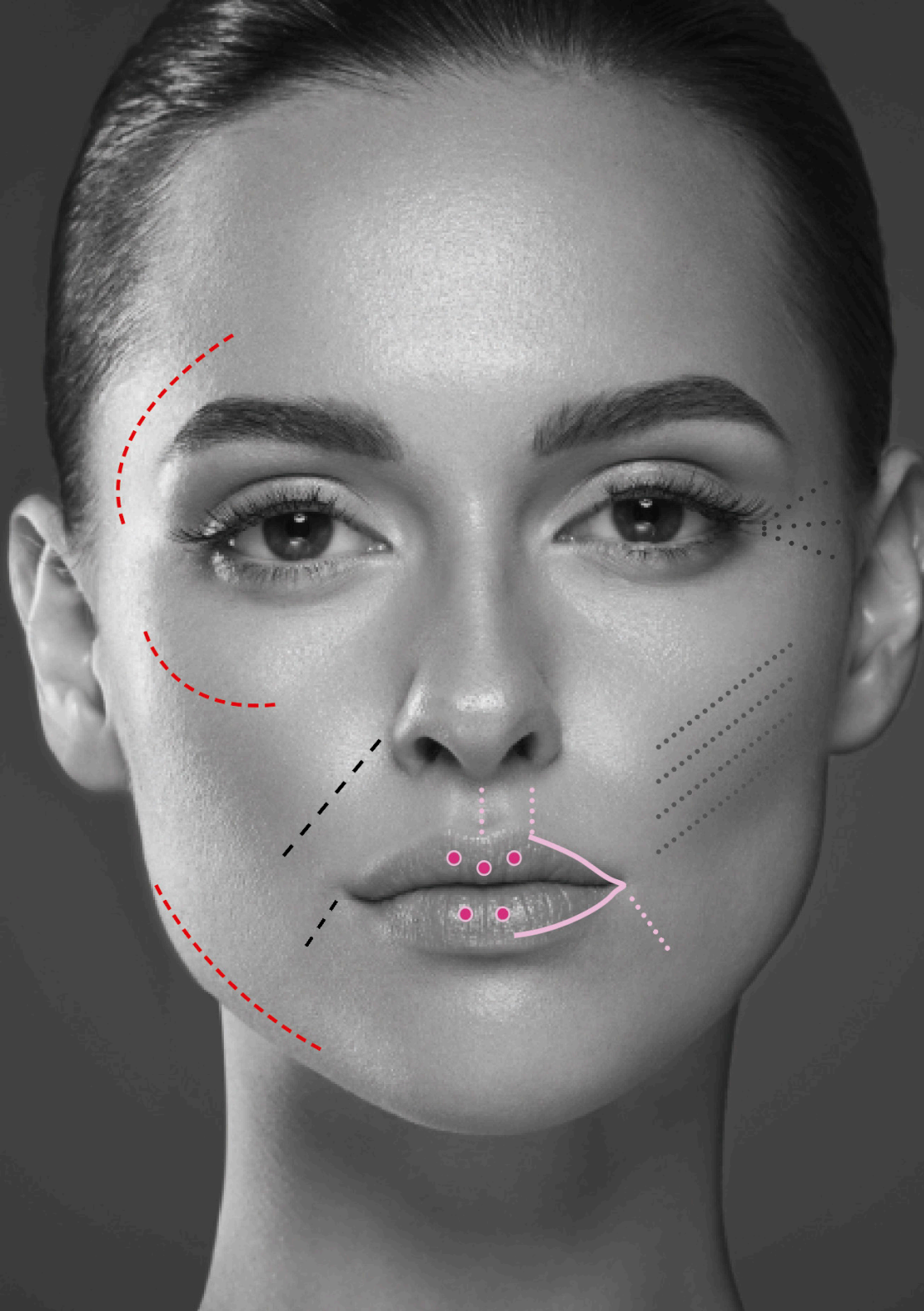

MAINTAIN YOUR *Results*

Skincare designed to complement aesthetic procedures.

SKIN PERFUSION is an innovative high-grade cosmeceutical range especially designed to maintain and enhance the results of cosmetic procedures.

It consists of a comprehensive range of cleansers, serums, creams and boosters to support a 360° skin rejuvenation approach.

Ask your Healthcare Practitioner about which **SKIN PERFUSION** products are right for you.



Frequently Asked Questions

What happens during treatment?

Once you are happy to proceed with your treatment and provide consent, your healthcare practitioner will ask you questions about your medical history, current medications and any other relevant medical information.

During the treatment session, ART FILLER® is injected into the specific area of the skin, using a very fine needle or cannula.

How long does the treatment take?

The length of treatment is unique to the individual and will depend on the area being treated. Your healthcare practitioner will be able to answer your specific treatment needs in more detail.

Does it hurt?

The treatment is relatively painless for some, but others may experience mild discomfort or pain. ART FILLER® contains lidocaine to reduce the sensation of pain you may feel during the injection to maximise comfort. For additional pain management options, please talk to your healthcare professional.

What is the downtime?

Some patients experience swelling, bruising or redness after treatment which usually subsides within 1-4 days. Some return to work immediately after treatment. We recommend talking to your healthcare practitioner about specific downtime questions.

How soon will I notice the results?

HA Dermal Filler results are visible immediately after treatment. However, often the area can experience swelling which will subside over a matter of hours to days.

How often do I need treatment?

Treatment results from ART FILLER® are not permanent. HA Dermal filler duration depends on many factors, such as the type of treatment, the treated area, the individual's skin type, skin condition lifestyle and age.

After the initial treatment, follow-up sessions are normally recommended to maintain results. Your healthcare practitioner will determine how frequently treatment is required.

What are the side effects?

ART FILLER® is composed of hyaluronic acid a component naturally found in the skin. Possible injection-related reactions may include slight redness, swelling, itching, bruising, and pain at the injection site. These reactions are normal, usually mild to moderate, and tend to fade shortly after the injection, resolving within a week. Before treatment, inform your practitioner about any allergies, particularly to lidocaine. Your practitioner can address any specific inquiries you may have regarding your treatment.



This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.





BEAUTY IS AN ART,
WE MADE IT A SCIENCE

FILLMED
LABORATOIRES

Powered by Cryomed Aesthetics | 1300 346 448 | info@fillmed.com.au | fillmed.com.au

ART FILLER® is a range of injectable medical devices. The ARTFiller range is intended to restore changes to skin structure caused by ageing: fills in lines and creases in the facial skin, restores volume. The Fillers contains Lidocaine to reduce any painful sensations for the patient during treatment. For full intended purpose and safety information, please refer to the individual

VOLUME, UNIVERSAL, LIPS, LIPS SOFT AND FINE LINES Instructions for Use.

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE.

THIS PRODUCT IS NOT AVAILABLE FOR PURCHASE BY THE GENERAL PUBLIC.

KEEP OUT OF REACH OF CHILDREN

Vigilance contact (notification of an adverse event): clinical@cryomed.com.au