


RESULTS YOU CAN SEE

Results shown are for this individual patient only. Your results may differ.

BEFORE

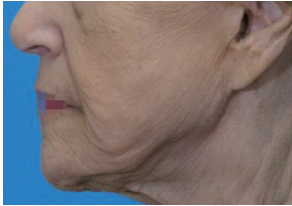

Courtesy of D Man, MD, USA

AFTER



Courtesy of SB Cho, MD, South Korea



Courtesy of Melanie Palm, MD, USA



Courtesy of Jill Waibel, MD, USA



Courtesy of Mark Rubin, MD, USA



{ ANSWERS TO YOUR questions }

Is the procedure Painful?

Everyone experiences pain or discomfort to varying degrees. Your doctor will determine how to minimize your discomfort during the procedure. Topical anesthetics are applied and sometimes oral medications are prescribed. Administering nerve blocks is another option to help minimise discomfort.

When will i see results?

Within a few days, new skin will appear that is smoother and more even-toned. Your skin will continue to improve over six to nine months as the new collagen builds replacing the old collagen.

Is the procedure safe?

Fractional treatments are approaching a decade of popularity with physicians and patients worldwide. The eCO2 laser features precision technology designed to support your doctor in delivering treatment.

Ask your practitioner if eCO2 is suitable for you.

Follow us:

 www.cynosurelutronicanz

 [cynosurelutronicanz](https://www.instagram.com/cynosurelutronicanz)

 info@cryomed.com.au

LUTRONIC
Intelligent Care™

Distributed in Australia by Cryomed Aesthetics.
©2021 LUTRONIC. ALL RIGHTS RESERVED. LUTRONIC, Lutronic Intelligent Care, Clarity, and associated logos are service marks, trademarks, and/or registered trademarks of Lutronic Corp. and/or its subsidiaries in the USA and/or other countries.

PAT_CL_eCO2Plus_Skin_Version1_07_2026_exp24months

REVITALISE YOUR SKIN




ASK FOR eCO2

HOW IT WORKS

Fractional resurfacing appeals to a broad range of patients, from those needing to repair light damage to those wanting more dramatic results.

The eCO2 treatment may help reduce the appearance of skin imperfections and support collagen regeneration. Results vary depending on individual factors.

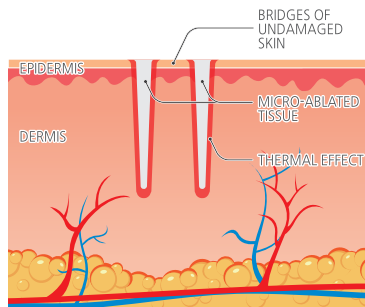

Laser energy micro-ablates tissue leaving small micro-wounds bridged by healthy tissue. Your body's natural healing process stimulates the creation of new tissue and collagen which can help improve skin texture and tone




THE HEALING PROCESS

Immediately following treatment you will experience some swelling and redness. Depending on the treatment parameters used by the doctor the area treated may appear red for a longer period of time. In most cases, this issue resolves in one to three weeks. During that time, with the exception of the first few days post-treatment, you may apply makeup to tone down the redness. Normal activities may be resumed, in most cases, within 5-7 days.


IMMEDIATELY AFTER

The laser creates micro-ablated columns to stimulate cells. The surrounding healthy tissue promotes rapid healing.

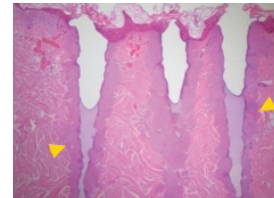


2 DAYS POST

Skin is healing creating new collagen during wound healing process.

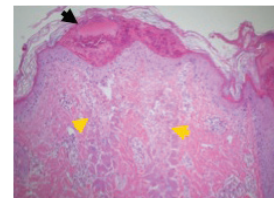

POST 14 DAYS

Outer layer of skin is completely healed and collagen remodeling continues for up to nine months.

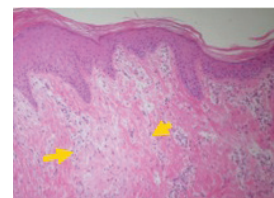

EXPECT RESULTS

Your physician will discuss your treatment needs with you during your consult. The results will vary depending on your age, degree of sun damage and skin type, as well as what parameters your physician uses. Patients have reported improvements following treatment. Individual outcomes vary based on age, skin type, sun damage and treatment parameters.

- Reduces facial fine lines and wrinkles
- Gives the skin a more firm, smooth appearance
- Improves scars
- Reduces sun damage, such as brown spots
- Reduces skin blotches and blends skin color
- Improves appearance of pigmented and vascular conditions

