

# REFORM YOUR YOUTH

Lifting | Tightening | Contouring

## ULTRAFORMER III

Non-Surgical System For Lifting, Tightening & Contouring

Micro & Macro Focused | Fast & Precise Shot Application  
Multi-functional & Multi-depth | Highly Effective

**CONSULTATION GUIDE**

## Why Ultraformer III?

The Ultraformer III is a non-invasive, face lifting and tightening, body tightening and contouring treatment that provides a customised treatment procedure for the entire face, neck, and body.

Utilising HIFU technology with Micro and Macro Focused cartridges, the Ultraformer addresses signs of ageing to restore youth and confidence to patients around the world.

The Ultraformer III has also been approved to the standard of international regulatory bodies and associations which include Australia, Brazil, South Korea and Europe.

The Ultraformer III maintains a firm and growing presence in clinics across more than 55 countries.



## The Ultraformer Way

The Ultraformer III is easy to incorporate into clinics and may be used in conjunction with other treatment modalities. With no downtime, anesthetics, incisions, or side effects that affect your daily routine, the Ultraformer III treatment is the perfect “lunchtime” procedure.

### **The Ultraformer III provides:**

- Quick and efficient treatments
- Micro & Macro focused cartridges
- Treatments for a wide demographic
- Solutions to address signs of ageing

**No Side Effects**

**No Surgery**

**No Scars**


**No Downtime**

**Immediate Results**



# Skin Lifting for Age Shifting. Ultraformer III. Ultra-focused. Ultra-quick.


Comparable to reflecting sunlight through a magnifying glass, the Ultraformer III delivers ultrasound energy into the target region of the skin to stimulate collagen regeneration. The precision in delivery of a high peak power triggers the process of neo-collagenesis naturally within the body.

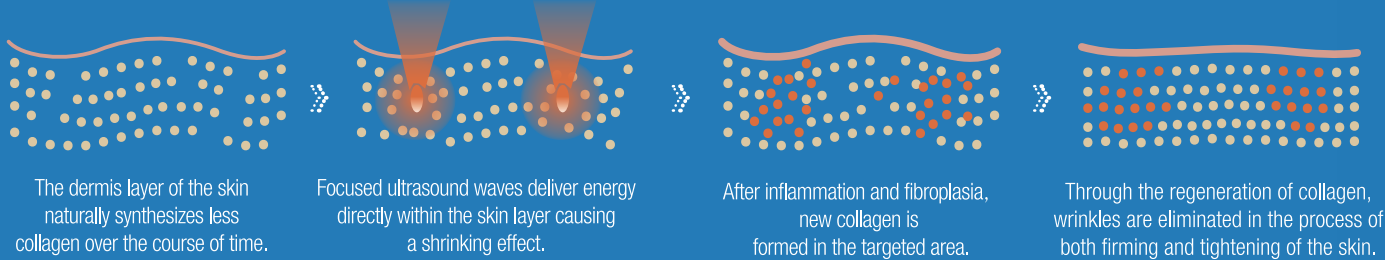


# Eliminate stubborn fat cells in the easiest way possible. Non-invasive, no cutting, no stitches, no downtime.



Stubborn fat cells are difficult to burn off even through strenuous exercises and extensive weight loss regimens. The Ultraformer III treatment provides both non-invasive and innovative solutions without including side effects or damage to skin.

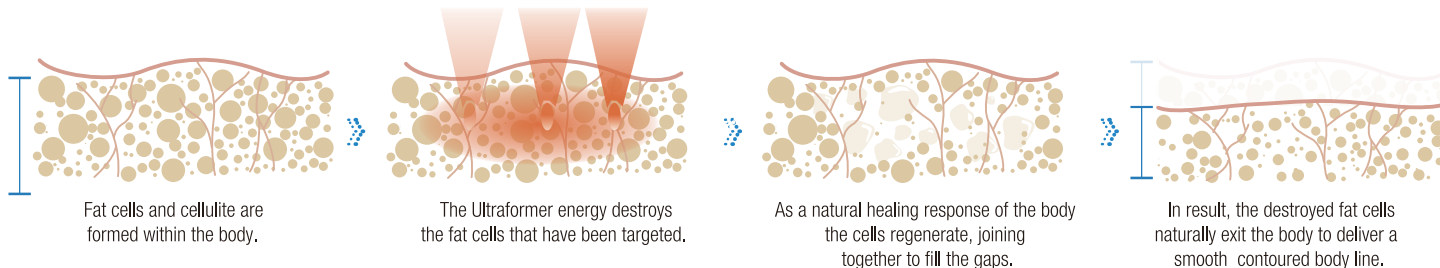



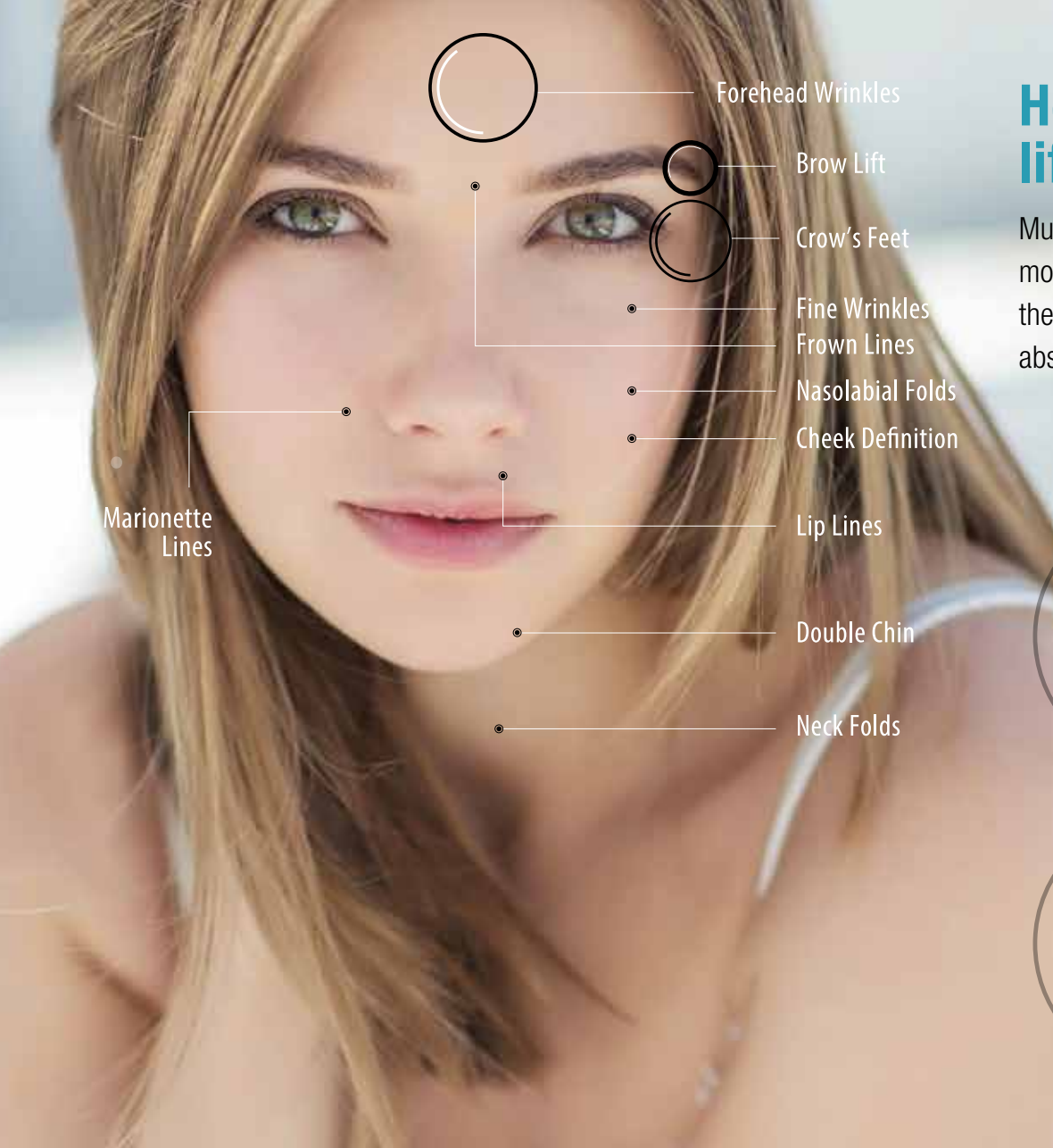

Forehead Wrinkles

Brow Lift

Crow's Feet

Fine Wrinkles

Frown Lines

Nasolabial Folds

Cheek Definition

Lip Lines

Double Chin

Neck Folds

Marionette Lines

## How does the Ultraformer III lift & tighten the face?

Multi-depth cartridges lift and tighten the face for a more youthful complexion. From the forehead down to the creases on your neck, the Ultraformer III provides the absolute solution to reduce signs of ageing.

### MICRO-FOCUSED CARTRIDGES FOR SKIN TIGHTENING



1.5 mm



2.0 mm



3.0 mm





4.5 mm

# How does the Ultraformer III tighten & contour the body?

Macro-focused depth cartridges damage fat cell membranes consisting of triglycerides and are then transported through the vascular and lymphatic systems to the liver. This process is made possible through the body's natural metabolism as the liver makes no distinction between fat disposed of by the Ultraformer III and fat originating from food consumption.

## MICRO-FOCUSED CARTRIDGES FOR BODY CONTOURING

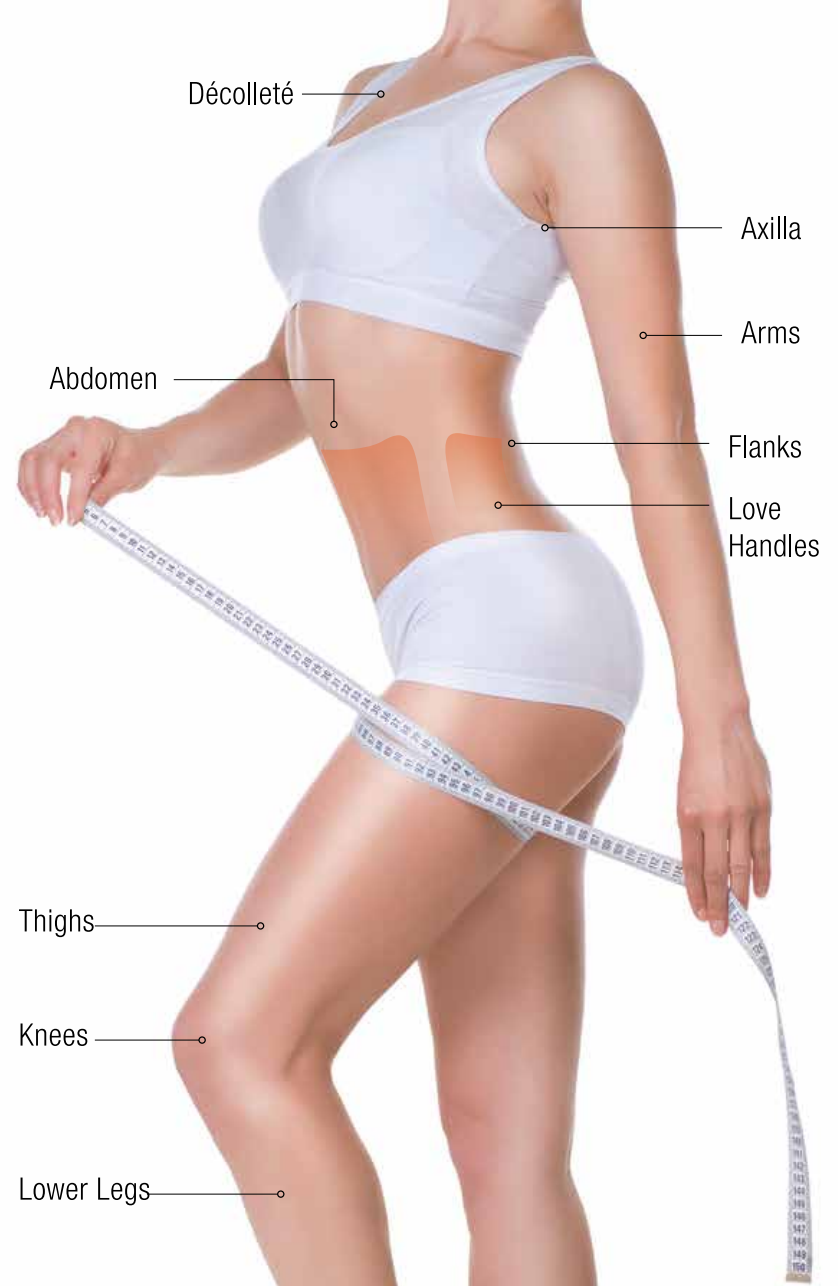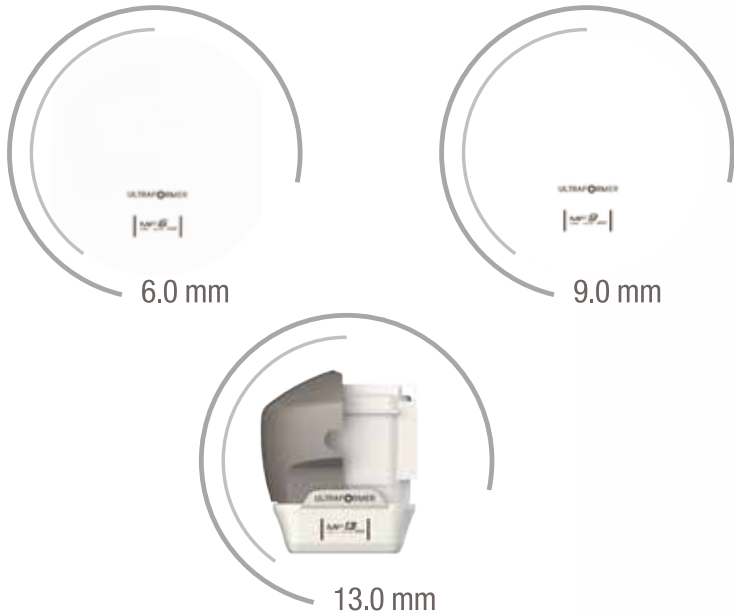

## Before treatment

- Consult with your doctor or practitioner to assess whether you are suitable for treatment
- Consult with your doctor if you are currently taking medication
- If you have had prior treatment within the desired application area, consult with your practitioner or doctor  
e.g fillers, botulinum toxins, or laser procedures



FACE

## ON THE DAY of treatment

- Wear comfortable clothing
- Remove make up
- Face treatment procedures vary between 20-50 minutes based on the targeted areas







**BODY**

## **ON THE DAY of treatment**

- Wear comfortable clothing
- Body treatment procedures vary between 30-60 minutes based on the targeted areas

## After Treatment

- No downtime necessary
- Makeup is applicable immediately following treatment
- The treated area may appear slightly red or pink. This will resolve within a few hours post treatment
- There are no post-procedure limitations on activity



# Clinical Cases

## Wrinkle Reduction & Cheek Definition



Baseline




Immediately Post Procedure

### Procedure Information

Patient info Female / 31

#### Application

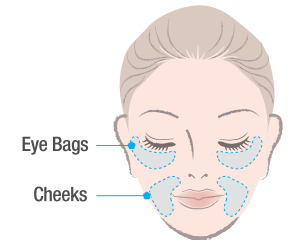

#### Result

Wrinkles visibly smoothed out, effective immediately after procedure.

#### Patient Comment

My fine lines were immediately less visible and it was such a simple procedure.

# Clinical Cases

## Wrinkle Reduction & Cheek Definition



Baseline




Immediately Post Procedure

### Procedure Information

Patient info Female / 50

#### Application

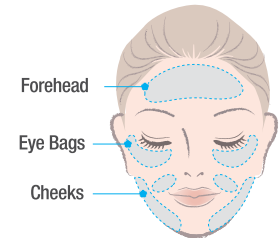

#### Result

Wrinkle lift visible.  
Fine lines less defined.  
Cheeks show a defined facial structure.

#### Patient Comment

There is more definition in my facial structure. I look younger already and I look forward to see what happens in the coming months!



# Clinical Cases


## Brow Lift



### Procedure Information

Patient info Female / 50

#### Application

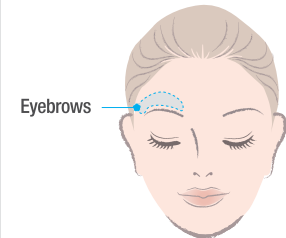

#### Result

Uneven eyebrow treated.  
Visible improvement, evening out  
eyebrow positioning.

#### Patient Comment

My eyebrows were uneven before  
the treatment but immediately  
afterwards I can see the outstanding  
difference.

# Clinical Cases

## Double Chin



Baseline




9 Months Post Procedure

### Procedure Information

Patient info Female / 42

#### Application

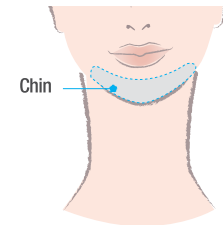

#### Result

Definition in the V-line of the chin is visible. Sagging form of the chin has been lifted.

#### Patient Comment

The sagging fat on my chin has tucked itself in. I can't wait to see the progressive improvement which is to come.

# Clinical Cases

## Neck Folds



Baseline




3 Months Post Procedure

### Procedure Information

Patient info Female / 50

#### Application

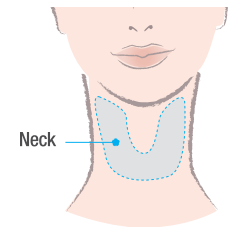

#### Result

Turkey neck has been stretched out. Exhibits a substantial improvement in wrinkle formation.

#### Patient Comment


My severe condition of turkey neck has disappeared. The results exceed my expectations immensely.

# Clinical Cases

## Axillary Folds



Baseline

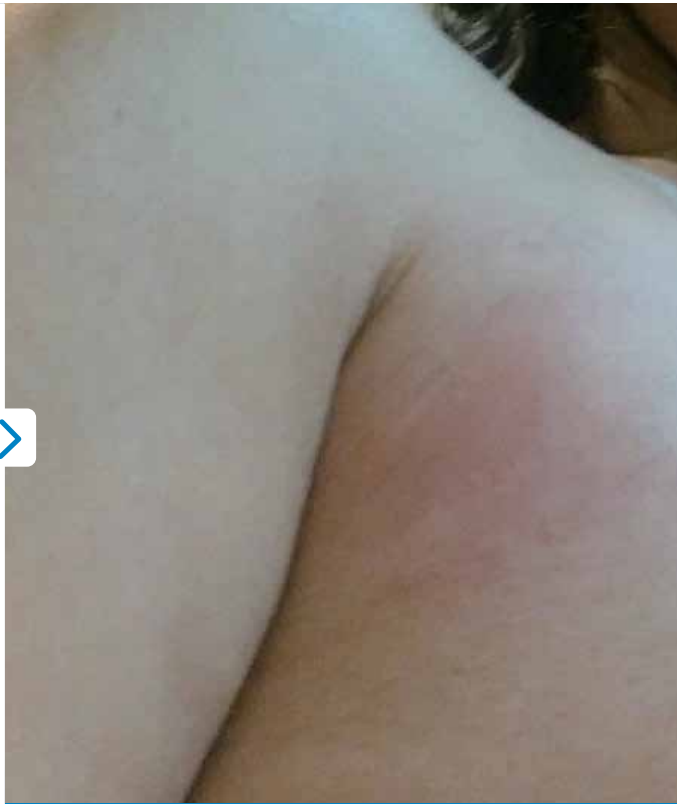


Immediately Post Procedure

### Procedure Information

Patient info Female / 30

#### Application

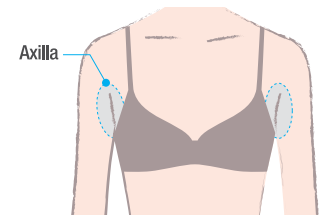

#### Result

Overlap of fat is not visible.  
Good outcome.

#### Patient Comment

I can't believe the changes  
that I'm seeing.  
My rolls are not visible already?  
The Ultraformer does not disappoint.



# Clinical Cases

## Sagging Skin




Baseline

3 Months Post Procedure

### Procedure Information

Patient info Female / 53

#### Application

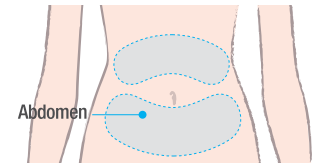

#### Result

Reduction of sagging skin.  
General lifting of sagging skin.  
Post 3 months of procedure.

#### Patient Comment

I didn't know the length and efficacy of results but after three months the results are outstanding.

## FAQ

### What are the advantages of the Ultraformer III?

The Ultraformer III utilises cartridges focused on multiple depths of skin for lifting and tightening the face as well as tightening body tissues for contouring purposes. The patient can receive customised treatments on the face and body with the stable delivery of ultrasound energy, fast application, and precise accuracy of Micro & Macro Focussed Ultra Sound powered transducers.

### How is fat naturally removed from the body by the Ultraformer III?

Macro-focused depth cartridges damage fat cell membranes consisting of triglycerides and are then transported through the vascular and lymphatic systems to the liver. This process is made possible through the body's natural metabolism as the liver makes no distinction between fat disposed of by the Ultraformer III and fat originating from food consumption.

### Are there any side effects?

Although both desired and competitive results for facial lifting and tightening and body contouring will likely show over the course of 3 months post treatment, patients may still experience slight redness or swelling for up to a few hours in the areas treated as well as temporary bruising which gradually subsides within 2-4 weeks.



## FAQ

### **Is there any discomfort during treatment?**

Many patients and practitioners report that in comparison with other devices, the Ultraformer III delivers nothing more than a mild discomfort.

### **Am I a candidate for the Ultraformer III?**

Patients with diverse skin conditions such as fine lines, wrinkles, and hyperpigmentation on the face, or stubborn belly fat and love handles desiring a more curvaceous body figure, serve as ideal candidates for the Ultraformer III.

### **How long do the results last?**

Patients may notice immediate results with progressive improvement up to 90 days post-treatment. These results can be maintained from several months to a year afterwards.



# News & Magazines





**Cryomed**  
Aesthetics

1300 346 448  
cryomed.com.au

For more information visit us  
[cosmeticme.com.au](http://cosmeticme.com.au)  
For further enquiries email us  
[info@cryomed.com.au](mailto:info@cryomed.com.au)

**Reform your youth with a lunchtime procedure. Non-surgical Lifting, Tightening & Contouring System.**

Classys products are available worldwide. For more information visit our website or email us at [info@classys.com](mailto:info@classys.com)  
The Ultraformer name and logo are registered trademarks of Classys Inc. Copyright© 2019. All rights reserved.

**ULTRAFORMER III**