


RESULTS YOU CAN SEE

{ ANSWERS TO YOUR questions }

REDISCOVER
YOUR YOUTH

BEFORE

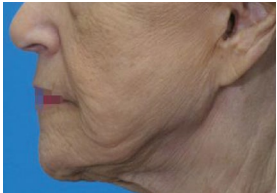

AFTER



Courtesy of D Man, MD, USA



Courtesy of SB Cho, MD, South Korea



Courtesy of Melanie Palm, MD, USA



Courtesy of Jill Waibel, MD, USA



Courtesy of Mark Rubin, MD, USA

Is the procedure Painful?

Everyone experiences pain or discomfort to varying degrees. Your doctor will determine how to minimize your discomfort during the procedure. Topical anesthetics are applied and sometimes oral medications are prescribed. Administering nerve blocks is another option that completely numbs the area being treated.

When will i see results?

Within a few days, new skin will appear that is smoother and more even-toned. Your skin will continue to improve over six to nine months as the new collagen builds replacing the old collagen.

Is the procedure safe?

Fractional treatments are approaching a decade of popularity with physicians and consumers worldwide. The eCO2 laser is designed with the finest operating features possible to provide you and your doctor the greatest margin of safety during the procedure.

SEE ECO2 FOR YOURSELF!

LUTRONIC
Intelligent Care™


©2021 LUTRONIC. ALL RIGHTS RESERVED. LUTRONIC, Lutronic Intelligent Care, Clarity, and associated logos are service marks, trademarks, and/or registered trademarks of Lutronic Corp. and/or its subsidiaries in the USA and/or other countries.
PAT_ECO2_BRF_v2_INTL

ASK FOR **eCO2**

HOW IT WORKS

Fractional resurfacing appeals to a broad range of patients, from those needing to repair light damage to those wanting more dramatic results.

The procedure lives up to its promise to reduce skin imperfections and promote collagen regeneration for long term wrinkle reduction, reclaiming the skin's youthful appearance, tone and color.

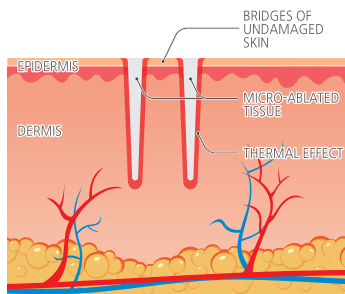

Laser energy micro-ablates tissue leaving small micro-wounds bridged by healthy tissue. Your body's natural healing process stimulates the creation of new tissue and collagen for a younger more youthful appearance of the skin.




THE HEALING PROCESS

Immediately following treatment you will experience some swelling and redness. If your doctor is aggressive during the procedure, the area treated will appear red for a longer period of time. In most cases, this issue resolves in one to three weeks. During that time, with the exception of the first few days post-treatment, you may apply makeup to tone down the redness. Normal activities may be resumed, in most cases, within 5-7 days.


IMMEDIATELY AFTER

The laser creates micro-ablated columns to stimulate cells. The surrounding healthy tissue promotes rapid healing.

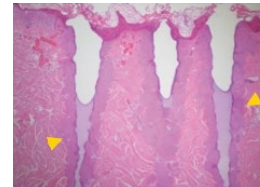


2 DAYS POST

Skin is healing creating new collagen during wound healing process.

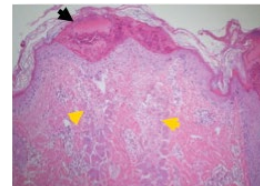

POST 14 DAYS

Outer layer of skin is completely healed and collagen remodeling continues for up to nine months.

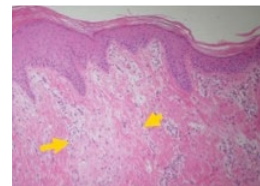

EXPECT DRAMATIC RESULTS

Your physician will discuss your treatment needs with you during your consult. The results will vary depending on your age, degree of sun damage and skin type, as well as what parameters your physician uses. However, patients who have had the eCO2 procedure, report significant changes in only a single treatment.

- Reduces facial fine lines and wrinkles
- Gives the skin a more firm, smooth appearance
- Improves scars
- Reduces sun damage, such as brown spots
- Reduces skin blotches and blends skin color
- Improves appearance of pigmented and vascular conditions

WHAT PATIENTS ARE SAYING

"The treatment was very comfortable. I am eight days out and very pleased with the results, my skin looks much smoother, and looks much better. I am looking forward to it looking better as time goes on."

eCO2 Patient, Florida, USA

