
















STEPS	TITLE	COMMENTS	HELP
1	UNPACK THE LASER. Disassemble and store the box for future shipping use	Take out the packing foam, pull off all the tape, fold the box flat and store it together with the foam	
2	Check that all accessories are in the box	Accessories include safety goggles, power cord / cable and foot switch	
3	Plug the cable into the back of the laser and then to the wall outlet	Make sure that the cable is properly connected to both the laser and the wall	
4	Check that emergency stop button is not pushed down or engaged	If it is pushed, twist it slightly while pulling out.	
5	Switch / power on the laser and wait about 30 seconds while the system goes through its start-up and the tip is cold.	The main power switch is located on the back of the laser	
6	When prompted, enter the password	The default password is 1234	
7	Read and accept the safety warning notice on the screen		
8	Depress the trigger and verify a small square icon appears on the screen	This process is a safety test to check ensuring the trigger works properly. It is also possible to pass this test with the foot pedal	
9	Place the lanyard around the emergency button to secure the handpiece to avoid accidentals drops	This is recommended at all times when the handpiece is going to be stored in the bay	
10	Put on your latex gloves		
11	Mark the area that is going to be treated with a white pencil	Divide the area into grids depending on the size of the region to be treated. • 10 x 10 cm - large area • 5 x 5 cm - small areas	
12	Check that the area to be treated is shaved and if not, shave it	Use one razor per patient.	
13	Apply a thin layer of ultrasound gel.	Gel is utilized to insure better sliding of the handpiece	

STEPS	TITLE	COMMENTS	HELP
14	Set the treatment parameters	Use a preset mode or an advance mode depending on your comfort and experience	
15	Choose the speed or repetition rate (Hz) you want to work at	It is recommended 1-2 Hz for facial areas, 3 Hz for medium areas (axillas, bikini, arms) and 4-6 Hz for large areas (legs, back, etc.)	
16	Place safety eyewear on	For both the patient and the operator	
17	Use one hand to hold the handpiece and put the lanyard on your wrist to secure it while using the other hand to hold a tissue	The tissue is to clean the gel and vaporized hair which can become stuck on the sapphire tip	
18	Unlock the laser using the touchscreen control panel	Click the green button "go to ready."	
19	Check that there is no ice accumulated on the tip and if so, remove it prior to treating	After 2-3 minutes ice may form due to the cooling system power. Gently remove it by passing your forearm over the sapphire	
20	Press the tip against the skin and start the treatment	Make sure that the handpiece is always in contact with skin	
21	Overlap the treated area at least 2 mm	Although the tip of the laser handpiece is 15 x 15 mm the effective working area of the laser is 10 x 10 mm	
22	Every 30-40 seconds stop and inspect the handpiece and clean the sapphire	If the hair is thick or the tip gets dirty quickly, it is advised to inspect and clean every 15-20 seconds	
23	Once the treatment is completed, lock the laser	Click the red button "go to standby." If the auto lock feature is engaged, system will lock automatically locked after the setting time	
24	Clean the tip, remove the remaining gel and place the handpiece in the cradle / bay	Use a disposable tissue or a wipe to clean	
25	Disinfect the tip of the laser handpiece including the sapphire with a proper cleaning agent	Do this after the treatment with the client is completed	
26	It is recommended to switch off the device after 2-3 hours of use or if the laser is not going to be used for 30 minutes	It is not essential to switch off the laser, however, it is recommended to do this during long workdays to conserve energy	
27	If the laser is to be packed to be shipped, please use the original box and foam	Don't forget to use tape to properly close the box	