

FILMED

LABORATOIRES



FILMED

SIGNATURE PROTOCOLS








TO PRECISELY TARGET INDICATIONS

TAILOR-MADE SOLUTIONS

FILMED SIGNATURE PROTOCOLS

TREAT ALL SKINS, ALL PHOTOTYPES, AT ANY AGE, ANY SEASON



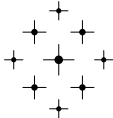




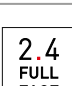
FILMED has developed a portfolio of protocols adapted to every patient and demand, from millennials to mature patients, from beautification to restoration.

CLINICAL ALGORITHM TO DECIDE WICH PROTOCOL		PATIENT'S CONCERNS	FOR WHO?
RADIANCE		Dull skin tone. Skin quality.	For everyone patients reluctant to injections and obsessed by quality of skin.
EYE		Crow's feet. Eye circles.	At any age patients with tired eyes looking for periorcular rejuvenation.
NECK		Loss of firmness in the neck. Neck lines. Dehydration, skin quality.	At any age patients looking for neck rejuvenation.
LIFT EFFECT		Cheek wrinkles. Lack of tonicity. Skin quality.	30-50 years old patients looking for revitalisation with lifting effects.
		Perioral wrinkles. Skin quality.	30-50 years old patients looking for perioral area rejuvenation.
FACE SHAPING		Oval sagging, loss of volume. Masculinization/Feminization. Skin quality.	30-65 years old patients looking for structure, face shaping and contouring.
FACIAL HIGHLIGHT		Beautification. Features enhancement.	18-35 years old patients looking for an instant beautification.
		Tired face. Negative expression.	35-50 years old patients looking for positive expression and improvement of facial features.


TAILOR-MADE SOLUTIONS

FILMED SIGNATURE PROTOCOLS

OPTIMIZED PROTOCOLS TO RESPECT THE SINGULARITY OF EACH PATIENT.

CLINICAL ALGORITHM TO DECIDE WICH PROTOCOL	PROTOCOLS		PRODUCTS
RADIANCE		BIONUTRIGLOW™	PEEL + TOPICAL POLYREVITALISING SOLUTION + SKIN CARE
EYE		BEAUTIFEYE™	PEEL + POLYREVITALISING SOLUTION with a specific device + SKIN CARE
NECK		BIONUTRINECK™	
LIFT EFFECT		BIONUTRILIFT®	POLYREVITALISING SOLUTION + DERMAL FILLERS
		BIONUTRILIPS™	
FACE SHAPING		BIONUTRISHAPE™	POLYREVITALISING SOLUTION + DERMAL FILLERS
FACIAL HIGHLIGHT		MEDICAL HIGHLIGHTING	ONLY DERMAL FILLERS
		2.4 FULL FACE™	

BIONUTRIGLOW™ RADIANCE

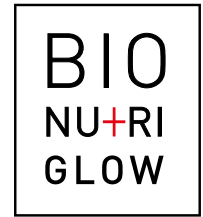

PATIENT'S CONCERNS

Dull skin tone – Skin quality

PRODUCTS

PEEL + TOPICAL POLYREVITALISING SOLUTION + SKIN CARE

TECHNIQUE



✓ Full face treatment

3 STEPS

1

SUPERFICIAL PEELING

gluconolactone,
suitable for sensitive skin

2

TOPICAL BIOREVITALISATION

> HA + polyrevitalising complex
to treat the epidermis
> with mesoneedling (for epidermal application)


3

SKIN CARE


vitamin B3 cream
to soothe and replenish the skin

RESULTS

GLOWY COMPLEXION
SKIN QUALITY IMPROVEMENT
HYDRATION
HOMOGENEITY

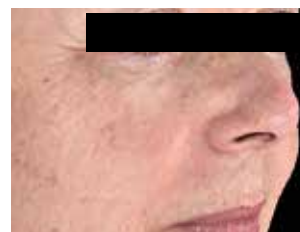

BEFORE

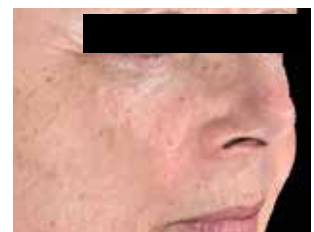

AFTER

TREATMENT PLAN / PROTOCOL

3 SESSIONS: 2 WEEKS APART⁽¹⁾



SESSION 1



SESSION 2



SESSION 3

15 DAYS

15 DAYS

To obtain an optimal result, 3 sessions in 1 month are recommended.


(1) If necessary two additional sessions can be recommended.

PATIENT'S CONCERNS	PRODUCTS
--------------------	----------

Crow's feet – Eye circles

PEEL + POLYREVITALISING SOLUTION with a specific device + SKIN CARE

TECHNIQUE

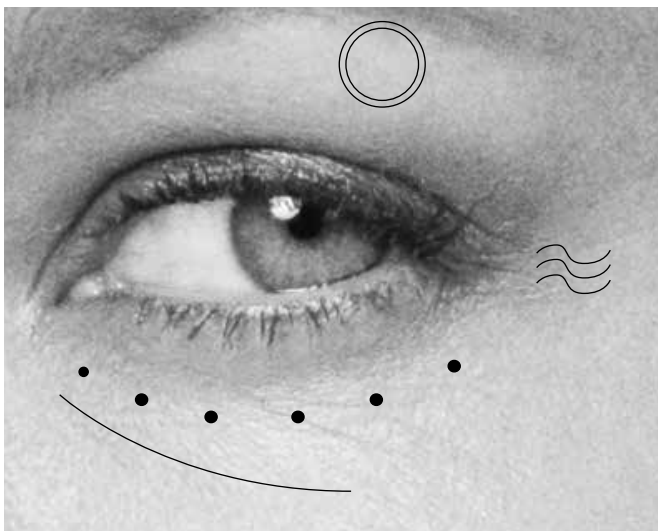

3 STEPS

1

SUPERFICIAL PEELING
gluconolactone,
suitable for sensitive skin

2

BIOREVITALISATION
> HA and 59 skin revitalising ingredients
to treat skin quality
> with intradermal multipuncture injection

3

SKIN CARE
hexylresorcinol/peptide eye cream
to regenerate the eye contour

- Polyrevitalising product with a specific device to target delicate areas.^[2]

RESULTS

SKIN QUALITY IMPROVEMENT
HYDRATION
HOMOGENEITY^[3] / DENSITY
FINE LINES



BEFORE



AFTER

TREATMENT PLAN / PROTOCOL

3 SESSIONS: 2 WEEKS APART^[1]



SESSION 1



SESSION 2



SESSION 3

15 DAYS

15 DAYS

[1] If necessary two additional sessions can be recommended. [2] The polyrevitalising product cannot be injected on any disruption of skin barrier of the treated area. A full recovery of the skin is necessary before injecting the polyrevitalising solution. [3] Clinical evaluation of anti-aging efficacy of mesotherapy with NCTF[®] 135 HA on healthy volunteers, 2014. Open, single center and prospective study. 20 patients included (49,25 years old on average), 2 of them left the study. 3 sessions spaced out by 15 days and 2 sessions by 30 days.

BIONUTRINECK™

NECK REVITALISATION

BIO
NU+RI
NECK


PATIENT'S CONCERNS

Neck lines – Dehydration – Skin quality

PRODUCTS

PEEL + POLYREVITALISING SOLUTION with a specific device
+ SKIN CARE

TECHNIQUE

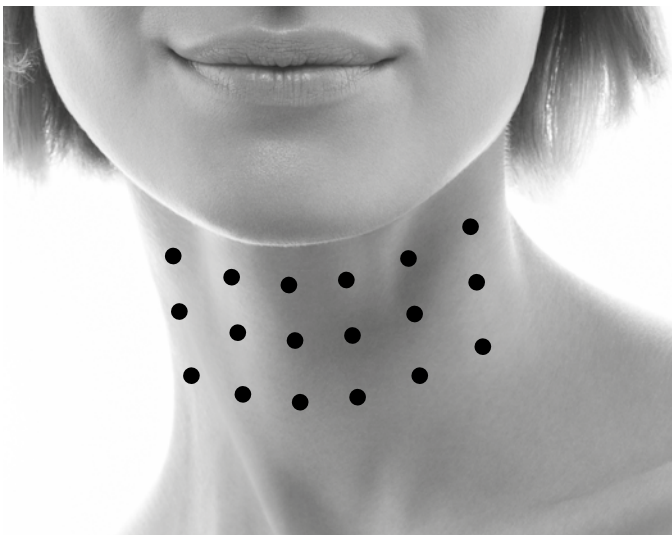

3 STEPS

1

SUPERFICIAL PEELING

gluconolactone, suitable for sensitive skin

2

BIOREVITALISATION

> HA + polyrevitalising complex
to treat skin quality

> with intradermal multipuncture injection

3

SKIN CARE

HA + Vitamin B5 serum
to hydrate the skin

- Polyrevitalising product with a specific device to target delicate areas.^[4]

RESULTS

SKIN QUALITY IMPROVEMENT

HYDRATION



TREATMENT PLAN / PROTOCOL

3 SESSIONS: 3 WEEKS APART^[1]



SESSION 1



SESSION 2



SESSION 3

3 WEEKS

3 WEEKS

[1] If necessary two additional sessions can be recommended. [4] The polyrevitalising product cannot be injected on any disruption of skin barrier of the treated area. A full recovery of the skin is necessary before injecting the polyrevitalising solution.

BIONUTRINECK™

NECK REJUVENATION

BIO
NU+RI
NECK

PATIENT'S CONCERNS

Loss of firmness in the neck – Neck lines –
Dehydration

PRODUCTS

PEEL + POLYREVITALISING SOLUTION with a specific device
+ SKIN CARE

TECHNIQUE



3 STEPS

1

SUPERFICIAL PEELING

gluconolactone, suitable for sensitive skin

2

HA SOLUTION

> High concentration of free HA
to treat skin quality
> with intradermal multipuncture injection

3

SKIN CARE

HA + Vitamin B5 serum
to hydrate the skin

○ Hydration solution (high concentration of free HA) with a
specific device to target delicate areas.^[5]

RESULTS

NECK LINES CORRECTION
SKIN TEXTURE IMPROVEMENT
ELASTICITY
HYDRATION



BEFORE



AFTER

TREATMENT PLAN / PROTOCOL

3 SESSIONS: 3 WEEKS APART^[1]

BIO
NU+RI
NECK

SESSION 1

BIO
NU+RI
NECK

SESSION 2

BIO
NU+RI
NECK

SESSION 3


3 WEEKS

3 WEEKS

[1] If necessary two additional sessions can be recommended. [5] The hydration solution cannot be injected on any disruption of skin barrier of the treated area. A full recovery of the skin is necessary before injecting the hydration solution.

BIONUTRILIFT®

LIFT EFFECT

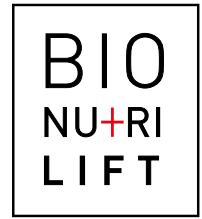


PATIENT'S CONCERNS	PRODUCTS
--------------------	----------

Cheek wrinkles – Lack of tonicity –
Skin quality


POLYREVITALISING SOLUTION + DERMAL FILLERS

TECHNIQUE

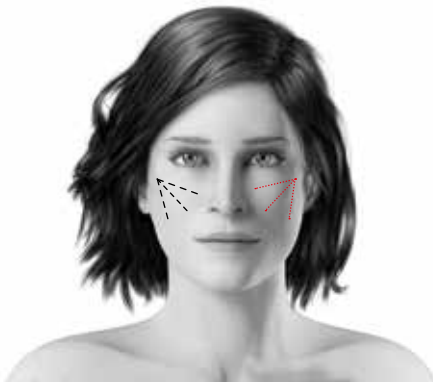


TYPE I - MILD

● + ●

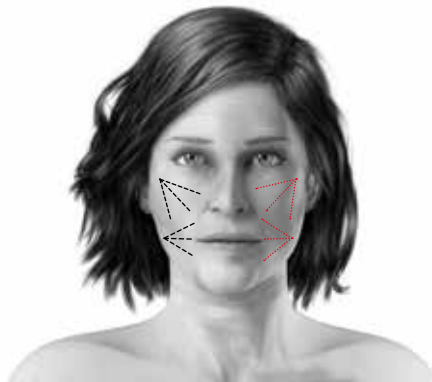

TYPE II - MODERATE

● ● + ● ●

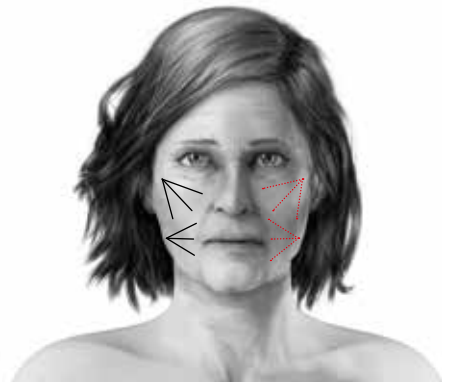

TYPE III - IV ADVANCED - SEVERE

● ● + ● ●

● **Dermal filler**
Quantity and rheology have to be adapted to patient's needs and age

● **Polyrevitalising product**
Quantity has to be adapted to patient's needs and age

RESULTS

LIFT AND SMOOTH EFFECT
SKIN QUALITY IMPROVEMENT




BEFORE



AFTER

TREATMENT PLAN / PROTOCOL

THE TREATMENT CAN BE REPEATED EVERY 12 TO 18 MONTHS⁽⁶⁾

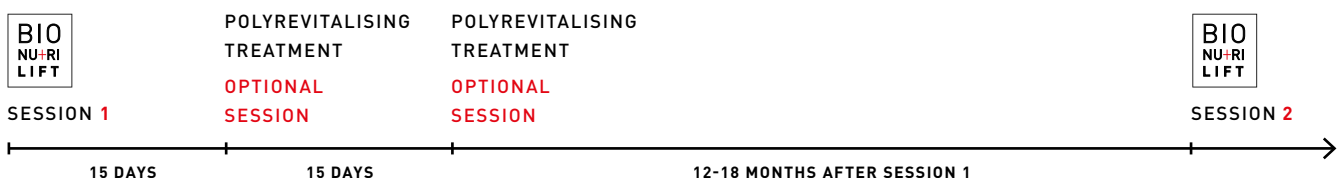


(6) The treatment plan is developed according to the patient's needs.

BIONUTRILIPS™

PERIORAL DEFINITION

LIFT EFFECT

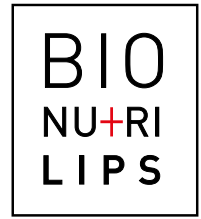

PATIENT'S CONCERNS	PRODUCTS
--------------------	----------

Perioral wrinkles – Skin quality

POLYREVITALISING SOLUTION + DERMAL FILLERS

TECHNIQUE



- Soft lip dermal filler
- Polyrevitalising product

RESULTS

- LIP REDEFINITION
- RESTORATION OF THE PERIORAL AREA
- SKIN QUALITY IMPROVEMENT




BEFORE



AFTER

TREATMENT PLAN / PROTOCOL


THE TREATMENT CAN BE REPEATED EVERY 12 TO 18 MONTHS⁽⁶⁾

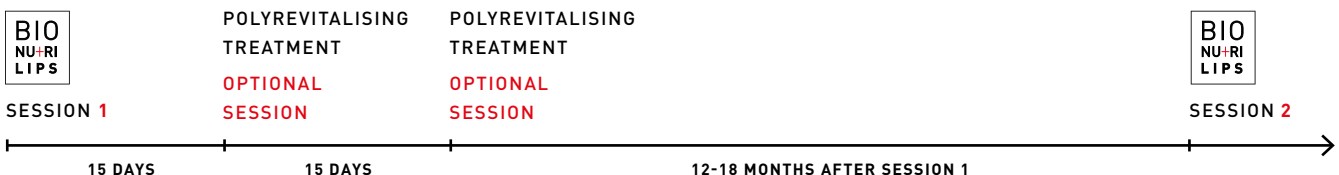

⁽⁶⁾ The treatment plan is developed according to the patient's needs.

BIONUTRISHAPE™

FACE SHAPING

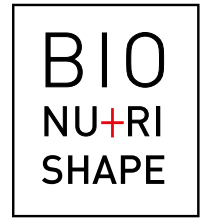


PATIENT'S CONCERNS	PRODUCTS
--------------------	----------

Oval sagging, loss of volume –
 Masculinization/Feminization – Skin quality

POLYREVITALISING SOLUTION + DERMAL FILLERS

TECHNIQUE

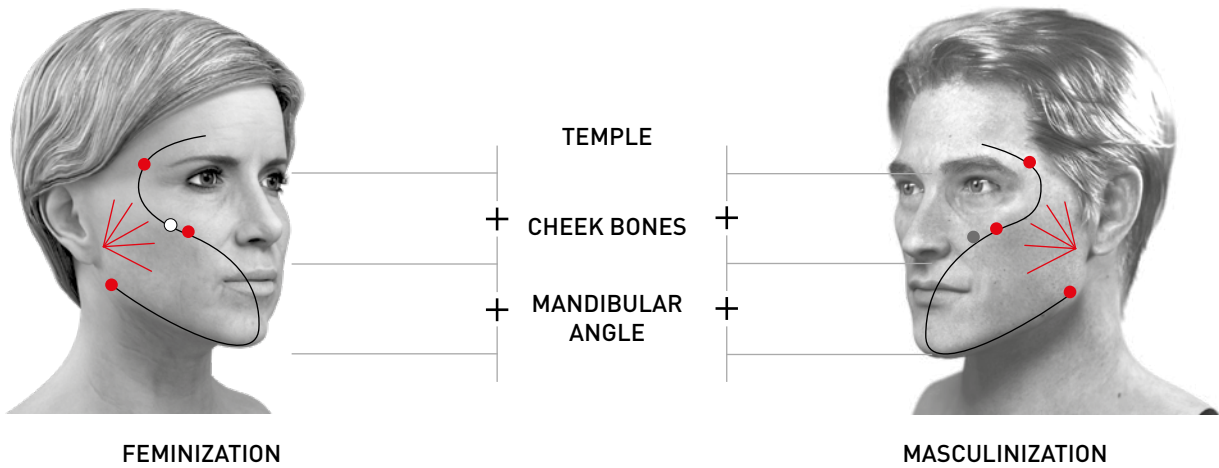

<p>STRUCTURE AND SHAPE WITH A DERMAL FILLER</p> <ul style="list-style-type: none"> ● Common point ○ Feminization point ● Masculinization point 	<p>SKIN QUALITY</p> <p>↖ Polyrevitalising product</p>
--	--

RESULTS

- SKIN TEXTURE IMPROVEMENT
- VOLUME RESTORATION
- OVAL OF THE FACE REDEFINITION
- FEMINIZATION / MASCULINIZATION



BEFORE

AFTER



BEFORE

AFTER

BIONUTRISHAPE™

FACE SHAPING

BIO
NU+RI
SHAPE


PATIENT'S CONCERNS

Oval sagging, loss of volume –
Mascularization/Feminization – Skin quality

PRODUCTS

POLYREVITALISING SOLUTION + DERMAL FILLERS

TECHNIQUE

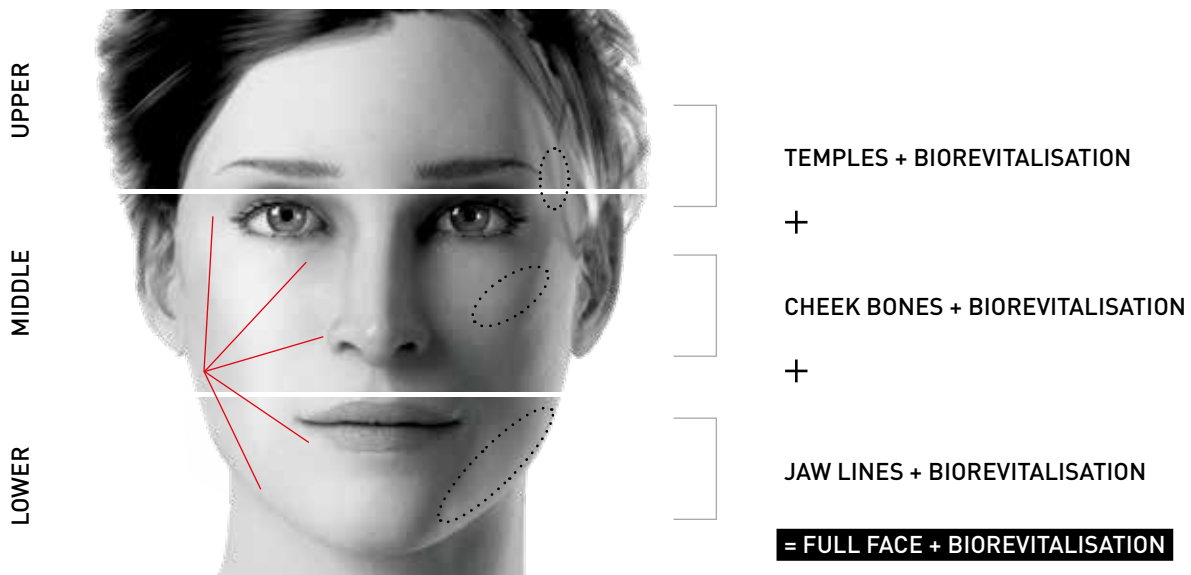

Zone by zone treatment – Temples, cheek bones or jaw lines

TREATMENT PLAN / PROTOCOL

THE TREATMENT CAN BE REPEATED EVERY 12 TO 18 MONTHS⁽⁶⁾



⁽⁶⁾ The treatment plan is developed according to the patient's needs.

MEDICAL HIGHLIGHTING BEAUTIFICATION

MEDICAL
HIGH
LIGH
+ING


PATIENT'S CONCERNS

Beautification – Features enhancement

PRODUCTS

ONLY DERMAL FILLERS

TECHNIQUE

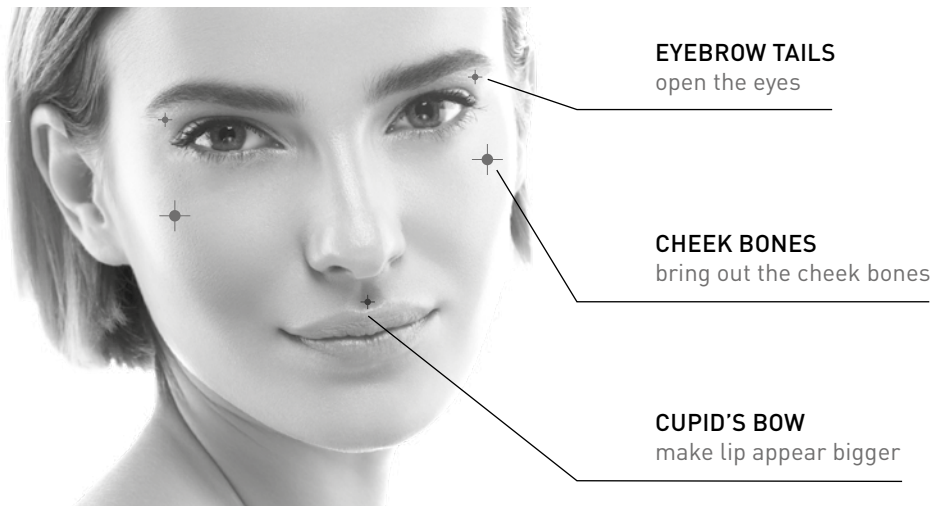

+ Dermal filler injected in convex zone to catch the light

RESULTS

INSTANT BEAUTIFICATION

NATURAL RESULT

FACE LIGHTENING



BEFORE



AFTER

TREATMENT PLAN / PROTOCOL

THE TREATMENT CAN BE REPEATED EVERY 12 TO 18 MONTHS.

POLYREVITALISING TREATMENT IS ALSO AVAILABLE FOR SKIN IMPROVEMENT.^[6]

[6] The treatment plan is developed according to the patient's needs.

2.4 FULL FACE™ FACIAL HIGHLIGHT

2.4
FULL
FACE

PATIENT'S CONCERNS

Tired face – Negative expression

PRODUCTS

ONLY DERMAL FILLERS

TECHNIQUE

SHADOWS
AND DEPRESSIONS

NASOLABIAL FOLDS


MARIONNETTE LINES

LIGHT
AND PROJECTION

CHEEK BONES LATERAL POINT

CHEEK BONES MEDIUM POINT

MANDIBULAR ANGLE

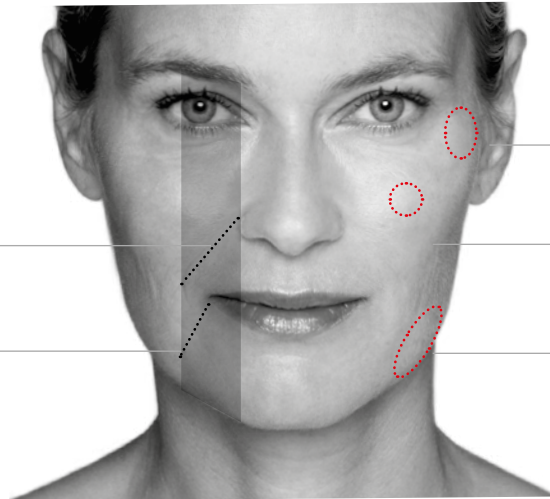

■ Facial negative line

● Moldable dermal filler / ● Projecting dermal filler

RESULTS

POSITIVE EXPRESSION

VOLUME RESTORATION



BEFORE



AFTER

TREATMENT PLAN / PROTOCOL

THE TREATMENT CAN BE REPEATED EVERY 12 TO 18 MONTHS.
POLYREVITALISING TREATMENT IS ALSO AVAILABLE FOR SKIN IMPROVEMENT.^[6]

[6] The treatment plan is developed according to the patient's needs.

FILMED PORTFOLIO

NCTF® 135 HA, NCTF® BOOST 135 HA & M-HA® 10



PRODUCTS	NCTF® 135 HA	M-HA® 10	NCTF® BOOST 135 HA
INDICATIONS AND TREATED AREAS	Injectable product (CE mark)		Sterile cosmetic (topical use)
	<p>Indicated for intense revitalisation and hydration of tired or dull skin, filling of superficial wrinkles and redensification of mature or slack skin.</p> <p>Face, neck, décolleté and back of the hands.</p>	<p>Indicated to improve the hydration, the radiance and the elasticity of the skin and to serve as a filling agent for small wrinkles.</p> <p>Face, neck, décolleté and back of the hands.</p>	<p>Indicated for enhancing the skin's appearance, it nourishes the epidermis by providing the ingredients that it needs to thrive.</p> <p>Face, neck and décolleté.</p>
HA CONCENTRATION (mg/mL)	5 mg/mL	10 mg/mL	5 mg/mL
COMPOSITION	Free HA + 12 Vitamins + 24 Amino acids + 6 Co-enzymes + 5 Nucleic acids + 6 Minerals + 6 other ingredients (Glutathione, Polysorbate 80, Glucuronic acid, Glucuronic acid lactone, Glucosamine, Dextrose anhydrous)	Free HA	Free HA + 12 Vitamins + 24 Amino acids + 6 Co-enzymes + 5 Nucleic acids + 6 Minerals + 6 other ingredients (Glutathione, Polysorbate 80, Glucuronic acid, Glucuronic acid lactone, Glucosamine, Dextrose anhydrous)
PACKAGING	<p>5 x 3 mL vials + 5 withdrawing needles (18G 1½) + 10 injection needles (5 x 30G½ and 5 x 32G) + 5 syringes or 10 x 3 mL vials</p>	<p>3 x 3 mL vials + 3 withdrawing needles (18G 1½) + 3 injection needles (30G½) + 3 injection needles (32G) + 3 syringes</p>	<p>10 x 3 mL vials cosmetics</p>

NANOSOFT™ DEVICE

EXCLUSIVE PENETRATION TECHNOLOGY

QUICK & EASY PROCEDURE



PRODUCT	NANOSOFT™
INDICATIONS AND TREATED AREAS	Standardized intradermal injection of NCTF® or M-HA® 10 to target delicate areas (eye contour, neck & décolleté).
BENEFITS	Optimal safety, nearly painless, minimal bruising, less invasive, easy to handle, optimal design can be mounted on any standard syringe.
DESCRIPTION	3 microneedles, 0.6 mm in length.
PACKAGING	Dispenser of 30 sterile devices

BIONUTRILIFT™ CANNULAS DEVICE

SOFT AND PRECISE

FACIAL REJUVENATION



PRODUCT	BIONUTRILIFT® CANNULAS
INDICATIONS AND TREATED AREAS	<p style="text-align: center;">Sterile hypodermic needles with round tips, single-use, designed to fill in wrinkles, restore volume and reshape facial contours by fanning injection, mainly in the hypodermis, subcutaneous and supra-periosteal areas.</p>
BENEFITS	<p style="text-align: center;">Improved tolerance, reduced extrusion force, precision, better maneuverability, symmetrical treatment.</p>
DESCRIPTION	<p style="text-align: center;">Sterile and single-use injection kit containing a sharp needle in its protective cap and a round-tipped needle (micro-cannula) in its protective cap.</p>
PACKAGING	<p style="text-align: center;">20 individual sterile kits with 1 micro-cannula 25G – 55 mm + 1 sharp needle 24G – 55 mm</p>

ART FILLER®

FINE LINES, UNIVERSAL, VOLUME, LIPS SOFT, LIPS



	WRINKLES		VOLUME
PRODUCTS	ART FILLER® FINE LINES	ART FILLER® UNIVERSAL	ART FILLER® VOLUME
INDICATIONS AND TREATED AREAS	Fine lines and superficial wrinkles: crow's feet, forehead, upper lip and cheeks.	Medium to deep wrinkles: naso-labial folds and marionettes lines.	Face volume: mid-face, jawline and temples.
HA CONCENTRATION (mg/mL)	20 mg/mL	25 mg/mL	25 mg/mL
COMPOSITION	Cross-linked HA + 0.3% lidocaine		
CROSS-LINKING	●○○○	●●●○	●●●●
PACKAGING	2 x 1 mL syringes 4 x 30G½" needles	2 x 1.2 mL syringes 4 x 27G½" needles	2 x 1.2 mL syringes 4 x 27G½" needles



	LIP MENU	
PRODUCTS	ART FILLER® LIPS SOFT	ART FILLER® LIPS
INDICATIONS AND TREATED AREAS	Perioral wrinkles and soft volume: naso-labial folds, marionette lines, lip outline and red lip.	Lip volume: naso-labial folds, marionette lines, lip outline and red lip.
HA CONCENTRATION (mg/mL)	25 mg/mL	25 mg/mL
COMPOSITION	Cross-linked HA + 0.3% lidocaine	
CROSS-LINKING	●●●○	●●●○
PACKAGING	1 x 1 mL syringe 2 x 30G½" needles	2 x 1 mL syringes 4 x 27G½" needles

PEELING

TIME PEEL® , BRIGHT PEEL® , LIGHT PEEL®



PRODUCTS	TIME PEEL®	BRIGHT PEEL®	LIGHT PEEL®
INDICATIONS	Wrinkles firmness	Dark spots uneven complexion	Fine lines radiance
COMPOSITION	Complex peeling 50%: Gluconolactone + Glycolic acid	Complex peeling 50%: Gluconolactone + Phytic acid + Citric acid + Glycolic acid	Complex peeling 20%: Gluconolactone + Mandelic acid
pH	pH≈1.2 (+/-0.2)	pH≈1.5 (+/-0.3)	pH≥3.5 (+/- 0.3)
PACKAGING	100 mL bottle + dosing tube + brush		



PRODUCTS	PRE PEEL	POST PEEL
INDICATIONS	Preparing lotion	Neutralising & soothing treatment
COMPOSITION	Gluconolactone (prepares the skin and balances the skin pH)	Bicarbonate of soda (neutralises the action of acids) + Dextran sulfate (soothes the skin)
PH	pH≈4	pH≈8
PACKAGING	100 mL bottle	

SKIN PERFUSION COSMETICS

TO ACCOMPANY AESTHETIC PROCEDURES

DAILY ROUTINE TO MAINTAIN SKIN QUALITY



CLEANSERS
EXFOLIATING CREAM
CLEANSING OIL
MICELLAR WATER

INTENSIVE TREATMENT TO IMPROVE SKIN QUALITY



PRE-TREATMENT: SOFT EXFOLIATE
PERFECTING SOLUTION
GLYCOPEEL MASK



ANTI-AGEING
5HP-YOUTH CREAM
6HP-YOUTH CREAM
HXR-EYE CREAM
GR-YOUTH MASK
NOURISHING BODY BALM



POST-TREATMENT: REPAIR
B3-RECOVERY CREAM
E-YOUTH 50 SUN CREAM
HYALURONIC YOUTH MASK
EYE-RECOVER MASK



TARGETED SERUMS
HAB5-HYDRA SERUM
RE-TIME SERUM
AA-LIFT SERUM
P-BRIGHT SERUM
BD-BALANCE SERUM
C-LIGHT SERUM

SHORT CURE - 1 MONTH



ENHANCING TARGETED BOOSTERS
HYDRA-BOOSTER
TIME-BOOSTER
LIFT-BOOSTER
BRIGHT-BOOSTER
BALANCE-BOOSTER



BEAUTY IS AN ART, WE MADE IT A SCIENCE

A division of the Cryomed Group in Australia & New Zealand. www.fillmed.com.au

NCTF® 135 HA, ART FILLER®, M-HA® 10 are ranges of injectable medical devices. NANOSOFT™ is a medical device designed for intradermal injections by microneedles. BIONUTRILIFT® CANNULAS are medical devices designed for fillers' injection. Ask your doctor for advice. If any adverse event occurs, please inform your doctor. For safety information, please refer to the instructions for use of each product. Vigilance contact (notification of an adverse event): vigilance@fillmed.com.