

### 1 Turn the System ON

Press the 'I' symbol on the AC Power Switch at the rear of the Main Body and proceed to turn the key clockwise to turn the System ON.



### 2 Set the Parameters

Adjust the treatment preference via the Graphical User Interface (GUI).



### 3 Apply Ultrasound Gel

Apply a sufficient amount of ultrasound gel on the treatment area.



### 4 Apply the Cartridge

Place the Cartridge on the treatment area ensuring there is uniform contact with the skin.



### 5 Check for Patient Reactions

Press the READY button on the Graphical User Interface (GUI) and proceed to conduct a few test shots to check the patient's tolerance level.



### 6 Commence Treatment

Proceed to treat or adjust the parameters on the Graphical User Interface (GUI) based on the patient's tolerance level.



DO NOT apply shots without making proper contact between the Cartridge and skin.  
DO NOT press the shot button of the Hand-piece without applying ultrasound gel onto the treatment area.

## Contraindications

**ULTRAFORMER III is contraindicated for use in patients with:**

- Pregnancy or when breastfeeding
- Open wounds or lesions on the treatment area
- Severe or cystic acne on the treatment area
- Metal stents/electrical implants in the treatment area
- Bioabsorbable mechanical implants
- Skin infections of any type
- Hemorrhagic disorders or dysfunctions

## Precautions for Patient

**Determine whether treatment is appropriate for patients with the following conditions:**

- Anticoagulant therapy
- Skin conditions caused by autoimmune diseases, skin cancer, and Herpes Simplex
- Diabetes or epilepsy
- Severe skin diseases
- Decortication or heart disease
- Hypersensitiveness or hypertrophic scars
- Hemostatic dysfunctions

## Precautions for Procedure

**To avoid safety hazards from occurring during treatment with the ULTRAFORMER III:**

- Do not use the ULTRAFORMER III without completion of training by an authorized CLASSYS representative.
- To prevent unauthorized use of the device, ensure access is strictly limited to trained and certified personnel only.
- To avoid possible injury during treatment, do not apply the ULTRAFORMER III Hand-pieces near the patient's eyes.
- Avoid applying treatment on the thyroid gland, thyroid cartilage, and trachea.
- Avoid applying treatment directly on the breast tissue and major vessels.
- If any malfunctions or adverse events occur, immediately stop the treatment
- Always apply ultrasound gel on the target areas before proceeding with the treatment.
- Before conducting shots, make sure the Cartridge has uniform contact with the skin.


## Precautions for System Safety

**To ensure safety of the System and proper maintenance of the ULTRAFORMER III:**

- To avoid risk of electric shock, always inspect the Cartridge, Hand-piece, and Cable before use.
- Do not use a Cable or Cartridge that has been damaged or is leaking fluid.
- Do not remove the covers of the ULTRAFORMER III Main Body or Hand-pieces as modification of this equipment is not allowed.
- ULTRAFORMER III contains no user serviceable components. Contact a local distributor or CLASSYS representative if the System requires service.
- Do not lean over the equipment obliquely and do not apply a physical external shock to the equipment, as it may cause permanent System damage.
- Do not use the equipment with anesthetic gases or flammable materials in the surrounding areas.


- Fine Lines & Wrinkles
- Forehead
- Perioral (Lips)
- Periorbital (Around the eyes)
- Hollywood Lift

- 1.5mm
- or
- 2.0mm

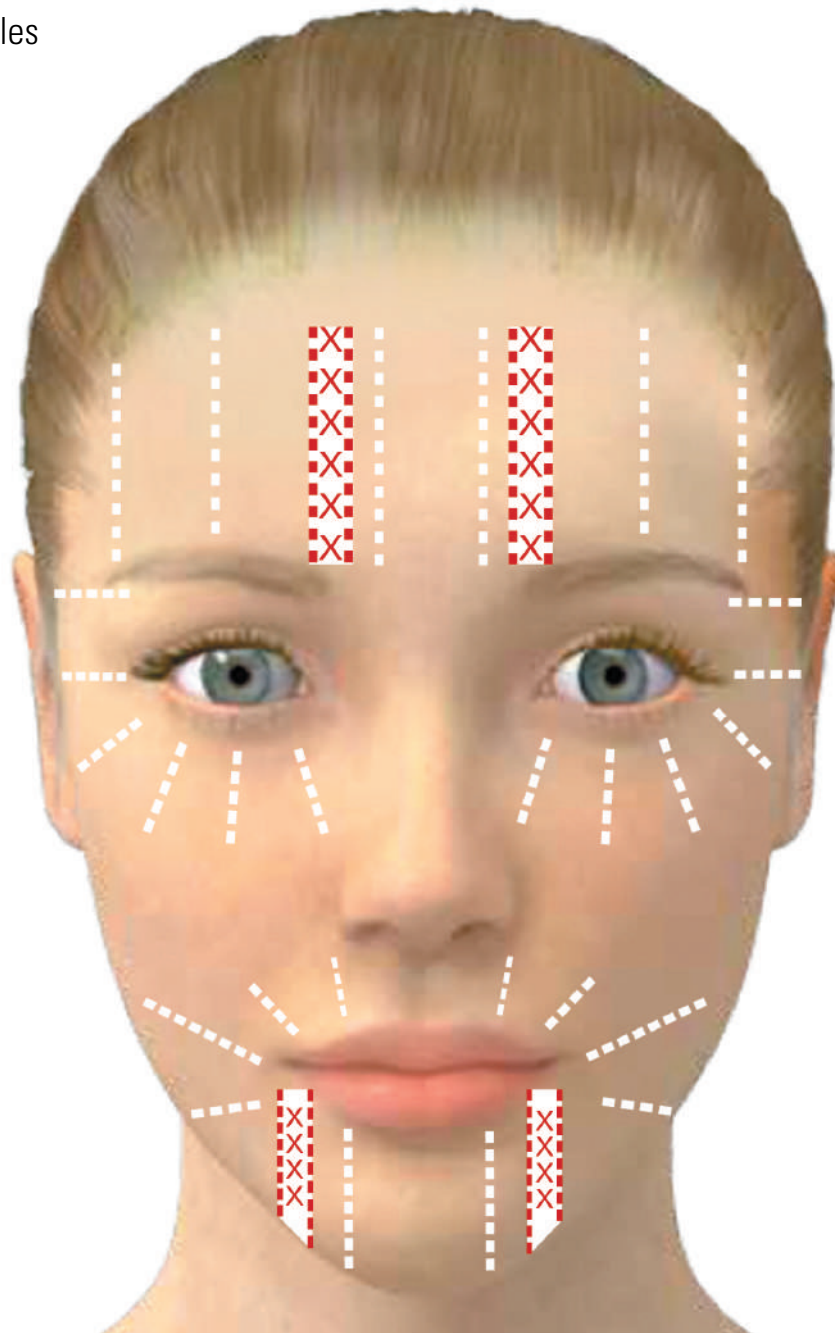


- 1) Use 1.5mm or 2.0mm Cartridge for all the marked areas indicated above.
- 2) Skin on the forehead is thinner and closer to bone; therefore, take extra care when treating. Always ask the patient for their tolerance level.
- 3) You can cross-hatch if necessary to achieve your targeted number of shots.
- 4) On narrow areas, try to angle the cartridge to fit the treatment areas.
- 5) Use light marker or white pencil if possible for markings.
- 6) Note the no-go zones marked with red X's.

## ○ Cheeks / Lower Face

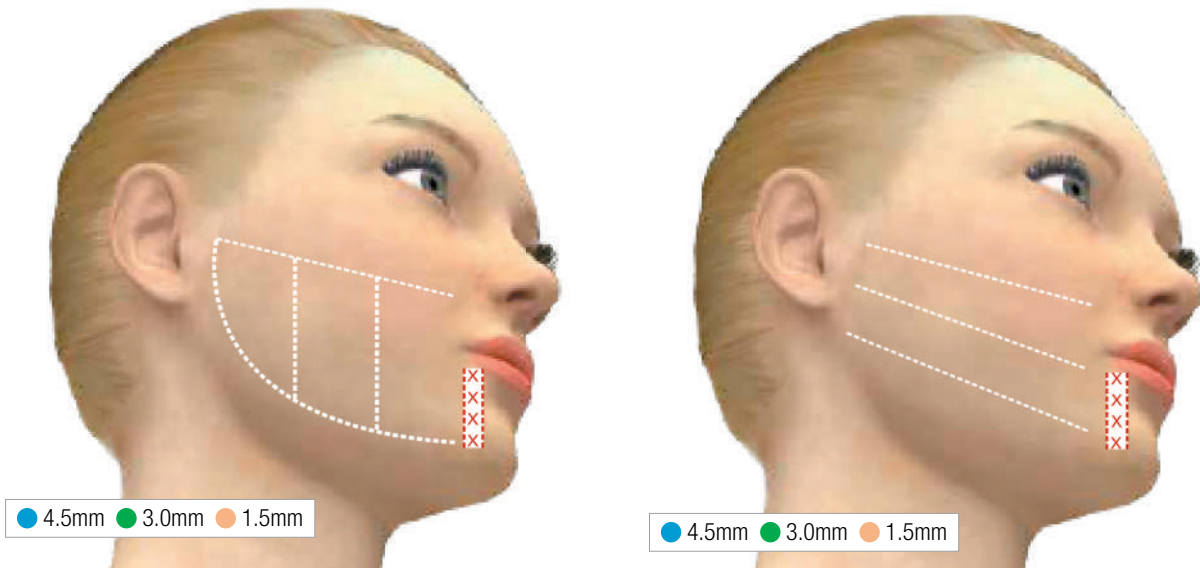


- 1) For lower face you may utilise vertical and horizontal columns, or both in cross-hatching fashion, to achieve your target shots.
- 2) Take extra care when using the 4.5mm cartridge on bony areas as this may cause discomfort, e.g., cheek bones and jaw bone.
- 3) Note the no-go zones marked with red X's.
- 4) Draw the top line across the cheek just under the zygoma (cheekbone) to avoid treating directly on bone and causing unnecessary discomfort.

## ○ Submental Area

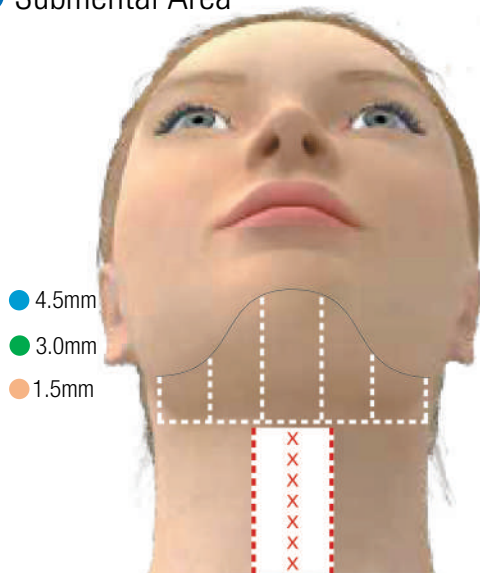

## ○ Neck

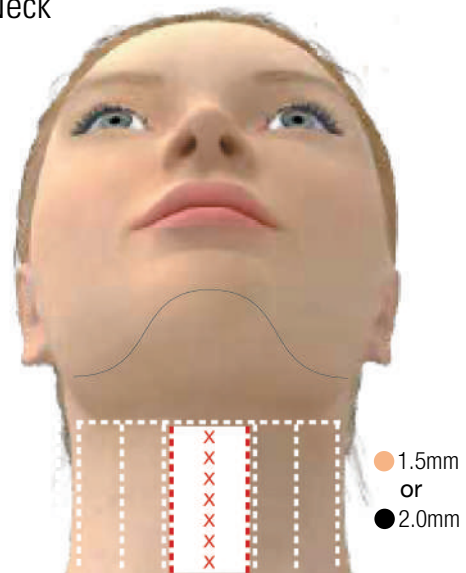

- 1) Submental area and neck are two separate sections.
- 2) The submental area starts from the jaw bone and extends down to the thyroid notch.
- 3) The neck starts from where the submentum area ends downwards to the clavicle area.
- 4) Take extra caution when treating neck tissue, as this is a sensitive area.

### Graphical User Interface

The screenshot shows the ULTRAFORMER III control panel. At the top, it displays 'Current 100' and 'Remain 6300'. The central display shows '0.4 J' and a 'START' button. Below the display are 'M1' and 'M2' buttons, a 'Shot count information' icon, and a 'Total Used 200' indicator. On the right, there are buttons for 'Single/Repeat Shot Mode', 'Settings', 'Shot Count Setting Alarm set by User', and 'Total Shot Count'. The interface also shows two cartridge options: 'L7-3.0' and 'L4-4.5'. The 'START / STANDBY' label is positioned below the main display area.

Remaining Shot Count of Cartridge

Shots Delivered by Current Cartridge

Pitch

Length

Preset Configuration  
Hold down the M1/M2 Button to save the current configuration. Press the M1/M2 Button to recall.

Power Adjustment (J/cm<sup>2</sup>)

Shot Count Information

Displays the Hand-piece is inactive

Single/Repeat Shot Mode

Settings


Shot Count Setting Alarm set by User

Total Shot Count

START / STANDBY

### Fine Lines & Wrinkles


### Hollywood Lift

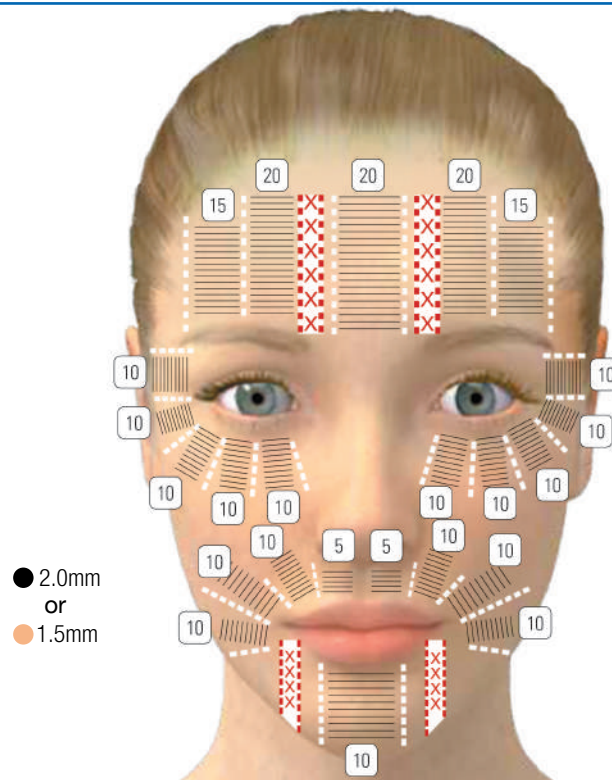

Area	Cartridge	Energy	Pitch	Passes	Shots
Forehead					180
Around the Eyes	● 2.0mm or ● 1.5mm	0.1-0.2J	1.4mm	2	200
Around the Mouth					160

- Do not apply treatment directly on the supraorbital or temporal nerves.
- Have patients close their eyes when treating the periorbital region, and look upwards when treating under the eye.
- When treating near the brow line, angle the cartridge upwards towards the forehead to reduce discomfort.
- Start with lower energy and work your way up.
- Select between the 2.0mm and the 1.5mm cartridge according to skin thickness. For thicker skin use the 2.0mm, and for thinner skin opt for the 1.5mm.



### Forehead & Periorbital Lift

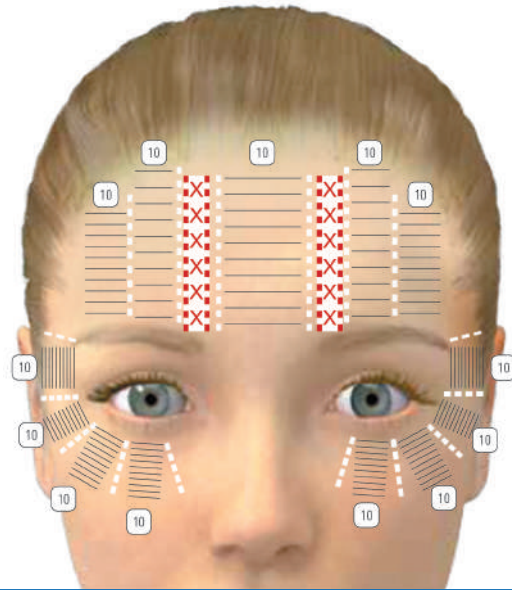

Cartridge	Energy	Pitch	Passes	Shots
● 1.5mm	0.1-0.2J	1.4mm	3	390

- Do not apply treatment directly on the supraorbital or temporal nerves.
- Have patients close their eyes when treating the periorbital region. When treating near the brow line, angle the cartridge upwards towards the forehead to reduce discomfort.
- Start with lower energy and work your way up.

### Lower Face Lift

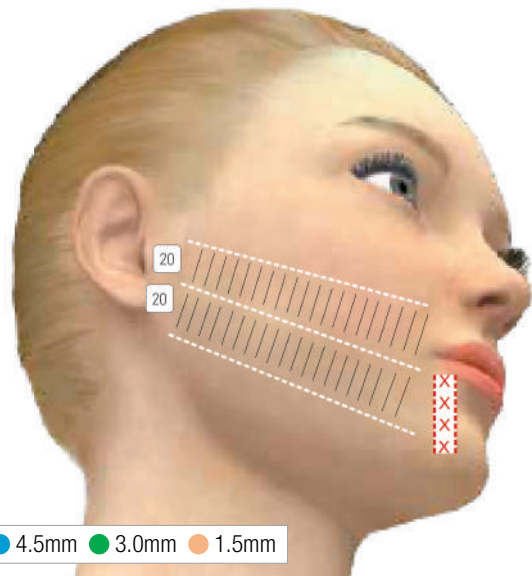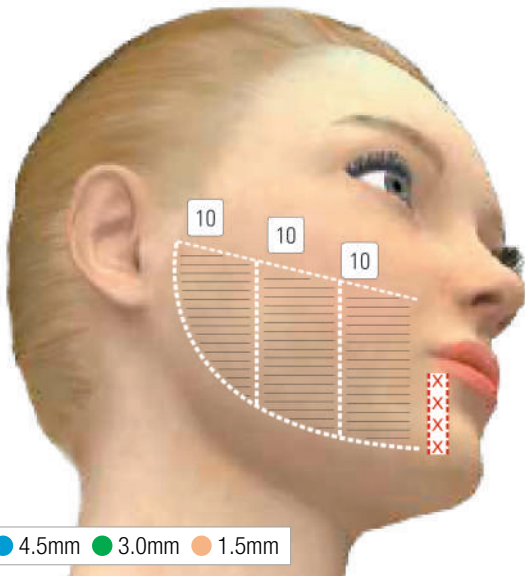


Cartridge	Energy	Pitch	Shots
● 4.5mm	0.5-0.7J	1.4mm	70 per side
● 3.0mm	0.3-0.4J	1.4mm	70 per side
● 1.5mm	0.1-0.2J	1.4mm	70 per side


- Do not apply treatment directly on the marginal mandibular nerve.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Patients can have paracetamol prior to treatment, if necessary.
- Start with lower energy and work your way up.

### Submental Area

- 4.5mm
- 3.0mm
- 1.5mm

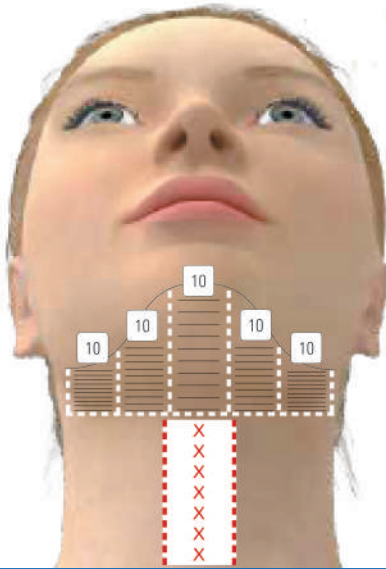

### Neck

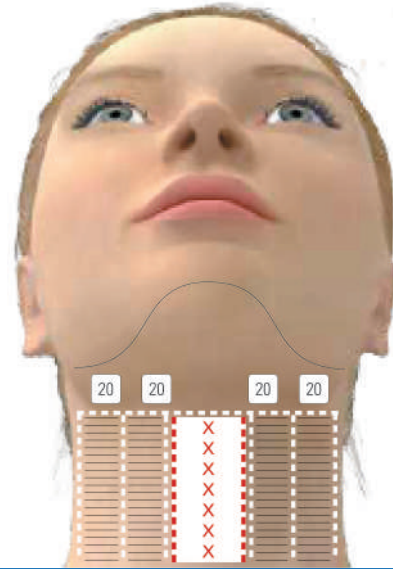


- 2.0mm or 1.5mm

Cartridge	Energy	Pitch	Passes	Shots
● 4.5mm	0.5-0.7J	1.4mm	3	150
● 3.0mm	0.3-0.4J	1.4mm	3	150
● 1.5mm	0.1-0.2J	1.4mm	2	100



Cartridge	Energy	Pitch	Passes	Shots
● 1.5mm	0.1-0.2J	1.4mm	2	160
● 2.0mm	0.1-0.2J	1.4mm	2	160

- Do not apply treatment directly on the marginal mandibular nerve, jaw bone, or thyroid area.
- When treating multiple layers, always treat from deep to shallow in sequential order, e.g., 4.5mm 3.0mm 1.5mm.
- Start with lower energy and work your way up.

### Décolletage

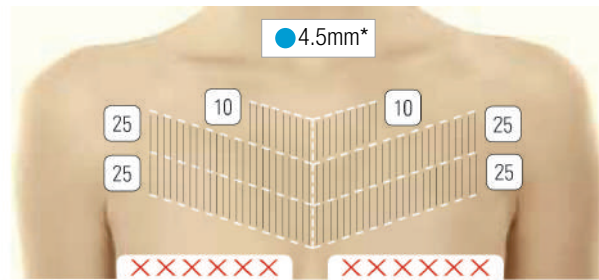


\*Optional

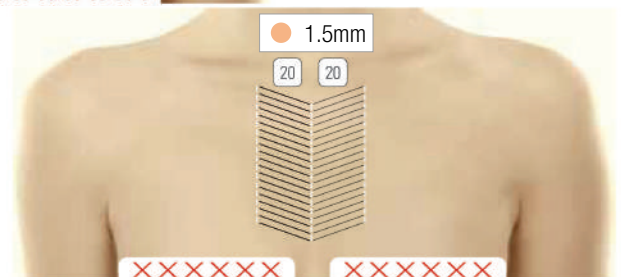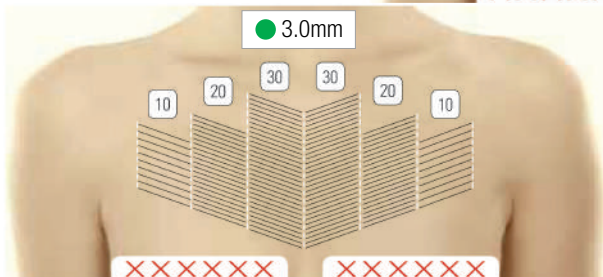

Cartridge	Energy	Pitch	Passes	Shots
● 4.5mm (Optional)	0.2-0.4J	1.4mm	1	120
● 3.0mm	0.2-0.3J	1.4mm	1	120
● 1.5mm	0.1-0.2J	1.4mm	1	40

- Do not apply treatment directly on the clavicle, breast tissue, or breast implants
- When treating multiple layers, treat from deep to shallow in sequential order, e.g., 3.0mm 1.5mm.
- The 4.5mm cartridge is optional on this treatment and often omitted. Use only on patients with visibly thicker skin. You can use the 2.0mm cartridge at 0.1-0.2J as a substitute.
- Start with lower energy and work your way up.


### Abdomen Tightening

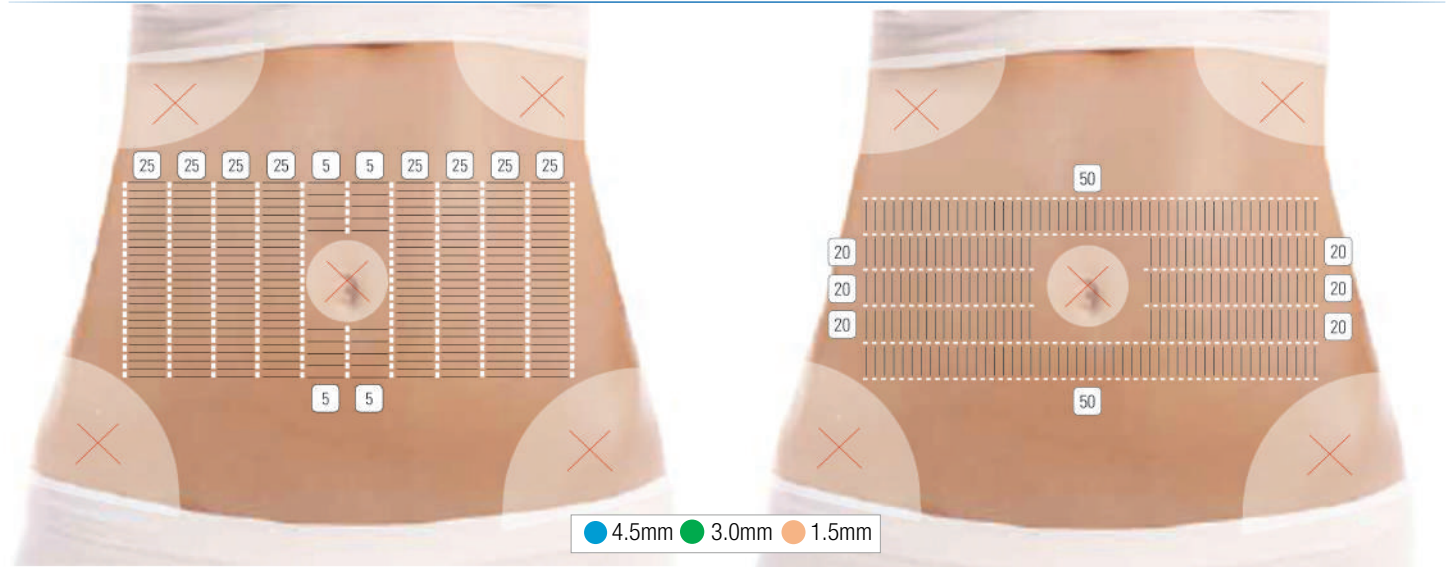

Cartridge	Energy	Pitch	Passes	Shots
● 4.5mm	0.5-0.7J	1.4mm	2-3	440-660
● 3.0mm	0.3-0.4J	1.4mm	2-3	440-660
● 1.5mm	0.1-0.2J	1.4mm	2-3	440-660

- Do not apply treatment directly on the navel, ribs, or pelvis.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Omit 4.5mm cartridge and use 2.0mm instead if patient is slim or has thin tissue.
- Do not treat together with body cartridges (for abdomen contouring) in one session as this may likely be too much treatment for one area.
- Start with lower energy and work your way up.


### Abdomen Contouring

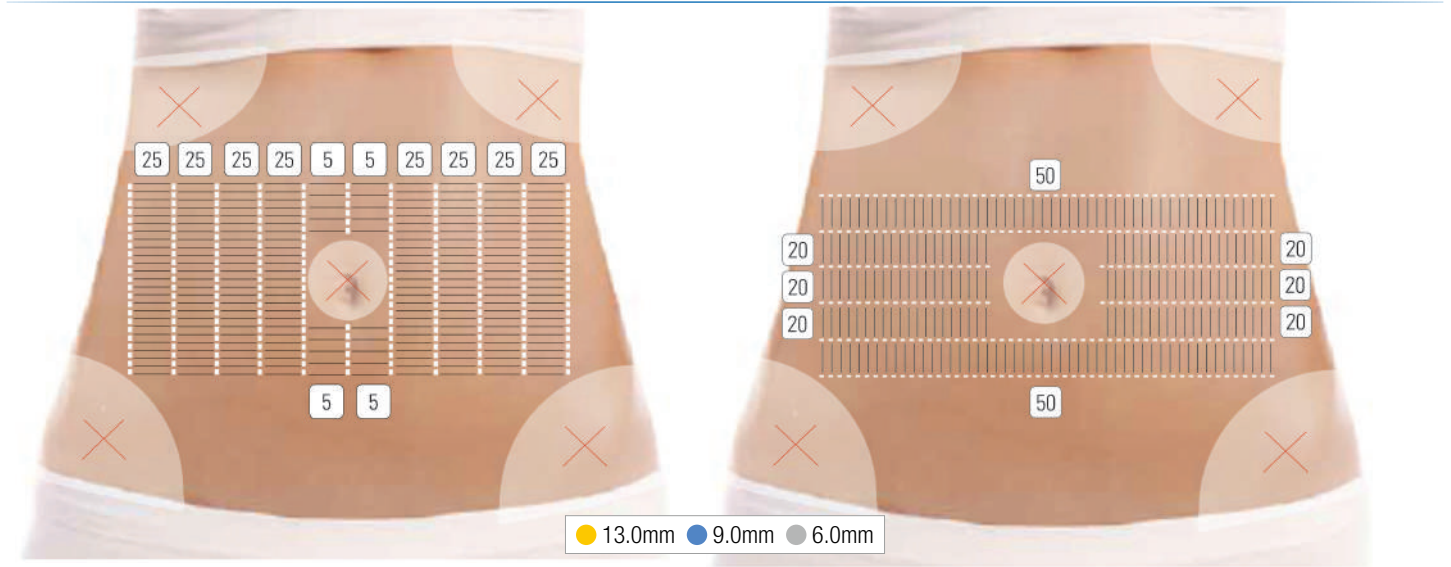

Cartridge	Energy	Pitch	Passes	Shots
● 13.0mm	1.5-2.2J	1.4mm	3	660
● 9.0mm	1.5-2.5J	1.4mm	3	660
● 6.0mm	1.5-2.2J	1.4mm	3	660

- Do not apply treatment directly on the navel, ribs, or pelvis.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for a contouring effect. It is important to choose the right patient for this treatment.

### Flanks Contouring

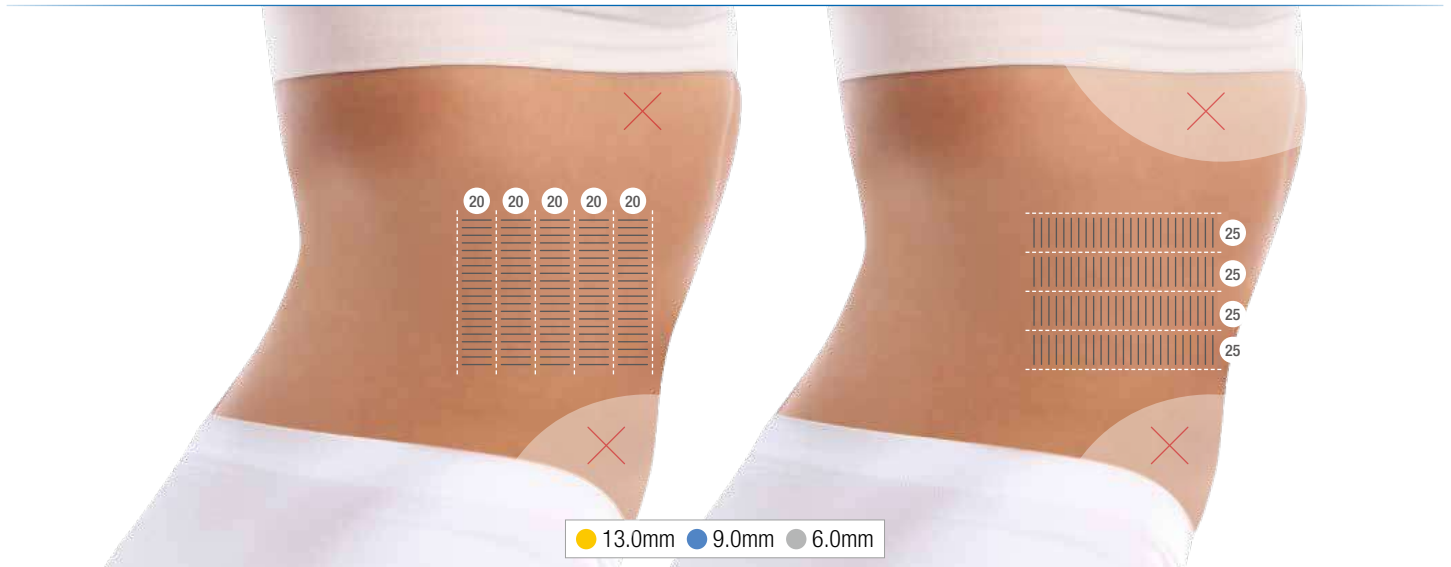


Cartridge	Energy	Pitch	Passes	Shots
● 13.0mm	1.0-1.8J	1.4mm	3	300 per side
● 9.0mm	1.0-1.9J	1.4mm	3	300 per side
● 6.0mm	0.8-1.3J	1.4mm	3	300 per side

- Do not apply treatment directly on the ribs or pelvis.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for contouring effect. It is important to choose the right patient for this treatment.



### Love Handles Contouring

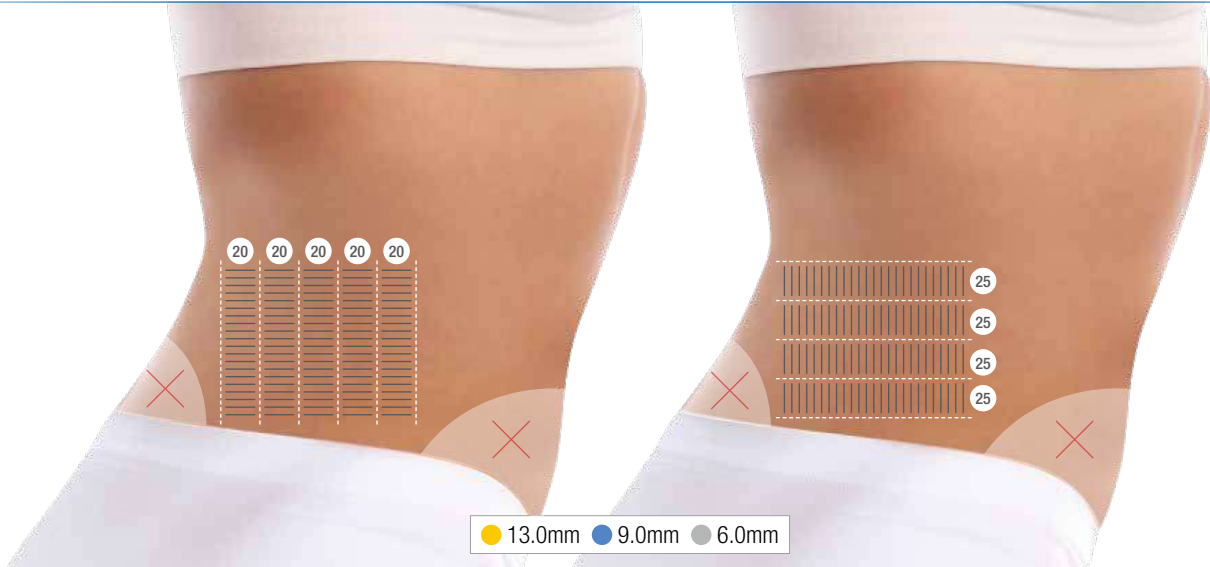

Cartridge	Energy	Pitch	Passes	Shots
13.0mm	0.8-1.3J	1.4mm	3	300 per side
9.0mm	0.8-1.5J	1.4mm	3	300 per side
6.0mm	0.6-1.2J	1.4mm	3	300 per side

- Do not apply treatment directly on the posterior superior iliac spine or the pelvis.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for a contouring effect. It is important to choose the right patient for this treatment.


### Knees Tightening and Contouring



Cartridge	Energy	Pitch	Passes	Shots
6.0mm	1.0-2.0J	1.4mm	4	120
4.5mm	0.6-0.9J	1.4mm	4	120
3.0mm	0.6-0.8J	1.4mm	4	120

- Do not apply treatment directly on the kneecaps.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for contouring effect. It is important to choose the right patient for this treatment.
- Knee treatments use a combination of face and body cartridges due to the size of tissue to be treated.


### Inner Thighs Contouring

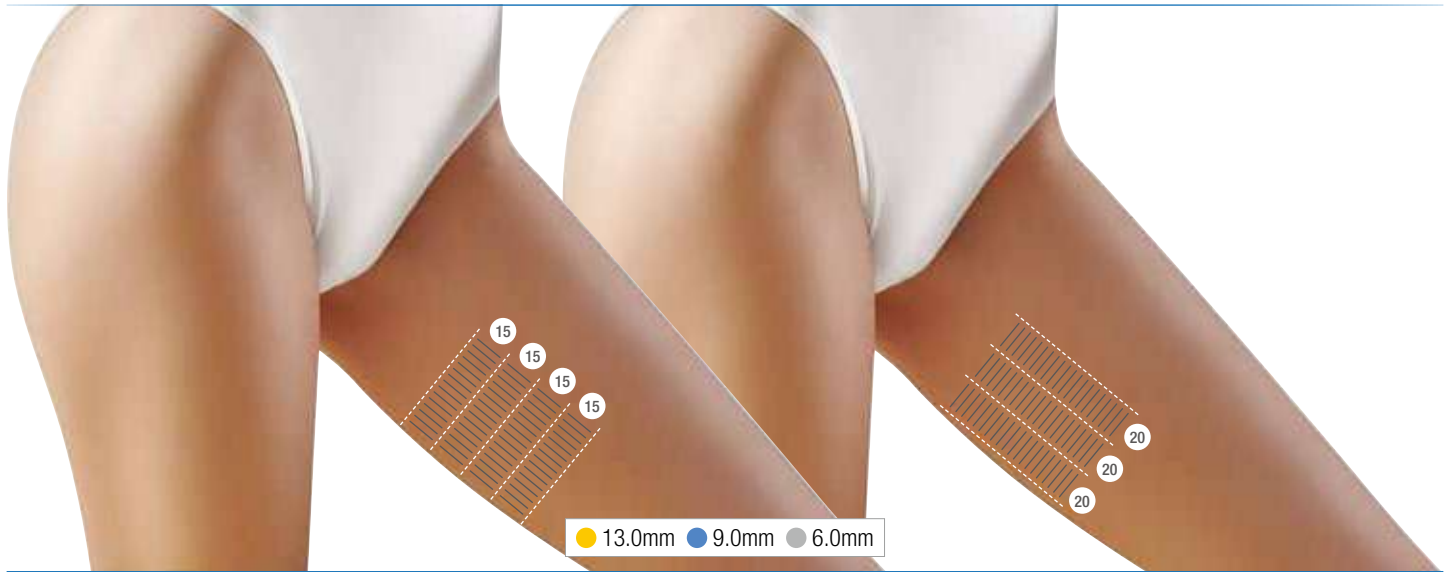

Cartridge	Energy	Pitch	Passes	Shots
● 13.0mm	0.5-1.2J	1.4mm	3	180 per side
● 9.0mm	1.0-1.5J	1.4mm	3	180 per side
● 6.0mm	1.0-1.5J	1.4mm	3	180 per side

- Do not apply treatment directly on the sciatic nerve.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for a contouring effect. It is important to choose the right patient for this treatment.


### Outer Thighs Contouring

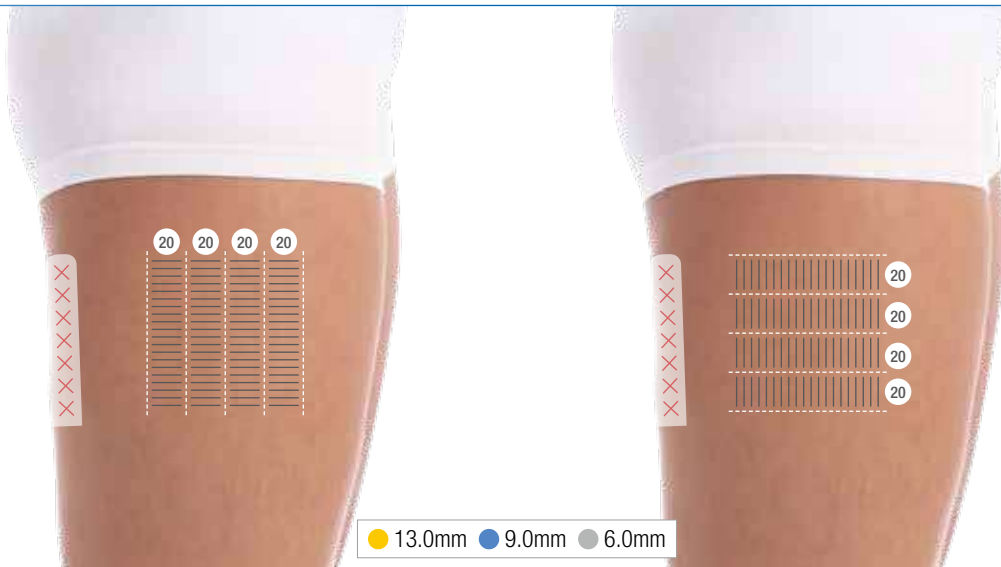

Cartridge	Energy	Pitch	Passes	Shots
● 13.0mm	0.8-1.5J	1.4mm	3	240 per side
● 9.0mm	1.0-2.0J	1.4mm	3	240 per side
● 6.0mm	1.0-2.0J	1.4mm	3	240 per side

- Do not apply treatment directly on the sciatic nerve.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for a contouring effect. It is important to choose the right patient for this treatment.


### Upper Arms Tightening & Contouring

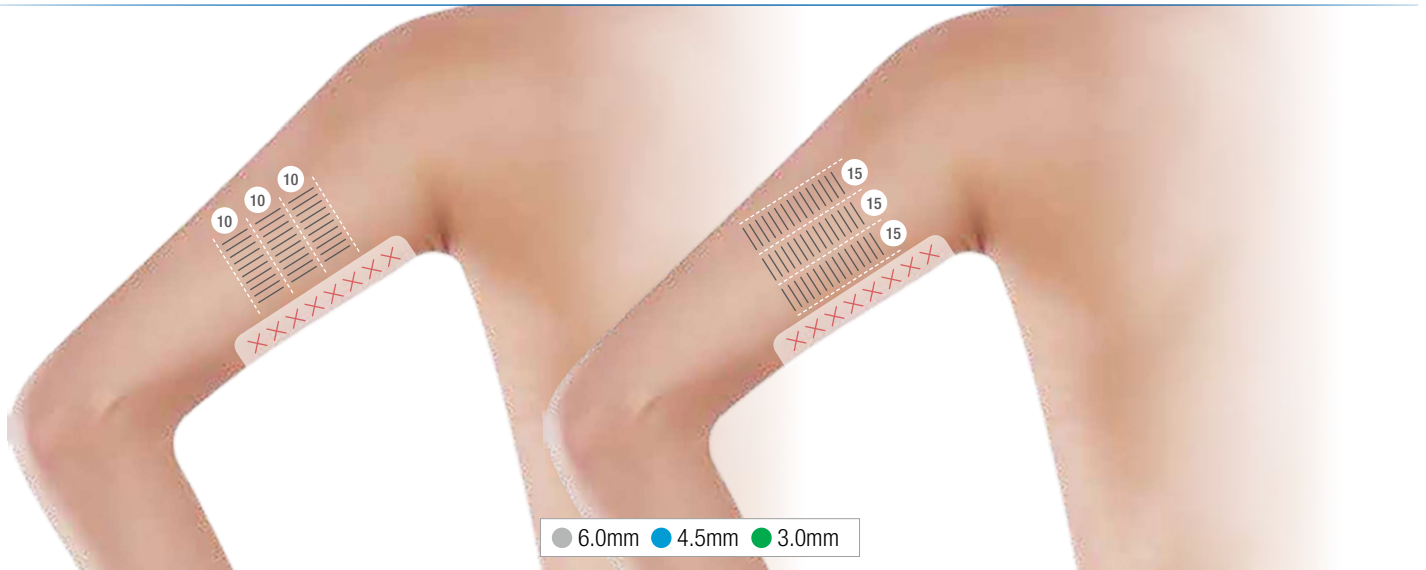

Cartridge	Energy	Pitch	Passes	Shots
6.0mm	0.8-1.5J	1.4mm	2	75 per side
4.5mm	0.6-1.0J	1.4mm	2	75 per side
3.0mm	0.6-0.9J	1.4mm	2	75 per side

- Do not apply treatment directly on the ulnar nerve.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for a contouring effect. It is important to choose the right patient for this treatment.
- Upper arm treatments use a combination of face and body cartridges due to the size of tissue to be treated.
- Omit 6.0mm cartridge if patient is slim or has thin tissue, use 2.0mm cartridge instead, or if laxity is a concern.

### Axilla Tightening and Contouring

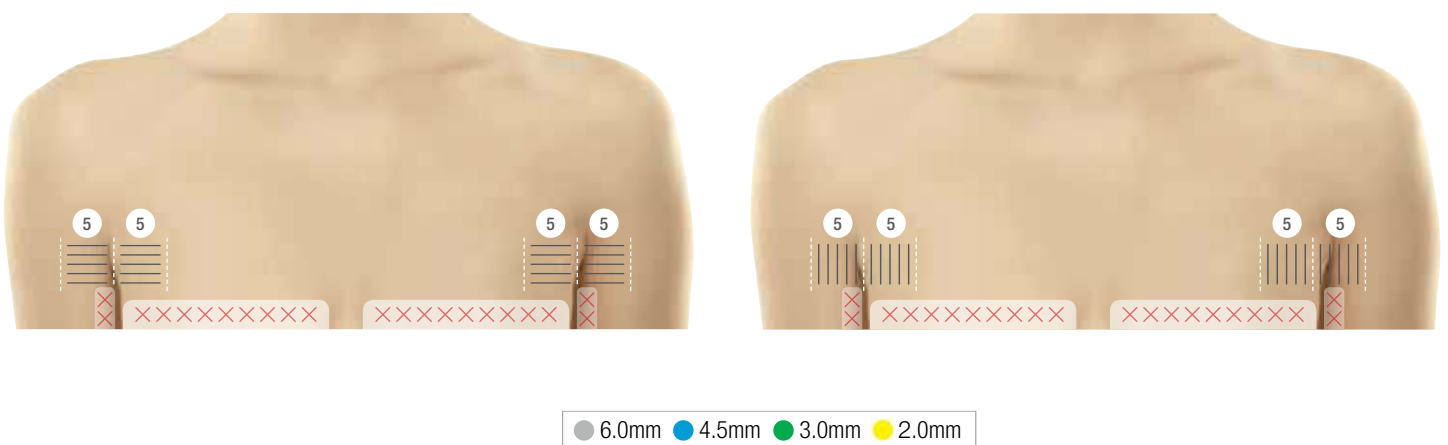

Cartridge	Energy	Pitch	Passes	Shots
6.0mm	0.6-0.9J	1.4mm	4	80
4.5mm	0.5-0.8J	1.4mm	4	80
3.0mm	0.5-0.8J	1.4mm	4	80
2.0mm	0.1-0.2J	1.4mm	4	80

- Do not apply treatment directly on breast tissue or the ulnar nerve.
- When treating multiple layers, treat from deep to shallow in sequential order.
- Start with lower energy and work your way up.
- Body cartridges work on subcutaneous micro-fat layer reduction for contouring effect. It is important to choose the right patient for this treatment.
- Axilla treatments use a combination of face and body cartridges due to the size of tissue to be treated
- Omit 6.0mm cartridge if patient is slim or has thin tissue, or if laxity is a concern, use 2.0mm instead.
- Same parameters apply to back axilla.