


# QUICK OPERATION GUIDE



# SKIN BOOSTER II<sup>®</sup>

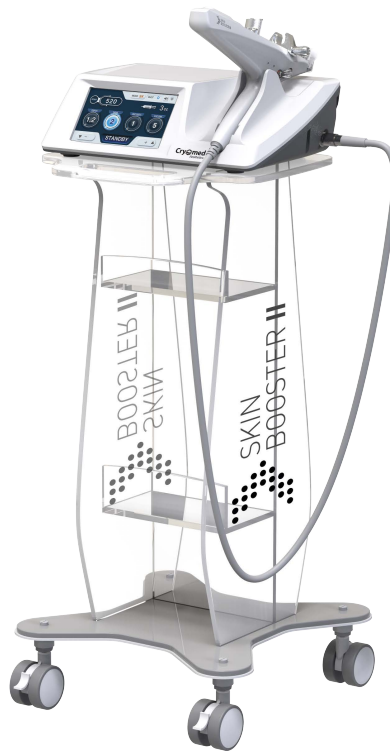

## Precautions for the procedure

### Patient series suitable for Skin booster II

- People who lack moisture in their skin
- People who want to improve fine lines on their skin
- People who want to increase skin elasticity
- Those who want to manage or prevent hair loss
- Clinical results may vary depending on the patient's condition, number of procedures, and drugs

## Precautions for the procedure

### Pain management and prohibited matters

Skin booster II procedure requires anesthesia.

During the procedure, the patient may feel some discomfort.

The feeling when the injector is operated is the feeling that the skin is pulled by a vacuum.

When the needle is inserted and the drug is injected, you may feel tingling or stiff.

Bruising or swelling may occur after the procedure



Prohibit the treatment if the patient to be treated with this device has the following conditions. Even if you do not correspond to the prohibition of the procedure listed below, perform the procedure after confirming whether the procedure is possible through sufficient consultation with the patient.

- Keloid patient
- Those with extremely sensitive skin
- Those with infectious diseases such as scar or suppuration on the skin
- Diabetic
- Pregnant woman
- Nursing
- Those with an anesthetic drug allergy
- Those with skin disease
- Malignant tumor

## Post-procedure care

### Recommendations post-procedure

#### Treatment procedure

Skin booster II procedure is invasive and requires the use of anesthetic ointment.

During the procedure, it may be accompanied by bleeding and tingling pain.

Since there may be a burning sensation after the treatment, cryo or ice pack is recommended after the treatment.

Bruising or swelling may occur after the procedure, and in some cases, the medicine may be taken at the discretion of the medical staff.

※ Cryo

Cleanly disinfected handpiece without applying the product to the skin,

Soothe lightly as stamping method.

## Treatment Procedure

### Injected drug and treatment guide by area

#### Cheek

Hyaluronic acid (non-crossing) standard



dose	Depth	vacuum
5~10	1.2~1.6	4


- ※ Precautions : Proceed in the vector direction based on the jaw. After the vacuum is completely applied to the skin, lift it slightly and pull the trigger to perform the procedure.  
If the trigger is pulled without the vacuum being completely applied, the skin's tightness may fall and loss may occur.

## Treatment Procedure

## Injected drug and treatment guide by area

**Around the eyes**

Hyaluronic acid (non-crossing) standard

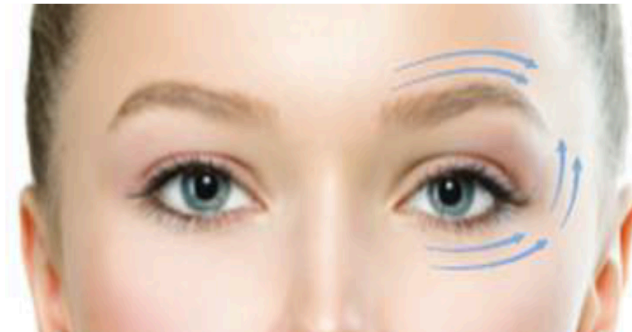


Dose	Depth	Vacuum
5~	1.0~1.2	3

- ※ Precautions : The area around the eyes is sensitive to pain, so be careful during the procedure.  
 Vacuum marks may remain in areas with thin skin during the procedure.

**Forehead**

Hyaluronic acid (non-crossing) standard

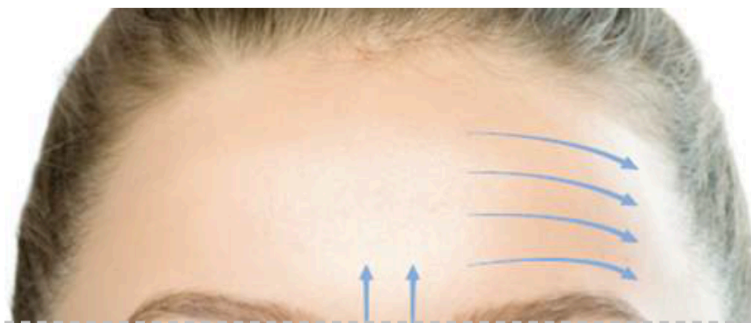


Dose	Depth	Vacuum
5	1.0~1.2	4

## Treatment Procedure

## Injected drug and treatment guide by area

**Neck**

Hyaluronic acid (non-crossing) standard

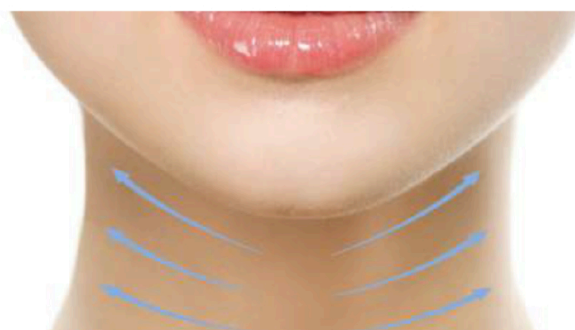


Dose	Depth	Vacuum
5	1.2-1.5	3

※ Precautions : Special care is required in the thyroid cartilage treatment. It is recommended to use the depth within 1.5mm. There is a possibility of bruising or swelling in the neck area with thinner skin than the face area and less sebaceous glands.

**Décolleté**

Hyaluronic acid (non-crossing) standard

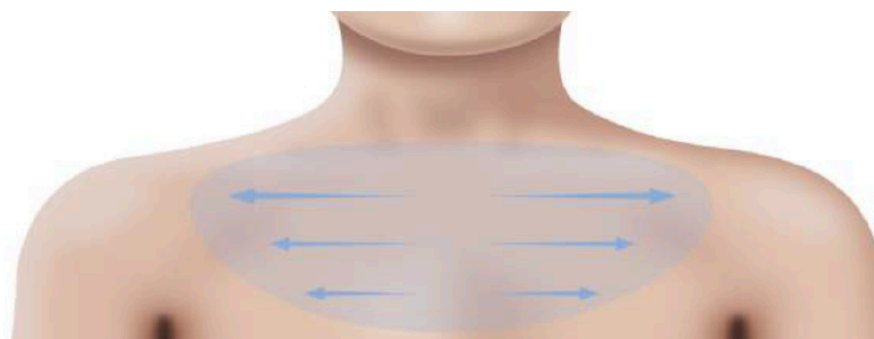

Dose	Depth	Vacuum
5~10	1.0-1.5	3


※ Precautions : It is recommended that the depth of the needle is within 1.5mm.

Treatment Procedure

Injected drug and treatment guide by area

Dorsum of hand

Hyaluronic acid (non-crossing) standard

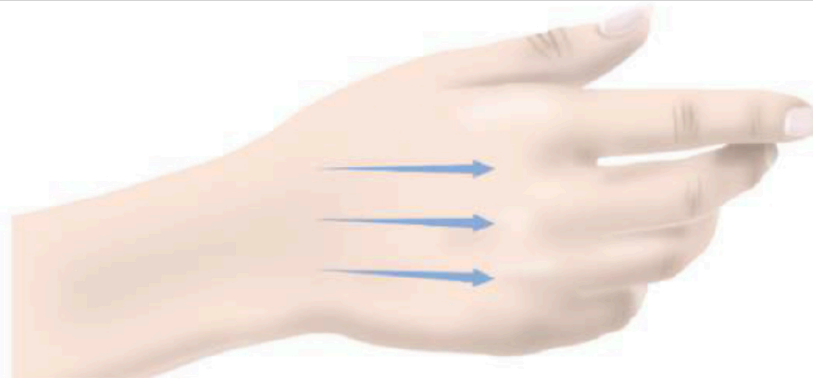


Dose	Depth	Vacuum
5~10	1.0-1.2	3

※ Precautions : After the vacuum is completely applied to the skin, lift it slightly and pull the trigger to perform the procedure.  
 If the trigger is pulled without the vacuum being completely applied, the skin's tightness may fall and loss may occur.  
 However, it is performed by avoiding the veins of dorsum of hand.

Scalp

Drug for the scalp only

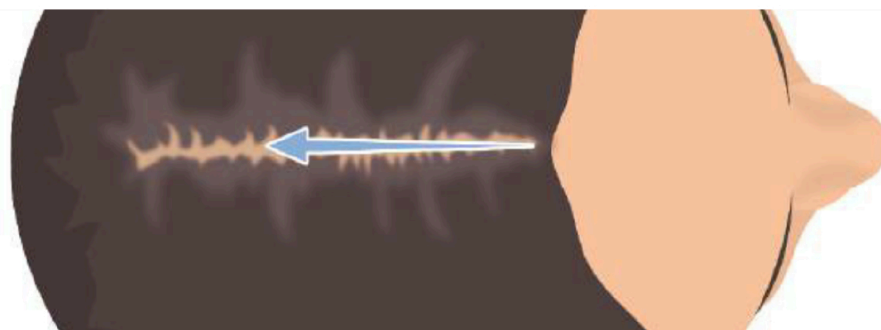

Dose	Depth	Vacuum
10~	1.0-	0

※ Precautions : Set the vacuum to Level 0 and press it slightly to perform the treatment,

## Treatment Procedure

### Recommendations for the procedure

During the procedure, perform in order to the needle's hub (border) is overlapped.

\*Dose and depth are controlled and treated according to the type of drug.

\*Needle depth and vacuum level are adjusted according to the condition of the patient's skin. (Vacuum level default : Level 4)

#### **Hyaluronic acid (non-crossing) standard**

Treatment example) Around the eyes, Neck : Depth 1.0~1.2mm - Vacuum Level 3

Cheek : Depth 1.2~ 1.6mm - Vacuum Level 4

Forehead : Depth 1.0~1.2mm - Vacuum Level 4

#### Recommendations

During the procedure, perform in order to the needle's hub (border) is overlapped.



\* After the procedure, use cryo or an ice pack to calm down.

\* Be sure to explain the precautions to the patient after the procedure.

# Drug Dosage

\* Available shots by gold standard syringe.

## Intermittent Mode [Preset : 10 µl]

Type	Dose	Available Shot	Dose	Available Shot
<b>1 cc Syringe</b>	7 µl (1.4µl/pin)	143	13 µl (2.6µl/pin)	77
	8 µl (1.6µl/pin)	125	14 µl (2.8µl/pin)	72
	9 µl (1.8µl/pin)	111	15 µl (3.0µl/pin)	67
	10 µl (2.0µl/pin)	100	20 µl (4.0µl/pin)	50
	11 µl (2.2µl/pin)	90	25 µl (5.0µl/pin)	40
	12 µl (2.4µl/pin)	83	30 µl (6.0µl/pin)	33

## Intermittent Mode [Preset : 10 µl]

Type	Dose	Available Shot	Dose	Available Shot
<b>2 cc Syringe</b>	7 µl (1.4µl/pin)	285	13 µl (2.6µl/pin)	153
	8 µl (1.6µl/pin)	250	14 µl (2.8µl/pin)	142
	9 µl (1.8µl/pin)	222	15 µl (3.0µl/pin)	133
	10 µl (2.0µl/pin)	200	20 µl (4.0µl/pin)	100
	11 µl (2.2µl/pin)	181	25 µl (5.0µl/pin)	80
	12 µl (2.4µl/pin)	166	30 µl (6.0µl/pin)	66


## Intermittent Mode [Preset : 10 µl]





Type	Dose	Available Shot	Dose	Available Shot
<b>3 cc Syringe</b>	7 µl (1.4µl/pin)	428	13 µl (2.6µl/pin)	230
	8 µl (1.6µl/pin)	375	14 µl (2.8µl/pin)	214
	9 µl (1.8µl/pin)	333	15 µl (3.0µl/pin)	200
	10 µl (2.0µl/pin)	300	20 µl (4.0µl/pin)	150
	11 µl (2.2µl/pin)	272	25 µl (5.0µl/pin)	120
	12 µl (2.4µl/pin)	250	30 µl (6.0µl/pin)	100

## Intermittent Mode [Preset : 10 µl]

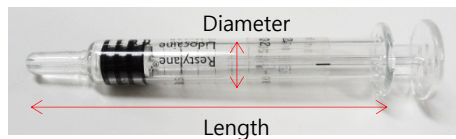Type	Dose	Available Shot	Dose	Available Shot
<b>5 cc Syringe</b>	7 µl (1.4µl/pin)	714	13 µl (2.6µl/pin)	384
	8 µl (1.6µl/pin)	625	14 µl (2.8µl/pin)	357
	9 µl (1.8µl/pin)	555	15 µl (3.0µl/pin)	333
	10 µl (2.0µl/pin)	500	20 µl (4.0µl/pin)	250
	11 µl (2.2µl/pin)	454	25 µl (5.0µl/pin)	200
	12 µl (2.4µl/pin)	416	30 µl (6.0µl/pin)	166

### [Gold Standard Syringe Size]



Syringe Type	Length	Diameter
1 cc 	72.5 ± 0.1mm	6.3 ± 0.1mm
2 cc 	96.2 ± 0.1mm	8.6 ± 0.1mm
3 cc 	83.3 ± 0.1mm	9.3 ± 0.1mm
5 cc 	92.3 ± 0.1mm	11.4 ± 0.1mm

[Important] If you use a different size of syringe than above, the available shots may differ from the table above.





## Treatment Procedure

### Procedure guide for each drug injected


\* 9-pin standard

Type	Depth	Dose	
Restylane vital	1.6mm~	5 Dose	1cc standard 100~200 shots
Restylane vital light	1.4mm~	5~10 Dose	
Teosyal skin booster	1.2mm~	5~10 Dose	
Meso Therapy	1.0mm~1.2mm	5~10 Dose	-
Scalp	1.0mm~	10 Dose~	-

- Depth adjustment is necessary according to the skin type

Other available Drug components	Skin HA, PRP, Vitamin, Peptide, Skin Booster, Derma Toxin
	Scalp Mix Injection (Meso), Peptide, PRP

PATIENT NAME: \_\_\_\_\_

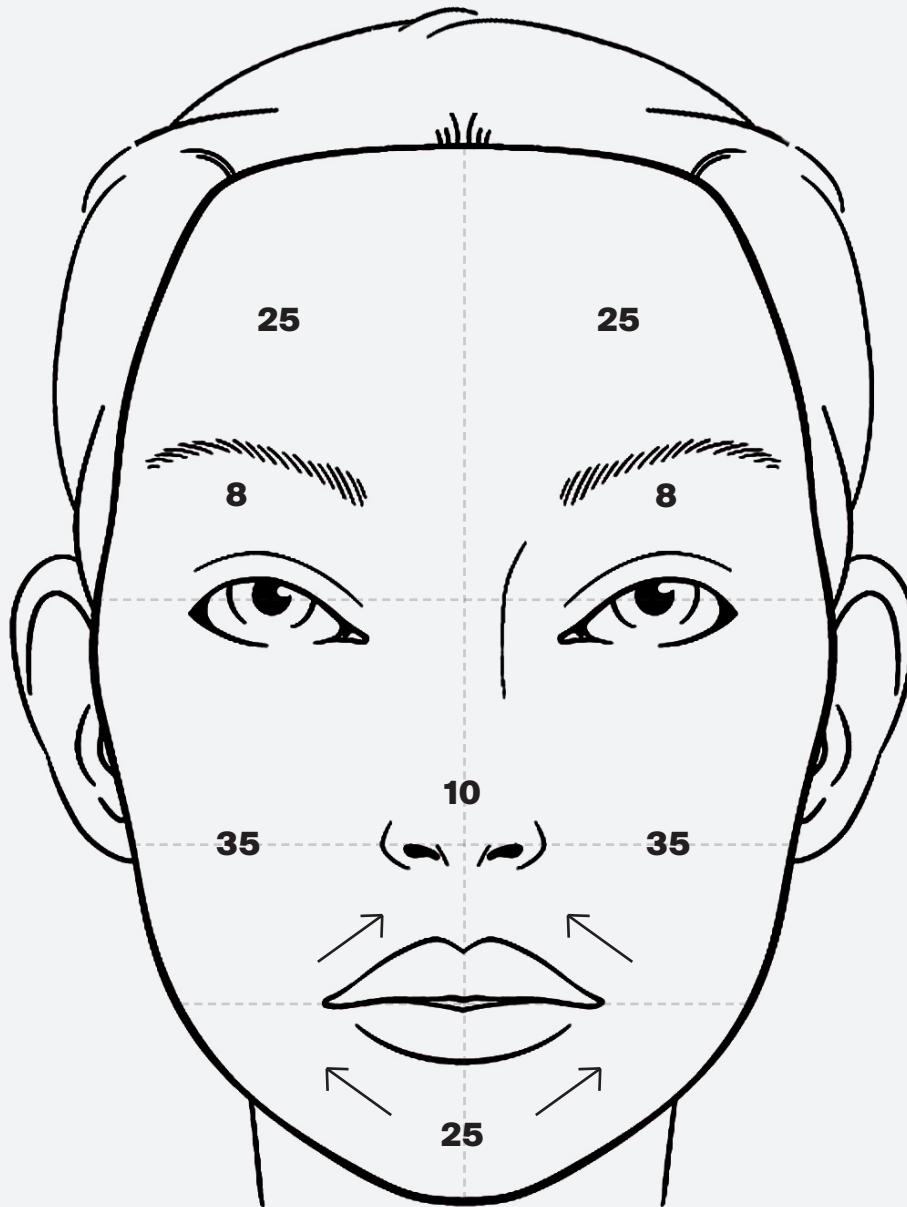

Neck - 80 Shots  
Decolletage - 150 shots - 3.0mL

TREATMENT NOTES:

FACE ONLY @ 20 ul = 171 SHOTS = 3.4mL - 5mL Syringe

NECK AND DECOLLETAGE @ 20ul = 230 SHOTS = 4.6mL - 5mL Syringe

