

Secret DUO

NON-ABLATIVE SKIN RESURFACING WITH 1540nm

Indication	Energy	Spacing	Stack	Passes	Mode
LEVEL 1; LIGHT Active Acne Texture	8-10mJ	0.7-0.8	1	2-4	0.2-0.5s
LEVEL 2; MEDIUM Mild Aging Skin Tightening	10-12mJ	0.7-0.9	1	2-3	0.2-0.5s
LEVEL 3; INTENSE Photo Aging Wrinkles	12-14mJ	0.8-1.0	1	1-3	0.8-1.0s
LEVEL 4; ADVANCED Atrophic Scars Acne Scars	14-16mJ	1.0-1.5	1	1-3	0.8-1.0s

TREATMENT NOTES:

- Treatment indicated for non-ablative skin resurfacing, Acne scars, Atrophic scar, Active acne, Wrinkle reduction, Skin lifting and tightening, Tone and texture improvement.
- The non-ablative fractional laser beams are delivered as multiple columns in the dermis and primarily targeted to dermal water, which will induce collagen heating and dermal remodelling with no ablative effect on the epidermal layer.
- Use Numbing cream and cold air to assist with patient tolerance
- Skin Types I-VI use caution with higher skin types with more cooling and conservative treatment.
- 1-3 treatments recommended 4 weeks apart