

# ART FILLER®



#### / LIP VOLUME & CONTOUR



ART FILLER® FINE LINES




ART FILLER® UNIVERSAL



ART FILLER® LIPS 1ML



VOLUME

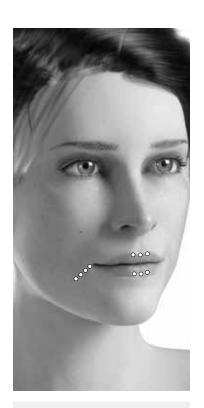

## **INDICATIONS**

- > Lip volume
- > Lip enhancement
- > Marionette lines

#### / FEATURES OF THE GEL

- > Elastic & volumizing lip product: suitable to give lip pronounced volume or projection
- > Also suitable for light projection of the tissues

#### / INJECTION PLAN

27G 1/2" needle

Labial mucosa (expert injection) and medium to deep dermis

## / INJECTION TECHNIQUE

### / RECOMMENDATIONS

- > To be injected slowly, deep and in small amounts
- > Massage thoroughly after injection

#### / RESULTS



BEFORE



AFTER



BEFORE



AFTER



## LIPS RHEOLOGICAL PROFILE


#### / FORMULATION (1ML SYRINGE)

CONCENTRATION	CROSS-LINKING	FREE HA	LENGTH	COHESIVITY
HA (MG/ML)	RATE	QUANTITY	OF HA	
25	•••	••00	•••	••••

<sup>+</sup> LIDOCAINE 0.3%


#### / TRI-HYAL® TECHNOLOGY

#### / LIPS PROPERTIES

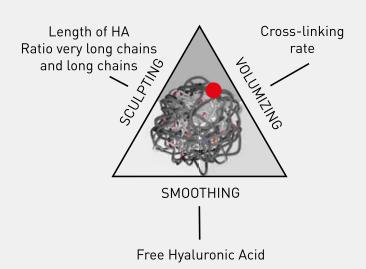


- > Good volumizing and sculpting potential
- > Cohesive product, easy to shape

#### **ELASTICITY G' (Pa)\***

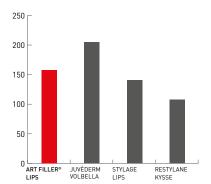


- > High elasticity: good volumizing effect
- > Natural dynamic outcome

#### FLUIDITY TAN δ\*\* (%)

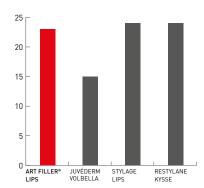

> Very high Tan delta for a good positioning in tissues

## RESISTANCE TO DESTRUCTURATION CROSS-OVER T (Pa)

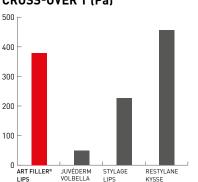

- > One of the highest for a late destructuration, important for lip movement
- > Long lasting results