

# ROBOLEX


Multi Functional RF Device

## Quick Manual



## Detail structure

### ▣ Handpiece

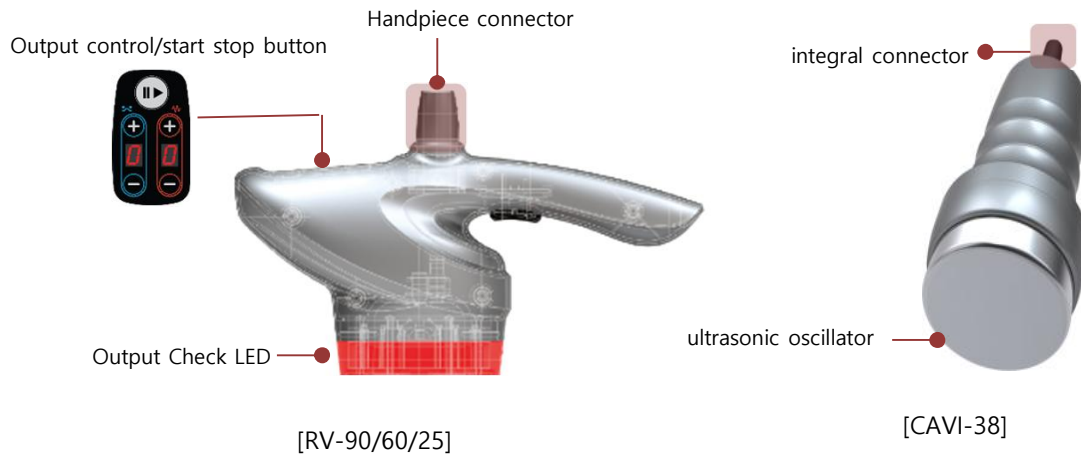


- Endomology Handpiece consists of three types of RV90/60/25 .
- Use one connection line. Connect for the handpiece you want to use and boot it.
- Use the handpiece top output control and start stop button. (procedure mode x)
- Endomology simultaneously implements multipolar high-frequency/low-output risers/negative pressure functions.
- The LED blinks and notifies you when the output is being emitted. (655nm Low Level Laser x)
- 38Khz ultrasonic waves are investigated through the cavitation oscillator.

### ▣ Type of Handpiece

Handpiece	Endormology			Cavitation
Size	RV-25	RV-60	RV-90	CAVI-38
Area of use	For face	For calf/arm	For abdomen&thigh	For body

## HOW TO USE

### RV-UI

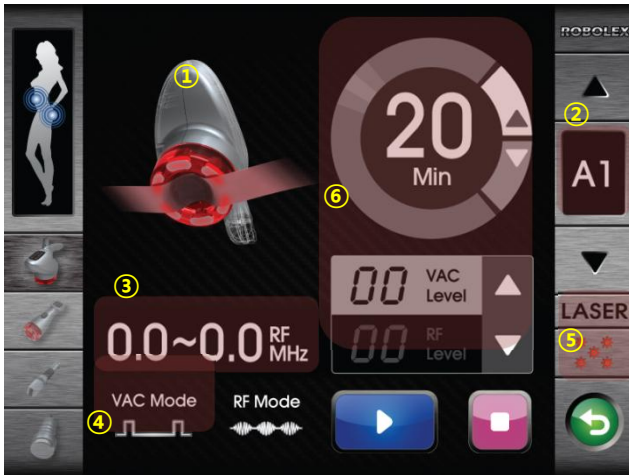

### CAVI-UI



1. Touch the application on the equipment initialization screen to move it to the corresponding treatment screen.
2. ① 1 to 3 Mhz frequency varies according to handpiece RV-90/60/25 size.
3. ② A1~A5 5 treatment modes are automatically saved and ③ frequency ④ sound pressure type ⑤ laser outputs are simultaneously connected.
4. ⑥ Adjust treatment time (0 to 60min)/negative pressure/high frequency (0 to 10Level) intensity
5. Five modes of CAVI function ⑦ A1 to A5 are saved by ⑧ treatment site (arm/calf/ abdomen/ thigh/hip).
6. ⑨ Adjusting the treatment time (0-60min) and output (10Level ↓ Recommended) power
7. The procedure screen can be moved through the picture icon on the left side of the procedure screen.

## HOW TO TREAT

### ▣ Protocol

Area	Handpiece	Auto mode & Level
Abdomen	90Ø	1. CAVI – A2 mode (BELLY) 5 Level ↓ (5~10min) 2. ① A2 mode (V4~5, R6~7) ② A4 (V4~5, R6~7) / Total 30min ↓
Hip	90Ø	1. CAVI – A3 mode (HIP) 5 Level ↓ (5~10min) 2. ① A2mode (V4~5, R 5) ② A4mode (V4~5, R 5) / Total 30min ↓
Thigh	60Ø	1. CAVI – A4 mode (THIGH) 5 Level ↓ (5~10min) 2. ① A2mode (V4~5, R 5) ② A4mode (V4~5, R 5) / Total 30min ↓
Leg	60Ø	1. CAVI – A5 mode (LEG) 5 Level ↓ (5~10min) 2. ① A2 mode (V4~5, R 5) ② A4 (V4~5, R 5) / Total 30min
Arm	60Ø	1. CAVI – A1 mode (ARM) 5 Level ↓ 2. ① A2 mode (V4~5, R 5) ② A4 (V4~5, R 5)
Back	90Ø 60Ø	1. ① A2 mode (V4~5, R 6) ② A4 (V4~5, R 6) / Total 30min
Face	25Ø	1. Select A4 or A5 ((V5 ↑, R 4~5) / Total 15min

- When applying two modes, the sound pressure type and frequency are stored differently depending on the automatic mode (A1 to A5) you can increase treatment effectiveness.
- You can use sound pressure and RF as a single function.
- The use of Cavitation may vary depending on the area of treatment.
- Micro vibration of the Cavitation may cause tinnitus when passed through the skin.
- Maintain a straight line of hands so that the handpiece contacts the skin, otherwise hard to use sound pressure carefully.
- Handle in the direction of the lymph nodes. Increases treatment effectiveness (toxic emissions, improved cellulite).
- Patient angles vary, so do not increase high frequency/negative pressure from the beginning.
- Due to the massage effect, pressure can sometimes occur 1 to 3 times, so the patient is advised in advance during the call.
- Depending on the combination procedure (invasive/non-invasive), the interval between treatments may vary. It is recommended twice a week on average.

## Q & A

---

---

### 1. What is the function of the equipment?

ROBOLEX is an all-in-one body contouring that combines 1~3Mhz high frequency, 650nm low power laser, sound pressure, and 38Khz cavitation functions into one. As a non-invasive device, it is widely used as a management device before and after liposuction.

### 2. What is the pain level, procedure interval, and recommended frequency?

As for the treatment feeling, the feeling of being massaged due to the physical sound pressure and the feeling of warmth from the RF heat are transmitted at the same time. The intensity can be adjusted according to the treatment mode/output intensity.

ROBOLEX is a non-invasive obesity device. The treatment interval is twice a week and the recommended frequency is at least 8 times.

### 3. What is the procedure time and price?

Our recommended time is about 30 minutes in total, and the treatment price per part is measured at least 70USD to 100USD. It may vary depending on the hospital/country/operation program used.

### 4. When will the treatment effect appear?

You can see that the size is reduced by about 1.5cm with just one treatment.

Since the skin tissue becomes flexible and shrinks due to RF heat and physical sound pressure, it should be performed at least 5 times to increase the lasting effect rather than a one-time procedure.

In most cases, the size reduction after RoboLEX treatment alone is minus 3cm-5cm.

### 5. Please tell me how to use it effectively.

The functions of RoboLEX are largely cavitation and endormology.

We recommend treating with cavitation first. The reason is that it plays a role in reducing the size of fat cells in response to the fat cell membrane clinically. And then, Endomology's high-frequency/low-power laser induces fat fusion, and physical sound pressure promotes lymph drainage, giving synergy effects on elasticity and size reduction.

Other medications/injections are also available to be treated together