

---

# SONOCARE PROGRAM GUIDE

for staff

---

A close-up photograph of a woman's face, showing her eyes, nose, and lips. Her hands are gently touching her cheeks. The background is a soft-focus bokeh of light circles.

  
sonocare

# MAIN MODE

No.	Program	Probe	Frequency	Level	Treatment time
1	Tightening	Blue	10, 3/10Mhz	Level 6	18mins
2	Wrinkle	Blue	10, 3/10Mhz	Level 6	12mins
3	Hydration	Blue	10, 3/10Mhz	Level 4	12mins
4	Sonophoresis	Blue	10, 3/10Mhz	Level 3	6mins
5	Acne	Blue	10, 3/10Mhz	Level 3	12mins
6	Soothing/Flush	Blue	10, 3/10Mhz	Level 2	12mins
7	Acne scar	Blue	10, 3/10Mhz	Level 4	18mins
8	Contusion	Blue	10, 3/10Mhz	Level 2	12mins
9	Tightening	Yellow	1, 3, 1/3Mhz	Level 6	15mins
10	Hydration	Yellow	3, 1/3Mhz	Level 4	15mins
11	Sonophoresis	Yellow	3, 1/3Mhz	Level 3	15mins

# USER MODE

No.	Program	Probe
1	1MHz	Yellow
2	3 MHz	Yellow
3	3 MHz	Blue
4	10 MHz	Blue
5	1/3 MHz	Yellow
6	3/10 MHz	Blue