

New Clatuu Shield 360 Gel Pad Standard Size Guide




The new Shield 360 gel pad is for use with all Clatuu hand pieces



Tear open the Shield 360 gel pad along the top from the tab, it is best to open 2 pads if 2 treatment areas are to be done.



Remove the Shield 360 gel pad and place on top of the packaging.

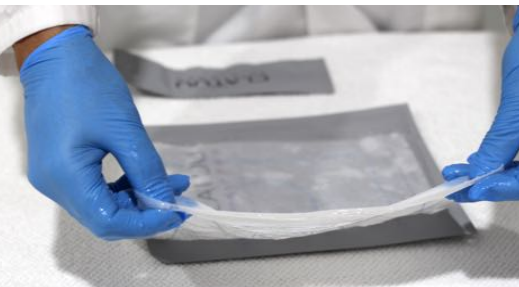

Remove the plastic mesh from between the folded Shield 360 gel pad.



Squeeze out any excess gel from the packaging onto the treatment area and spread evenly. Apply the new Shield 360 gel pad onto the treatment area. Ensure any air bubbles are carefully smoothed out.

* This guide also applies to Mini Shield 360 gel pads.