



QUICK PROTOCOL SHEET


THIGH

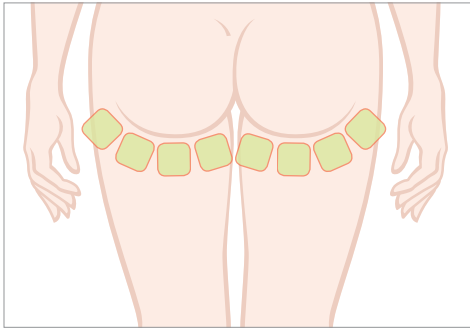


Cartridge	C9
Treatment Focus	Under Buttocks
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 40-40-40 Preset 2 : 40-35-30
Shots per side	4 sites x 3 passes
Total Shots	24

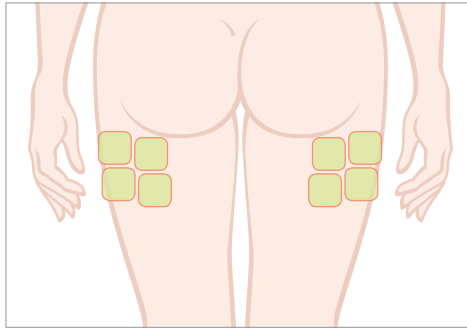


Cartridge	C9
Treatment Focus	Outer Thigh
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 40-40-40 Preset 2 : 40-35-30
Shots per side	4 sites x 3 passes
Total Shots	24

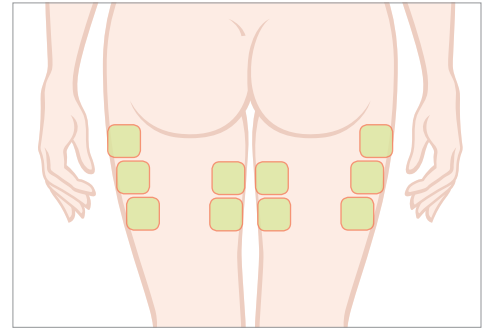


Cartridge	C9
Treatment Focus	Inner & Outer Thigh
Cooling Level	3-4
Fluence J/cm²	Preset 1 : Outer 40-40-40 Inner 30-30-30 Preset 2 : 40-35-30
Shots per side	5 sites x 3 passes
Total Shots	30

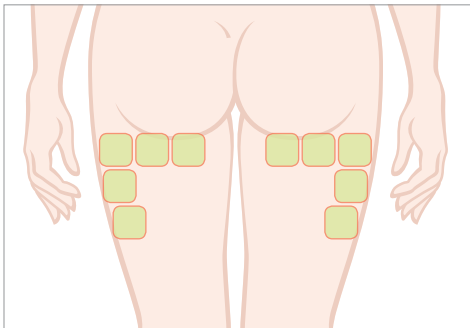

Cartridge	C9
Treatment Focus	Outer Thigh
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 40-40-40 Preset 2 : 40-35-30
Shots per side	5 sites x 3 passes
Total Shots	30

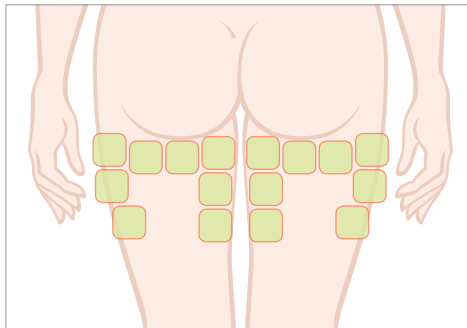

Cartridge	C9
Treatment Focus	Thigh
Cooling Level	3-4
Fluence J/cm²	Preset 1 : Upper 40-40-40 Lower 30-30-30 Preset 2 : 40-35-30
Shots per side	8 sites x 3 passes
Total Shots	48

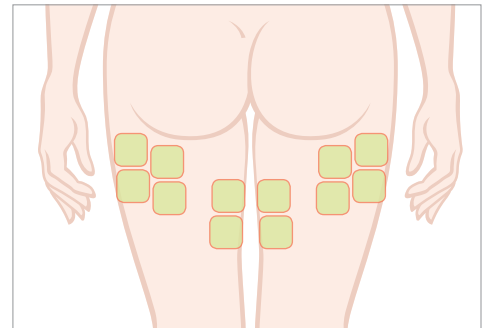


Cartridge	C9
Treatment Focus	Inner & Outer Thigh
Cooling Level	3-4
Fluence J/cm²	Preset 1 : Outer 40-40-40 Inner 30-30-30 Preset 2 : 40-35-30
Shots per side	6 sites x 3 passes
Total Shots	36


 **C9** : 9mm Cartridge **Sites** : Shot application squares **Passes** : Number of shots per site

ABDOMEN


- Use of the 9mm Scizer Cartridge (C9) is recommended for use only when patients comply with the 1.8 cm pinch test requirement
- Combinatory treatment is recommended for patients who comply with the 2.5 cm pinch test requirement.
- Additional Scizer treatment is recommended at least 30 days apart from the previous treatment

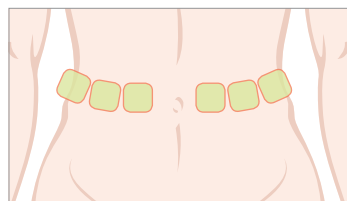


Cartridge	C9
Treatment Focus	Flanks
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	6 sites x 3 passes
Total Shots	18

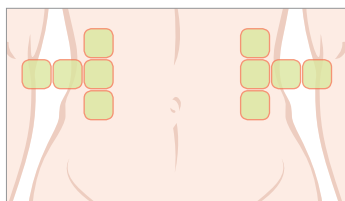


Cartridge	C9
Treatment Focus	Flanks
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	10 sites x 3 passes
Total Shots	30

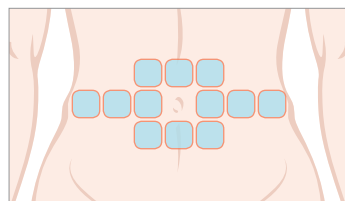


Cartridge	C13
Treatment Focus	Whole Abdomen
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	12 sites x 3 passes
Total Shots	36

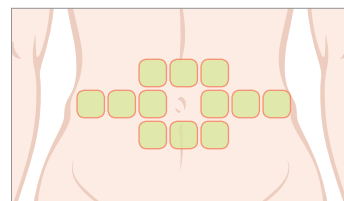


Cartridge	C9
Treatment Focus	Whole Abdomen
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	12 sites x 3 passes
Total Shots	36

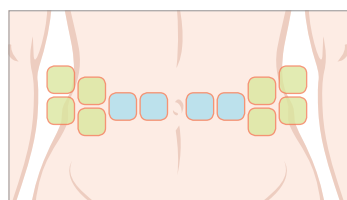


Cartridge	C9, C13
Treatment Focus	Flanks
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	C9 : 8 sites x 3 passes C13 : 4 sites x 3 passes
Total Shots	36

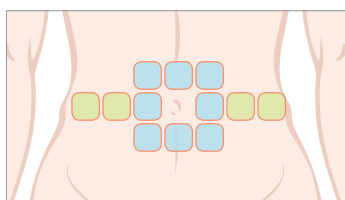

Cartridge	C9, C13
Treatment Focus	Whole Abdomen
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	C9 : 4 sites x 3 passes C13 : 8 sites x 3 passes
Total Shots	42

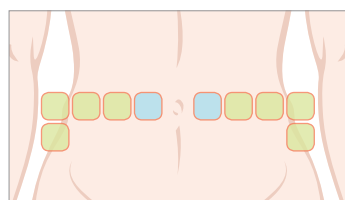

Cartridge	C9, C13
Treatment Focus	Love Handles
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	C9 : 8 sites x 3 passes C13 : 2 sites x 3 passes
Total Shots	36

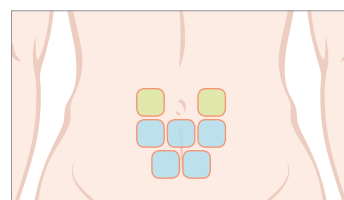

Cartridge	C9, C13
Treatment Focus	Mid Abdomen
Cooling Level	3-4
Fluence J/cm²	Preset 1 : 50-50-50 Preset 2 : 50-45-40
Shots	C9 : 2 sites x 3 passes C13 : 5 sites x 3 passes
Total Shots	42

■ C13 : 13mm Cartridge
 ■ C9 : 9mm Cartridge
 Sites : Shot application squares
 Passes : Number of shots per site