

AQUAPURE

Treatment Guide



Contents

Introduction

Patient Selection

AQUAPURE Graphical User Interface (GUI)

AQUAPURE Hand-pieces

AQUASOLUTION Range

Treatment Guidelines

Maintenance Tips

Clinical Photos

Introduction to AQUAPURE



- AQUAPURE is a non-invasive, multifunctional facial care device that treats a variety of skin conditions in patients with diverse skin profiles.
- Each AQUAPURE Hand-piece works to promote skin tightness and rejuvenation with skin nourishing solutions that delivers immediate results.

*Individual results may vary based on the skin conditions concerned



Patient selection

- Good for all skin types
- Treatment appropriate for both men and women
- Treats the following skin conditions:
 - Uneven skin tone
 - Sun damage/Hyperpigmentation
 - Dehydrated skin
 - Oily and acne-prone skin
 - Fine lines and wrinkles



Contraindications

Do not treat anyone with the following symptoms.

This system is contraindicated for patients with:

- Pregnancy or when breastfeeding
- Implanted electrical and metal devices
- Severe or cystic acne
- Skin allergies to AQUAPEEL's solutions that may react during treatment, including lactic acid, salicylic acid or aspirin.
- Open wounds, abrasions, and inflammation of the face
- Allergy to chemical peels



Precautions

Consult with medical experts to decide whether treatment is appropriate for the following conditions:

- Type 1 Diabetes
- Epilepsy and seizures
- Skin conditions caused by autoimmune diseases, skin cancer, and Herpes Simplex
- Cardiovascular diseases such as heart attacks and strokes
- Oral, topical or invasive treatments to skin prior to AQUAPEEL treatment
- Patch test 24 – 48 hours prior to treatment if reaction is a concern
- Surgical procedures done within 40 days
- Patients with bee or salicylic acid allergies.




Potential side effects

Patients may experience varied tolerance levels of pain and discomfort during the treatment procedure.

- Post procedure, the following side effects and risks from receiving the AQUAPEEL treatment may include redness, slight tightness, and swelling in treated areas which may last from a few hours to a couple of days following treatment. Scarring of the treated area, though rare, is also a risk.
- **Erythema (Redness):** The patient may show signs of mild redness in treated areas after treatment that will typically resolve within 48 hours.
- **Edema (Swelling):** The treatment area may show signs of mild swelling following treatment that will typically resolve within 48 hours.
- **Nausea and drowsiness:** Patients receiving electromagnetic stimulation on facial areas during treatment may experience nausea and drowsiness.



Skin Anatomy

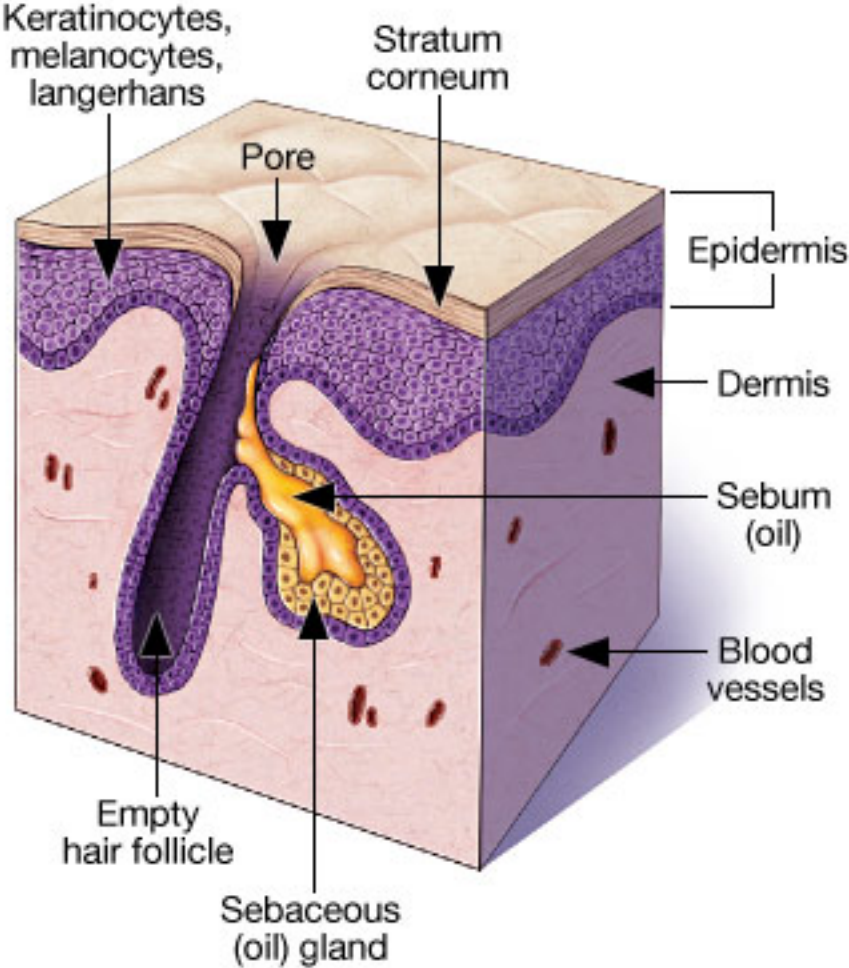


AQUAPURE Graphical User Interface (GUI)




Graphical User Interface (GUI)

AQUAPURE
Main Interface

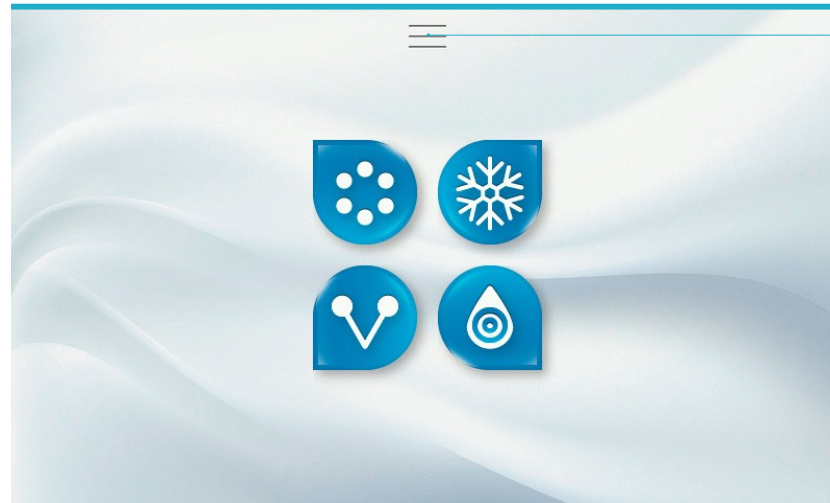

Menu Bar Icon
for Settings

AQUAPURE
System Preferences

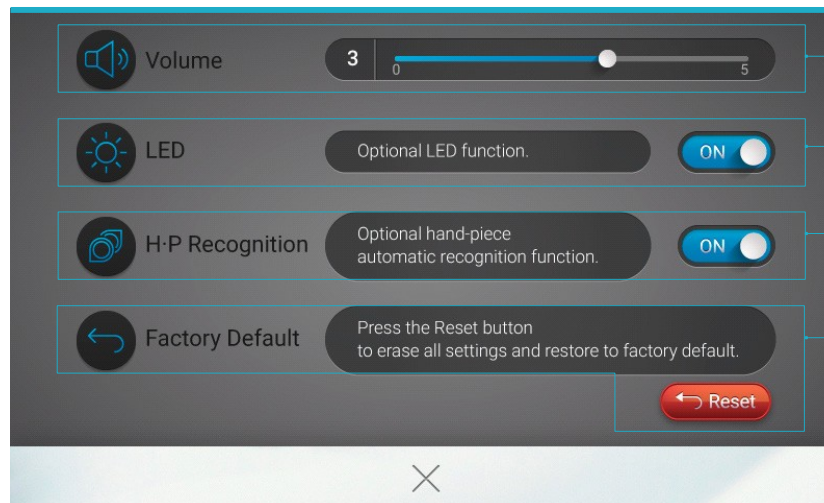

Volume
Adjust from Levels 0-5

LED Function ON/OFF

Hand-piece Automatic
Recognition Function
ON/OFF

Reset Factory Default


Graphical User Interface (GUI)

AQUAPEEL
(PEEL+/SEBO+/REJUVE+)




Graphical User Interface (GUI)

AQUAPEEL
(CLEANSYS)

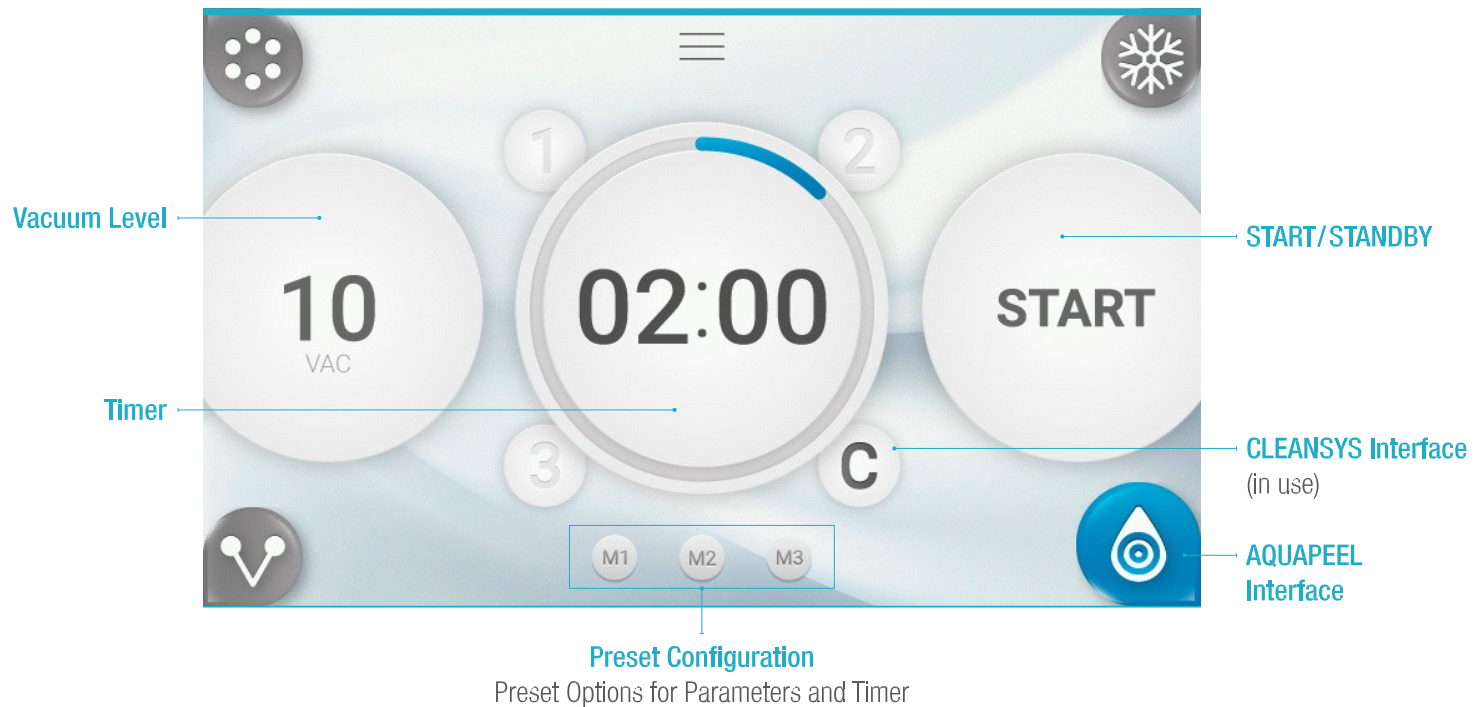


Graphical User Interface (GUI)

ELECTROPORATION

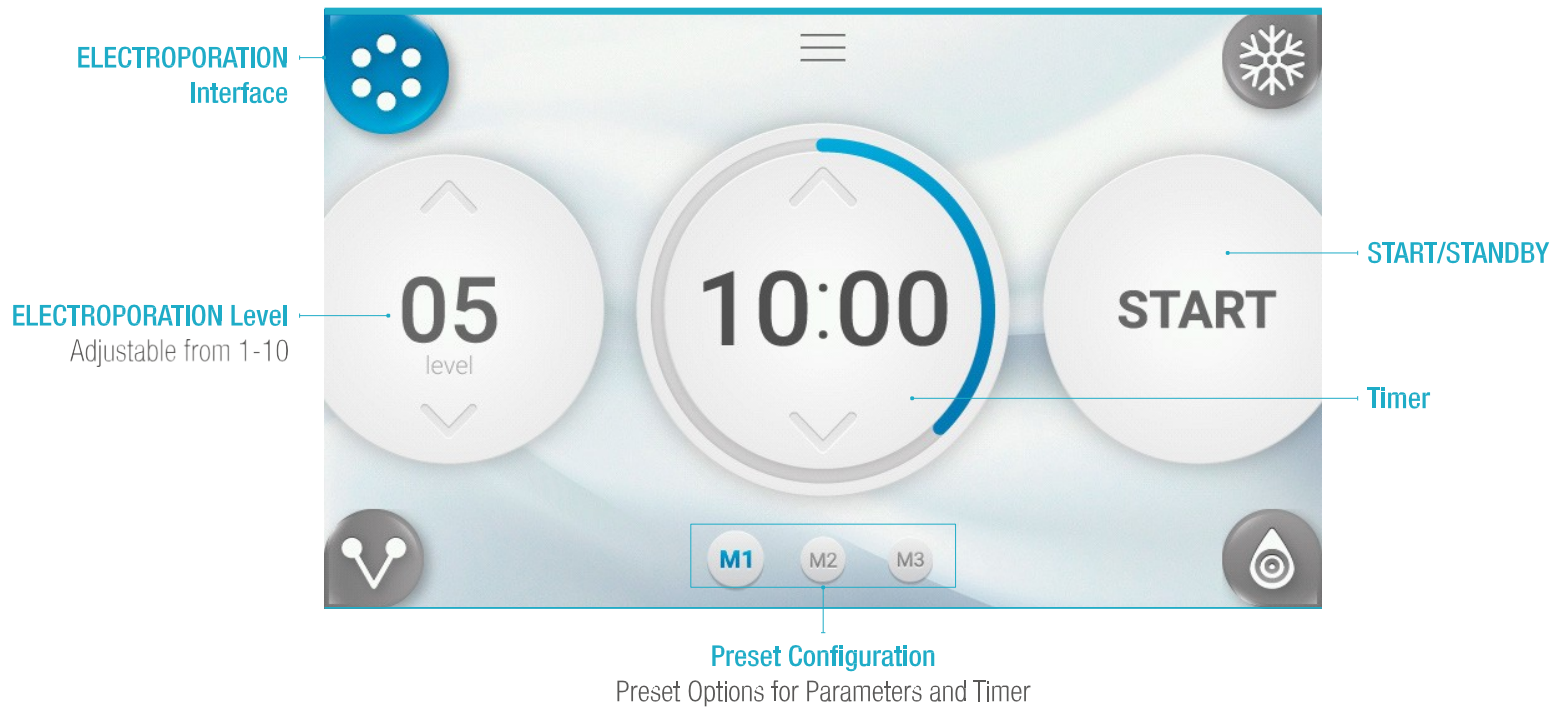


Graphical User Interface (GUI)

MICRO-CURRENT

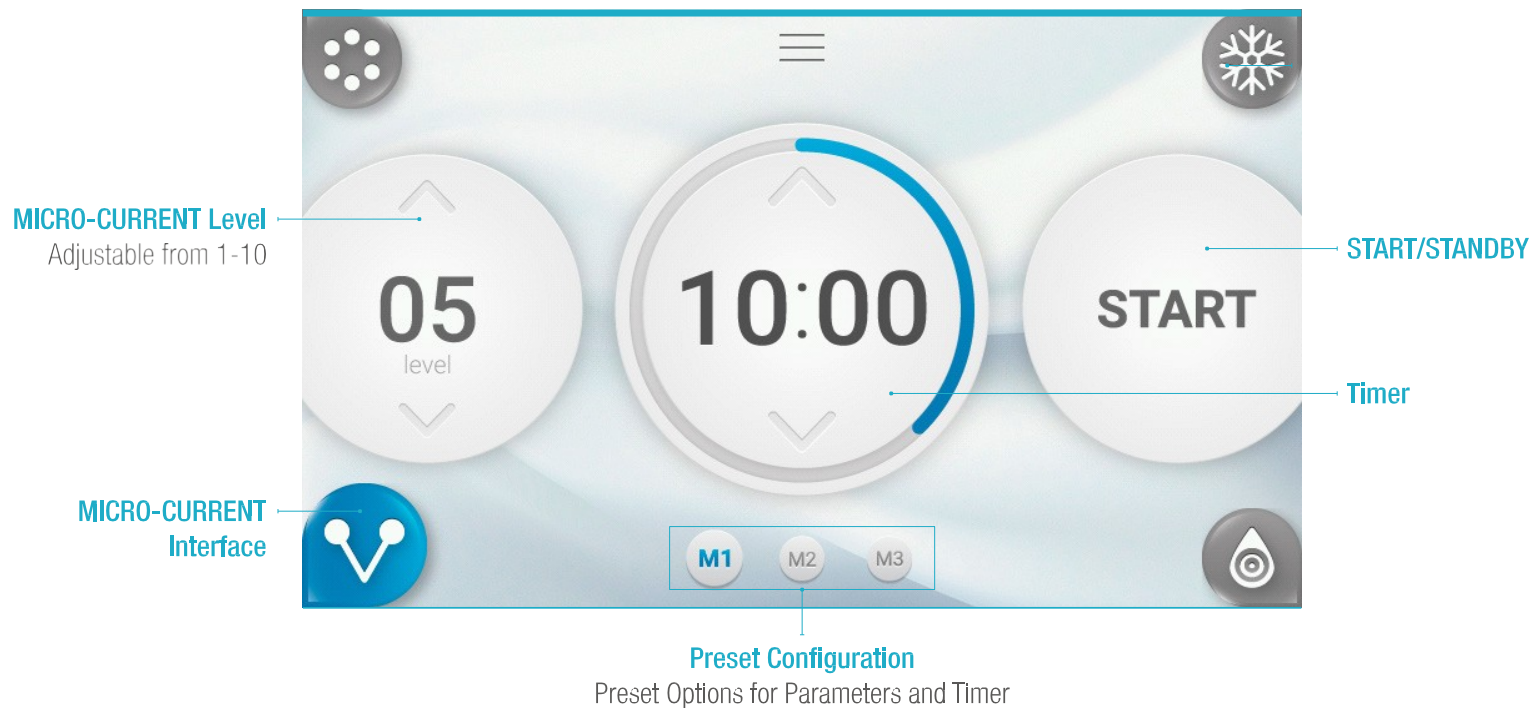

Graphical User Interface (GUI)

COOLING/HEATING

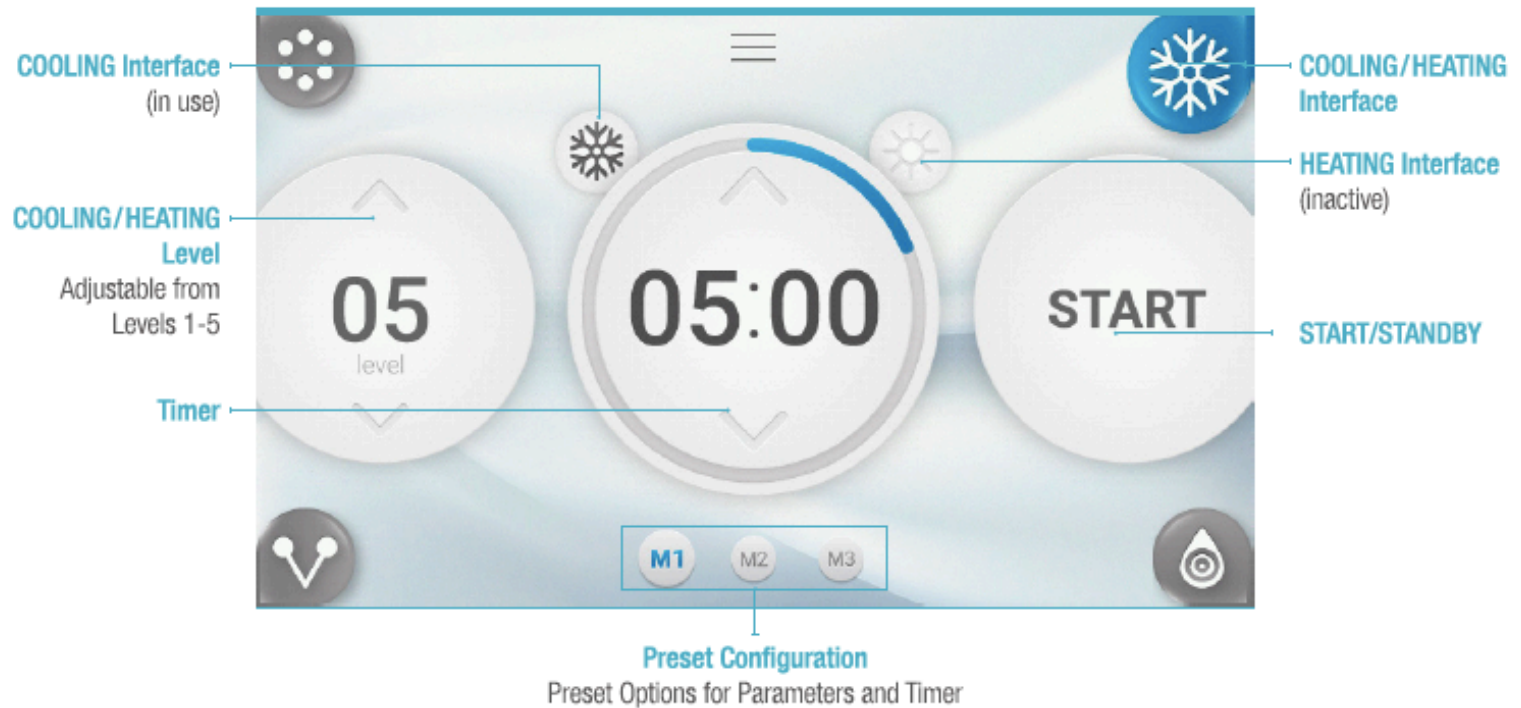

AQUAPURE Hand Pieces



AQUAPEEL

- Removes dead skin cells and other impurities
- Deep cleansing
- AQUAPURE System Cleaning



ELECTROPORATION

- Nourishes skin
- Boosts circulation
- Brightens skin



MICRO-CURRENT

- Tightens skin
- Lifts jowls and eyebrows
- Elastin and collagen regeneration
- Stimulates blood circulation



COOLING/HEATING

COOLING

- Reduces fine lines and wrinkles
- Enhances skin repair

HEATING

- Desensitizes skin
- Firms and tones

AQUAPURE Hand Pieces - AQUAPEEL



AQUAPEEL

- Operates with Small and Large Treatment Tip for peeling skin to thoroughly cleanse, and promote effective skin rejuvenation and hydration.
- Utilizes a Cleaning Cap to prevent system build up and to flush out previously used solutions.
- Operates with PEEL+, SEBO+, REJUVE+ and CLEANSYS.



AQUAPURE Hand Pieces - ELECTROPORATION



- Uses transdermal delivery to assist the penetration of the skin care solutions into the skin.
- A serum may be applied with this hand-piece to further enhance the treatment.

ELECTROPORATION



AQUAPURE Hand Pieces – MICRO-CURRENT



MICRO-CURRENT

- Operates with Y-shaped, double rollers that conduct electromagnetic stimulation on the patient's face to stimulate muscle and promote collagen regeneration.
- A serum or ultrasound gel may be used between the hand-piece and skin for the procedure.



AQUAPURE Hand Pieces – COOLING/HEATING



COOLING/HEATING

- A flat, metal-surfaced hand-piece that relieves hypersensitivity and promotes absorption of nutrients previously applied during the treatment.
- Cooling is at -5 degrees, or gentle warmth.
- A serum or ultrasound gel may be used between the hand-piece and skin for the procedure.



AQUASOLUTION Range

AQUAPURE



AQUASOLUTION Range



PEEL+

Facilitates the thorough process of exfoliation, removing impurities like dead skin cells and blackheads for a successful treatment

Active Ingredients:
Lactic Acid 9%



SEBO+

Helps the skin maintain a healthy balance between oily and moist tones for sebum dissolution

Active Ingredients:
Lactic Acid 9%
Salicylic Acid 1%



REJUVE+

Promotes elasticity by inducing collagen and elastin regeneration, improving water retention to prevent moisture loss

Active Ingredients:
Sodium Hyaluronate 7%
Oat Kernel Extract 1%



CLEANSYS

A primary cleaning agent formulated for an effective system maintenance procedure



AQUASOLUTION Range

STEP 1	STEP 2	STEP 3	STEP 4
<p>PEEL+ For peeling skin</p> <p>Removes dead skin cells and other impurities Exfoliates waste from the skin, keeping the skin clean and smooth.</p>	<p>SEBO+ For acne prevention</p> <p>Removes excess sebum and dead skin cells Removes excess oil, blackheads and other impurities maintaining a healthy balance of moisture to the skin.</p>	<p>REJUVE+ For smooth skin</p> <p>For smooth skin Infuses antioxidant-rich ingredients, nourishing and moisturizing the skin.</p>	<p>CLEANSYS For system cleaning purposes ONLY</p>

AQUASOLUTION Product Features

1. Healing properties that enhances the efficacy of treatment.
2. Contains patented ingredients
3. Immediate results
4. For all skin types and tones
5. Safe for skin
6. No harmful substances included (artificial colors, parabens, etc.)



PEEL+

Removes dead skin cells and other impurities and promotes formation of new collagen.

SEBO+

Removes excess sebum and dead skin cells from thickened skin, maintaining a healthy balance of moisture.

REJUVE+

Performs with infusing antioxidant-rich ingredients, packed with anti-aging and wrinkling and whitening properties.

CLEANSYS For System Cleaning Purposes

- For thorough and easy maintenance of the AQUAPURE device
- Prevents system buildup of ingredients during treatment
- Designed **ONLY** for the AQUAPEEL hand-piece




Solutions **MUST** be inserted into their correct positions respectively.



AQUAPURE

Treatment Guideline

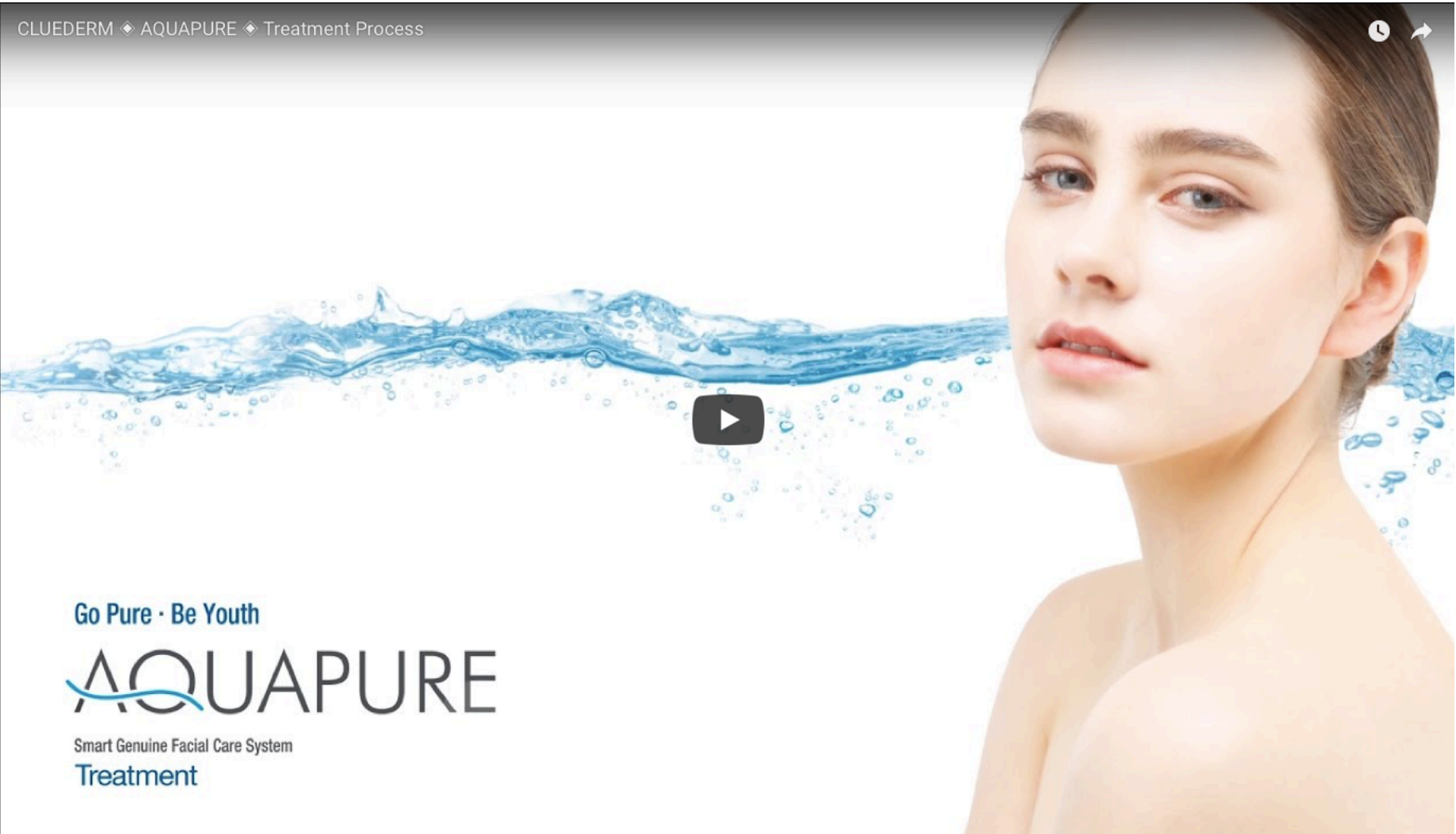

Go Pure · Be Youth

AQUAPURE

Smart Genuine Facial Care System
Treatment




Precautions for Procedure

- Do not use the equipment without prior training. To prevent unauthorized use of the AQUAPURE, ensure access is limited strictly to authorized and trained personnel only.
- Immediately stop the procedure if any adverse events or system malfunctions occur during treatment.
- Cover patient's eyes with damp cotton pads to avoid contact from PEEL+, SEBO+, REJUVE+, serum, and ultrasound gel during the AQUAPURE treatment.
- Check the patient's medical history for health assessments (allergic reactions, both internal and external health disorders, and injuries that might react negatively during the AQUAPURE treatment.)
- Make sure to adjust the levels based on the patient's skin sensitivity and procedure parameters.



AQUAPURE Treatment Steps

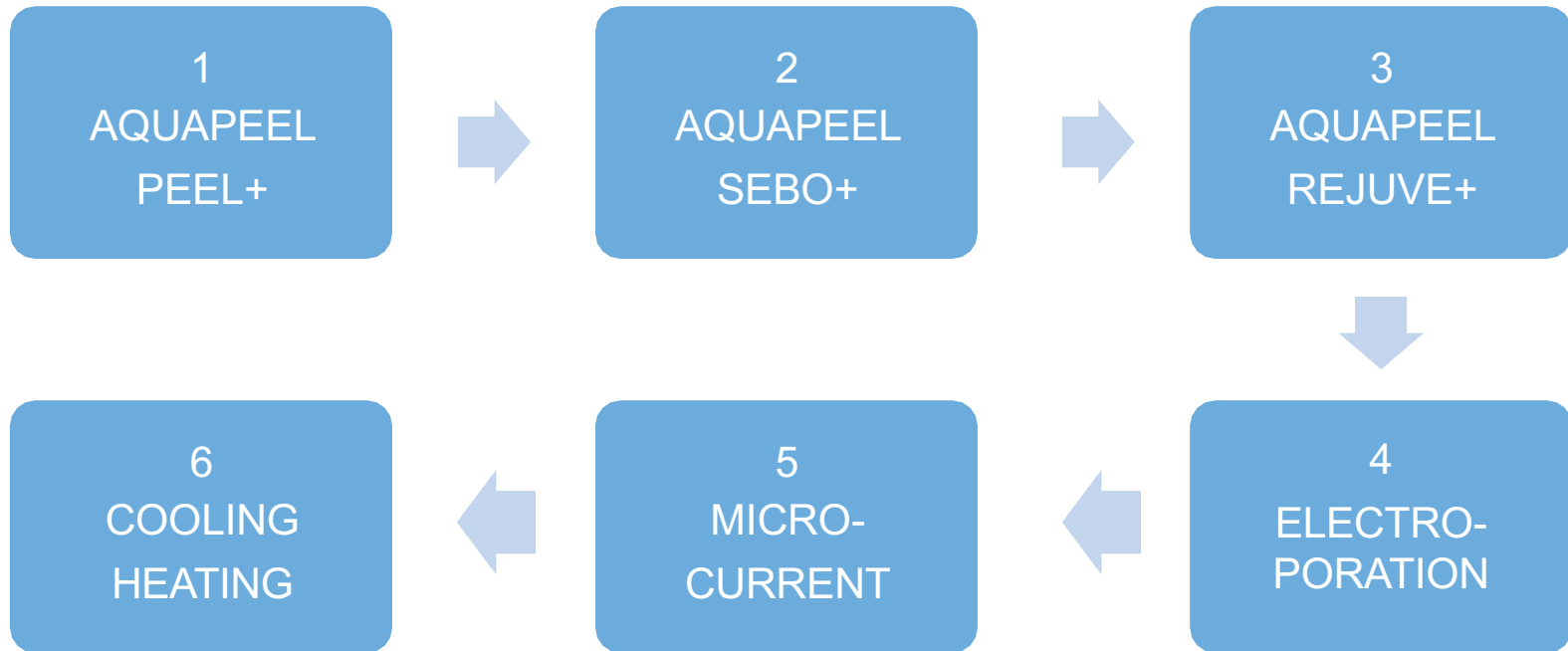

Preparation for Treatment



Latex Gloves



Ultrasound Gel



Ampoule/
serum



Tissues




Eye Protection

- There is no dress code for treatment. Comfortable attire is recommended.
- Patients must wear eye protection to avoid contact with PEEL+, SEBO+, REJUVE+, serum, and ultrasound gel during the AQUAPURE treatment. (Cotton pads are recommended)
- Prior to procedure, remove patients' makeup or other facial applications by a thorough face wash to ensure maximum effect of PEEL+, SEBO+, REJUVE+, ampoule during treatment.

Skin Type Programs

	PEEL+	SEBO+	REJUVE+	ELECTROPORATION	MICRO-CURRENT	COOLING	HEATING
Healthy Skin	5	5	5	5	5	X	5
	5	5	5	5	4	X	4
Anti-Aging	5	3	8	5	8	X	5
	4	4	5	3	4	X	3
Wrinkle Reduction	7	X	8	5	5	X	5
	5	X	5	3	4	X	3
Brightening	7	4	8	5	3	3	3
	5	5	5	4	4	3	3
Oily/ Congested	3	8	5	5	X	5	X
	5	5	5	4	X	4	X
Acne Skin	5	8	5	5	X	5	X
	4	4	5	4	X	4	X
Sensitive Skin	5	X	5	X	3	5	X
	3	X	3	X	2	3	X

PEEL+/SEBO+

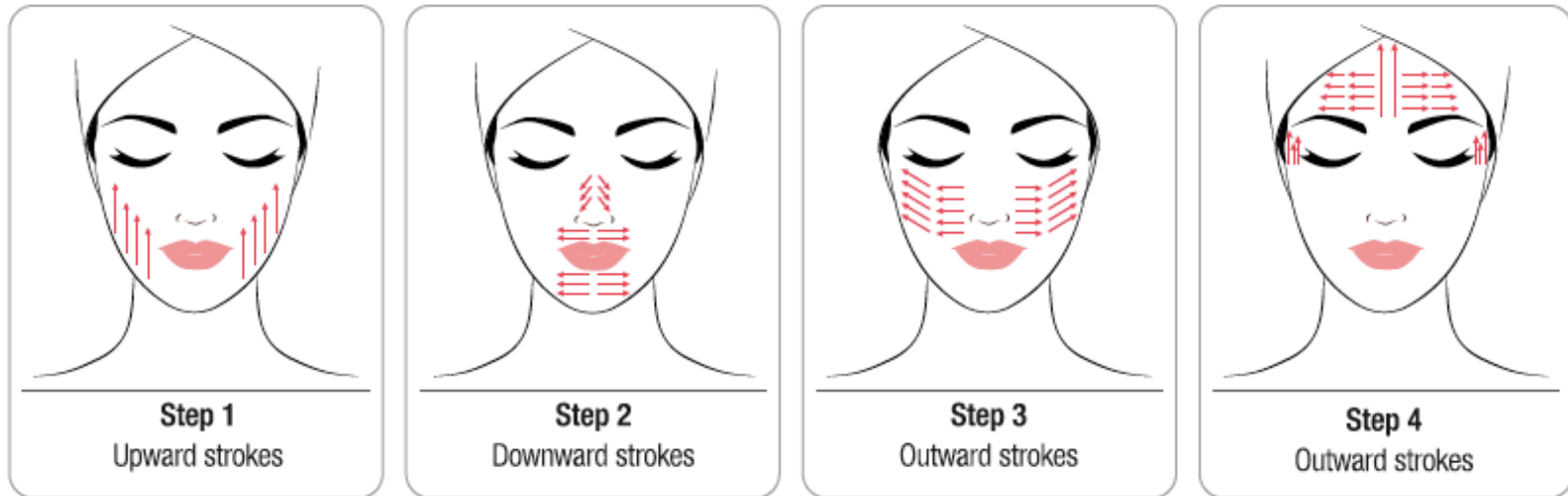

1. Perform a peeling function with PEEL+/SEBO+

- ① Press “1” (PEEL+) or “2” (SEBO+) on the AQUAPEEL Interface.
- ② Adjust vacuum levels (recommended at 5-7).
- ③ Press “START” to activate AQUAPEEL Hand-piece and conduct gentle strokes on the face.
- ④ Gently rub the applied solution after each stroke.
- ⑤ Repeat treatment steps in remaining time of procedure (5 min).


*Repeat procedure for Step 2 (SEBO+) on the AQUAPEEL interface.

AQUAPEEL: PEEL+ & SEBO+ Tips and Pearls

- Apply PEEL+ using large tip, complete one pass over the whole face, using long strokes and using other hand to anchor the skin.
- Tip needs to stay perpendicular to face to keep vacuum suction and flow of product consistent.
- SEBO+ product using small tip can first be dabbed onto congested areas (nose, chin, forehead) prior to first full pass to assist with extraction. Then complete one pass over full face as per previous step with the large tip.
- Do not use too much solution for PEEL+ and SEBO+ cycles so skin can uptake maximum in REJUVE+ cycle.




REJUVE+

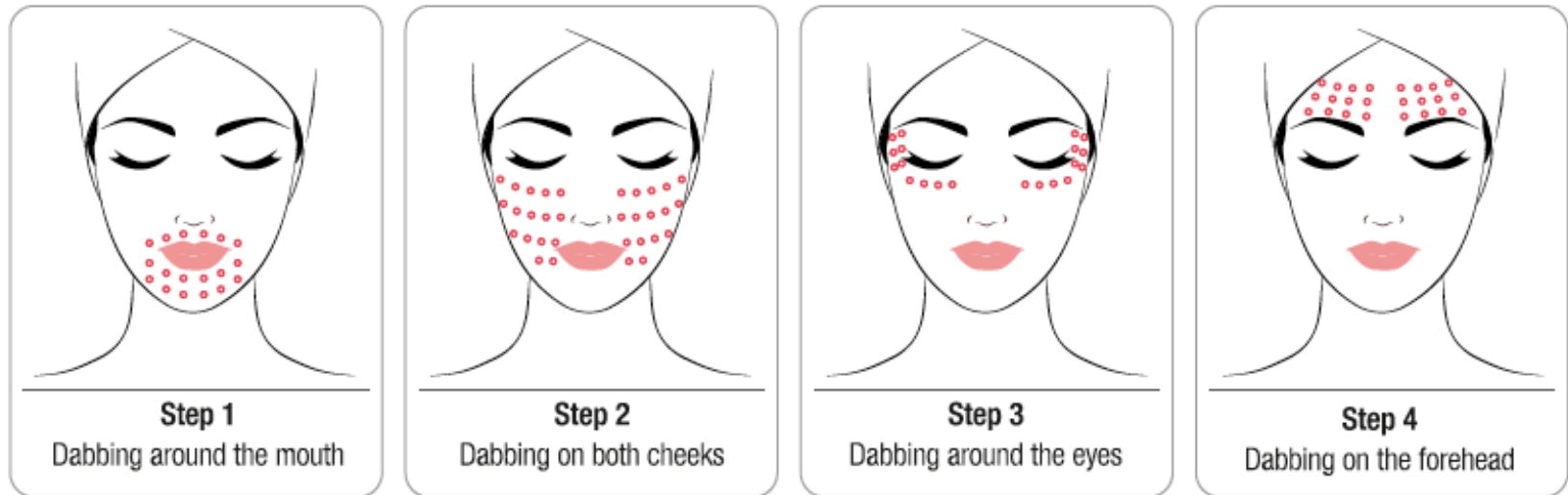

2. Hydrate with REJUVE+

- ① Remove residues of PEEL+ and SEBO+ from the patient's face with toner, baby wipe or micellar water.
- ② Re-insert treatment tip after cleaning and press "3" on the AQUAPEEL Interface.
- ③ Adjust vacuum levels (recommended at 5-7).
- ④ Press "START" to activate AQUAPEEL hand-piece and dab the facial area.
- ⑤ Gently rub the applied solution after each dabbing pattern.
- ⑥ Repeat dabbing treated area in remaining time of procedure (5 min).

AQUAPEEL: CLEANSYS

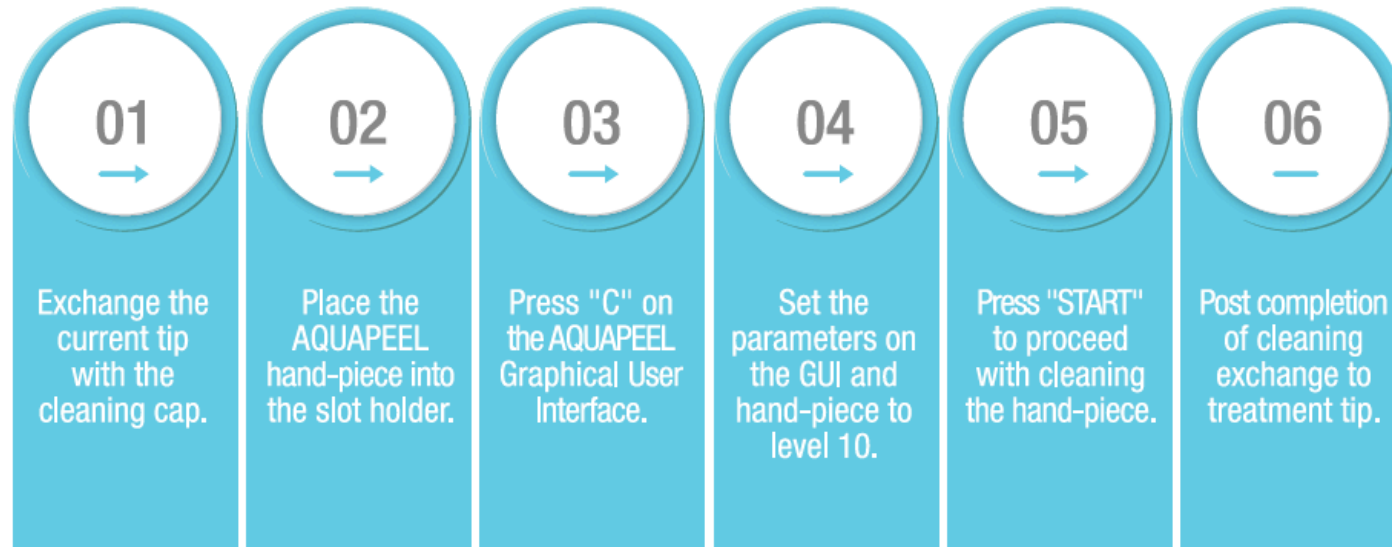


Cleaning the AQUAPEEL hand-piece with Cleaning Cap

- ① Press "C" and insert Cleaning Cap on the AQUAPEEL hand-piece.
- ② Press "START" to activate AQUAPEEL Hand-piece and the cleaning procedure with CLEANSYS should begin (2 min.).

Clean the AQUAPEEL Hand-piece after completion of SEBO+ and REJUVE+
Refer to the AQUAPURE Maintenance Guide for more information.




ELECTROPORATION

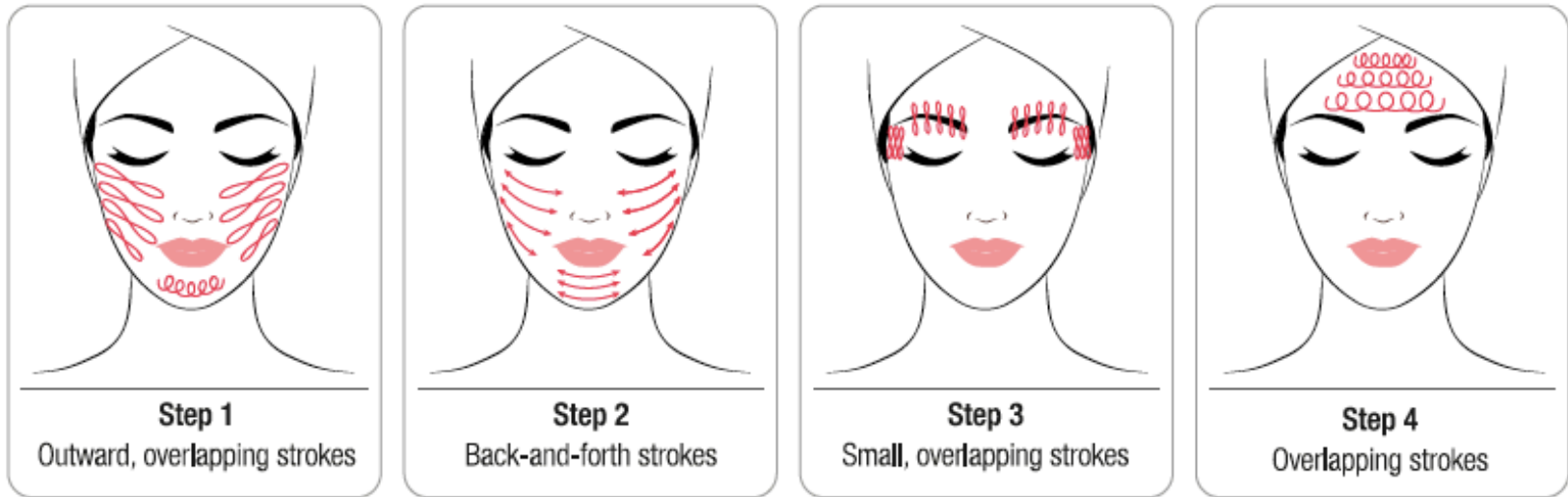

3. Treat the facial area with ELECTROPORATION

- ① Use any ampoule or serum as an agent between the hand-piece and skin during procedure.
- ② Adjust levels (recommended at 3-4). Ensure full contact with the skin.
- ③ Press "START" to activate ELECTROPORATION Hand-piece and gently stroke the facial area.
- ④ Repeat treatment steps in remaining time of procedure (5 min.).


MICRO-CURRENT

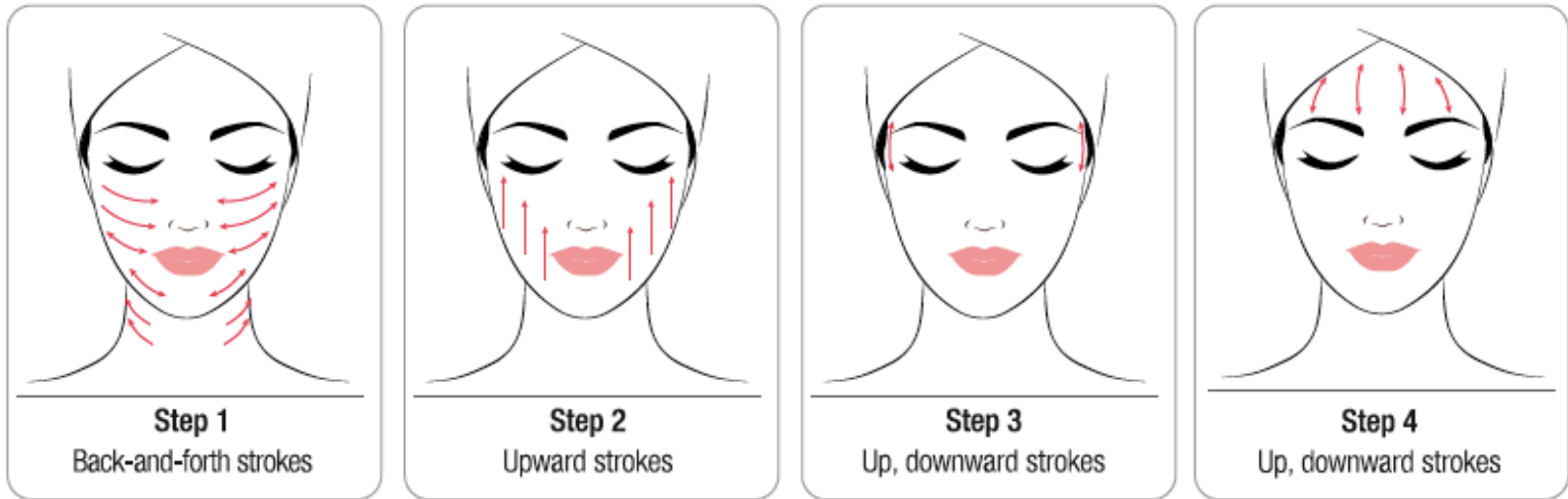

4. Treat the facial area with MICRO-CURRENT

- ① Use serum, ampoule or ultrasound gel as an agent between the hand-piece and skin during procedure.
- ② Adjust levels (recommended at 3-4). Ensure full contact with the skin.
- ③ Press "START" to activate MICRO-CURRENT Hand-piece and gently stroke the facial area.
- ④ Repeat treatment steps in remaining time of procedure (5 min).

COOLING/HEATING

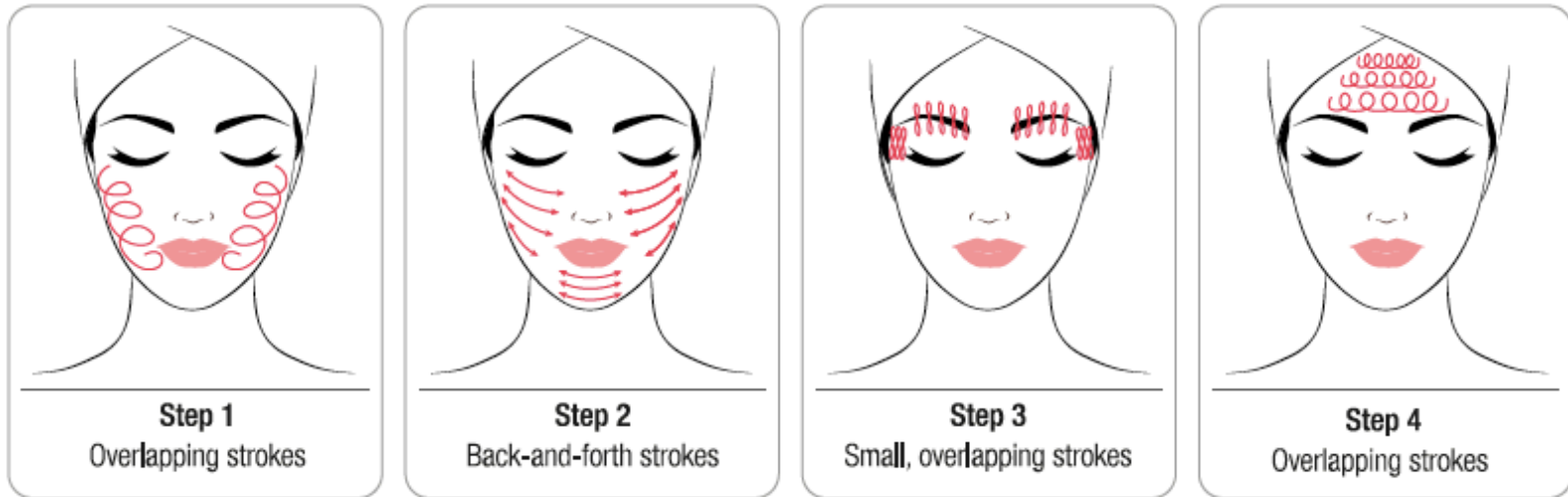

4. Completing the treatment with COOLING/HEATING

- ① Use ampoule or ultrasound gel as an agent between the hand-piece and skin during procedure.
- ② Press “COOLING” or “HEATING” on the COOLING/HEATING Interface.
- ③ Adjust “COOLING” or “HEATING” levels.
- ④ Press “START” to activate COOLING/HEATING Hand-piece and gently stroke the facial area.
- ⑤ Repeat treatment steps in remaining time of procedure (5 min. per each function).

*Practitioners performing the COOLING/HEATING procedure can start the procedure with either COOLING or HEATING.

Post Treatment Guidelines

- Apply sun cream to protect skin from ultraviolet (UV) ray exposure
- Avoid direct sunlight by limiting your time outdoors
- Apply soothing products to comfort the skin and reduce redness
- Do not over-moisturize or pick your skin
- Treat your skin gently to avoid aggravating the skin
- No downtime after treatment
- Patients may apply makeup immediately following treatment

Maintenance Tips

For all maintenance and troubleshooting information, refer to the the AQUAPURE Maintenance Guide.

- Clean and sterilize the hand pieces with alcohol after each treatment.
- Turn system off before changing or refilling with water. Use distilled water only.
- Apply lubricant or grease to the red O rings on the Waste Container and on hand pieces every 4 weeks, or more regularly to ensure proper connection.

Maintenance Guide
Version 2.0

AQUAPURE

Smart Genuine Facial Care System



Go Pure
Be Youth

Cryomed
Aesthetics

Cluederm™
Entire Beauty Solution



Precautions for System Safety

- Keep the surfaces of all hand-pieces clean after use.
- Do not use chemical agents such as benzene, acetone, or lacquer thinner for cleaning the AQUAPURE Hand-pieces.
- Observe for any scratches on the ELECTROPORATION Hand-piece when wiping and cleaning the applicator.
- If any of the hand-pieces have been dropped, replace individual applicators and inspect if they work properly so as to avoid any malfunction during treatment.

*For additional information, refer to the AQUAPURE Maintenance Guide.



AQUAPURE

Clinical photos

Clinical Cases



Baseline

Immediately Post Procedure

Gender : FEMALE

Age : 35

Skin Program : Anti-aging

Clinical Cases



Baseline

Immediately Post Procedure

Gender : MALE

Age : 30

Skin Program : Acne Skin

Clinical Cases



Baseline

Immediately Post Procedure

Gender : FEMALE

Age : 39

Skin Program : Sensitive Skin