

○ SAT(System Access Token) Card

A component of the CLATUU Alpha that is specifically designed for proper and safe use purposes in combination with the CLATUU Shield Pad during treatment.


○ SAT Card Product Info

A type	Quantity
SAT Card	1
CLATUU Shield 360 A	50
C type	Quantity
SAT Card	1
CLATUU Shield 360 C	50

○ Security Card Placement and GUI Display



Remaining Count Display

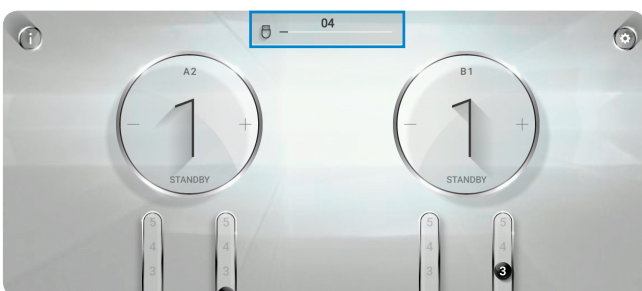

○ SAT Card

Application

Remaining Count: The number of remaining treatments of the currently applied SAT Card (total of 50 treatments available).

Precautions

- Remaining Count will be deducted by 1 for EACH Hand-piece.
- Remaining Count will be deducted by 2 for BOTH Hand-pieces.
- Count deductions are applied two minutes after the start of the procedure.
- In the case a Shield Pad is not attached properly, re-application of the Shield Pad should be performed using a new Shield Pad within two minutes without having a count deducted from the SAT Card.
- If the Remaining Count reaches 0, a new SAT Card with sufficient Remaining Counts must be inserted to perform treatment.
- Ensure that the Remaining Count is sufficient before proceeding to treatment.
- If any error of application is discovered after the two minute period, the procedure should be terminated for safety.

○ SAT Card Chart

If Remaining Count value is 1	Only one hand-piece will be functionable
If the SAT Card is disconnected during the procedure	Active hand-pieces will be stopped immediately
If the SAT Card has a green light	Able to proceed with treatment
If the SAT Card has a flashing green light	No remaining count or in progress of loading
If the SAT Card has a red light	Error recognized

Warning!

Inserting the SAT Card to any other device or inserting any other USB cable into the SAT Card USB port may cause malfunctions to occur.