



# CLATUU Shield 360 Gel Pad A TYPE

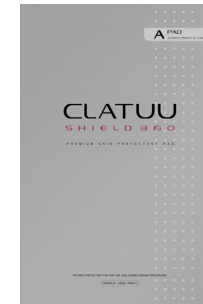

A Type

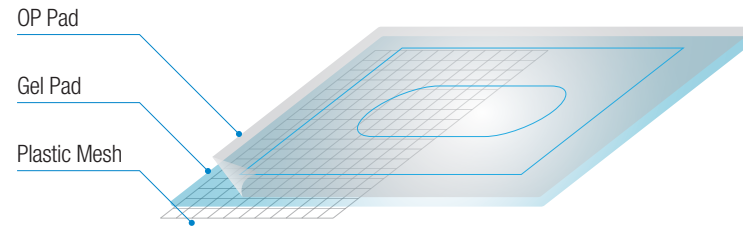

1 Tear the CLATUU Shield Pad A-Type open from the top of the packaging.




2 Remove the CLATUU Shield Pad A-Type from the packaging with care.



3 Open up the CLATUU Shield Pad A-Type and ensure all of the layers are firmly intact.



4 Apply the CLATUU Shield Pad A-Type at the center of the marked treatment area.

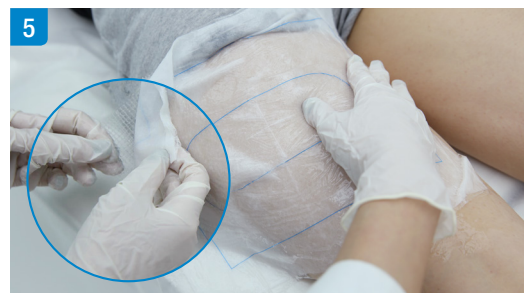

5 Remove the bottom layer of plastic mesh from the rest of the CLATUU Shield Pad A-Type.



6 Smooth out any air bubbles and wrinkles before applying the Hand-piece on the outlined center of the CLATUU Shield Pad A-Type.