



CLATUU Shield 360 Gel Pad C TYPE

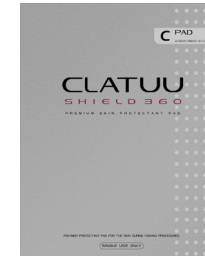


1 Tear the CLATUU Shield Pad C-Type open from the top of the packaging.

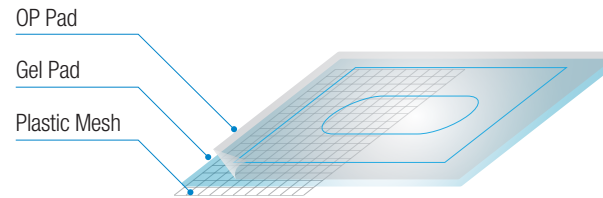


C Type




2 Remove the CLATUU Shield Pad C-Type from the packaging with care.

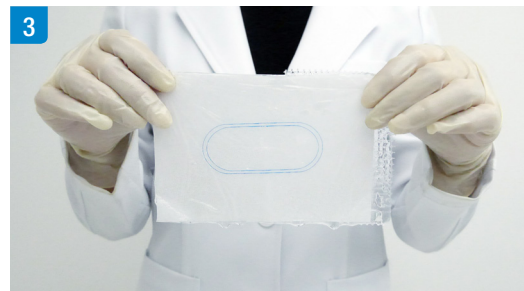

3 Open up the CLATUU Shield Pad C-Type and ensure all of the layers are firmly intact.

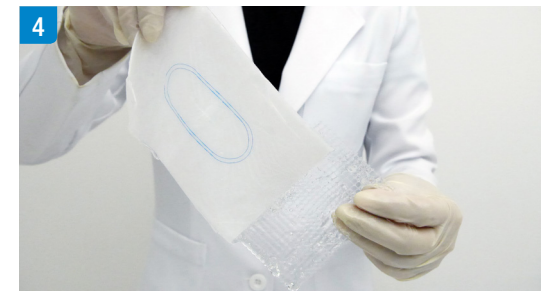

4 Remove the bottom layer of plastic mesh from the rest of the CLATUU Shield Pad C-Type.

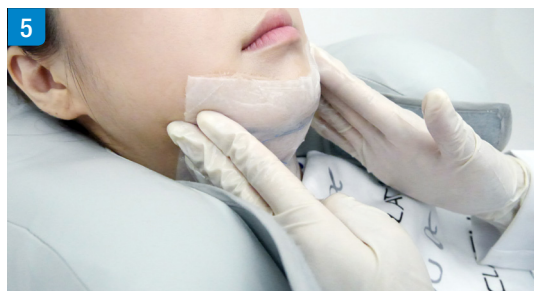

5 Apply the outlined center of the CLATUU Shield Pad C-Type on the marked treatment area. Smooth out any air bubbles and wrinkles before applying the Hand-piece.



6 Apply the Hand-piece on the marked treatment area.



7 Start the procedure.