



# CLATUU $\alpha$ TREATMENT CHECKLIST

1	Patient's pre-treatment weight and measurements taken.	<input type="checkbox"/>
2	Patient discussion about pre and post discomfort.	<input type="checkbox"/>
3	Take baseline photograph(s).	<input type="checkbox"/>
4	All materials are ready (pillows, towels, gel pads, stencils, marker, etc).	<input type="checkbox"/>
5	Checked machine ready for treatment (no error messages on screen, no strange sounds, etc).	<input type="checkbox"/>
6	Treatment areas marked up correctly with patient standing up.	<input type="checkbox"/>
7	Machine setting based on treatment area protocols and choose the correct handpiece (s).	<input type="checkbox"/>
8	Check the handpieces are connected securely.	<input type="checkbox"/>
9	Check the gel pad expiry date to verify it is useable.	<input type="checkbox"/>
10	Physically show unopened pad to patient and say, "I am now opening a new gel pad for your treatment."	<input type="checkbox"/>
11	<b>Important: Lay unopened gel pad on flat surface, check for any damage on the packaging, and spread gel evenly through the pad as gel may pool during storage.</b>	<input type="checkbox"/>
12	Ensure treatment areas are clean and dry.	<input type="checkbox"/>
13	Open gel pad and grab gel inside to place onto the treatment are.	<input type="checkbox"/>
14	Take out the Shield360 pad carefully and remove the plastic liner before applying to the skin.	<input type="checkbox"/>
15	Placement of gel pad in line with the marked area and <b>gel distribution is double checked</b> to promote treatment safety.	<input type="checkbox"/>
16	Ensure any air bubbles are carefully smoothed out.	<input type="checkbox"/>
17	Ensure <b>no unauthorised gel or other material</b> present in the treatment area before placement of the handpiece (s) correctly on the centre of the gel pad.	<input type="checkbox"/>
18	After suction is applied, perform a 360 check and say, "I am now performing a 360 check around the handpiece."	<input type="checkbox"/>
19	Check how patient is going, patient has a call bell and can reach out to clinic	<input type="checkbox"/>
20	You can perform a 360 check again after 10-15 minutes to ensure that the gel pad has not moved.	<input type="checkbox"/>