


COSJET DYE HANDPIECE INSTRUCTIONS

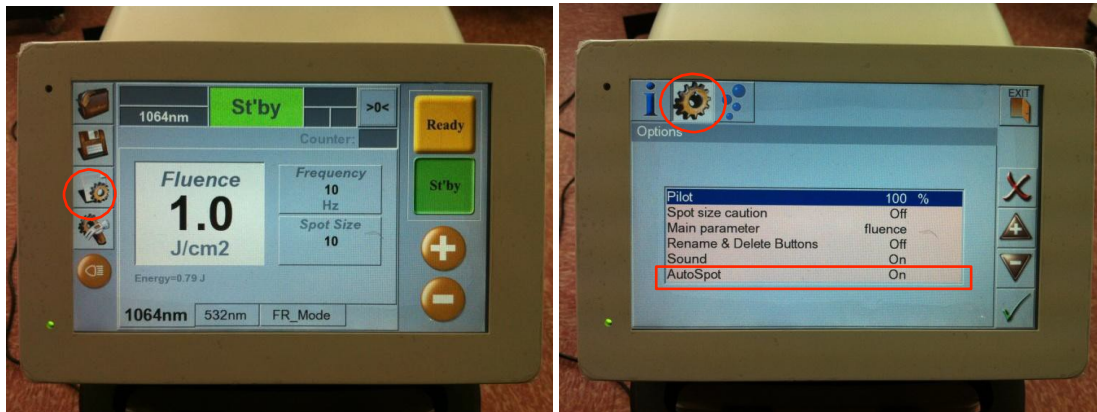

TO CONNECT DYE HANDPIECE

1. Select the COG icon on the main screen
2. Select the COG icon on the second screen
3. Highlight AutoSpot by touching it
4. Touch the Tic ✓
5. Touch the + symbol to change AutoSpot to OFF
6. Touch the Tic ✓
7. Touch EXIT
8. Turn laser OFF
9. Take off the Normal handpiece
10. Attach the Dye handpiece
11. Turn laser ON
12. Set wavelength to 532nm, Fluence to maximum J/cm², and Hz to lowest

TO REVERT TO NORMAL HANDPIECE

1. Select the COG icon on the main screen
2. Select the COG icon on the second screen
3. Highlight AutoSpot by touching it
4. Touch the Tic ✓
5. Touch the + symbol to change AutoSpot to ON
6. Touch the Tic ✓
7. Touch EXIT
8. Turn laser OFF
9. Take off the Dye handpiece
10. Attach the Normal handpiece
11. Turn laser ON