

MF2

Treatment Guide

ULTRAFORMER[®] III


MF2 Treatment Guide - Eye Bags



Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Eye Bags	0.1-0.2J	Left: 25x2passes Right: 25x2passes	100



MF2 Treatment Guide - Forehead

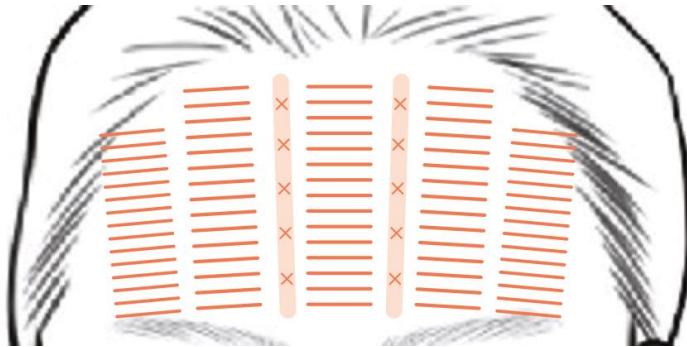

Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Forehead	0.1-0.2J	75x2passes	150



Baseline




Immediately Post Procedure

MF2 Treatment Guide - Frown Line



Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Frown Line	0.1-0.2J	25x2passes	50

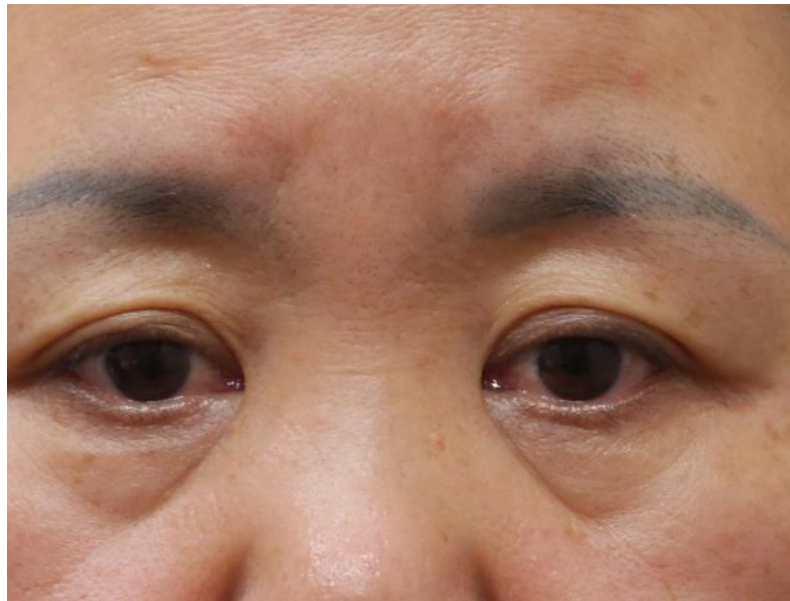


Baseline



Post 2 Sessions

MF2 Treatment Guide - Nasolabial Folds

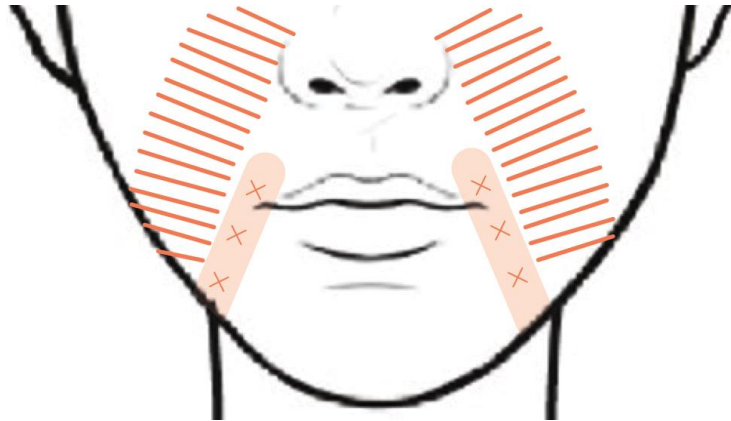

Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Nasolabial Folds	0.1-0.2J	Left: 15x3passes Right: 15x3passes	90



Baseline

Immediately Post Procedure

MF2 Treatment Guide - Around Eyes



Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Around Eyes	0.1-0.2J	Left: 10x3passes Right: 10x3passes	60




Baseline



Post 3 Sessions


MF2 Treatment Guide - Cheeks

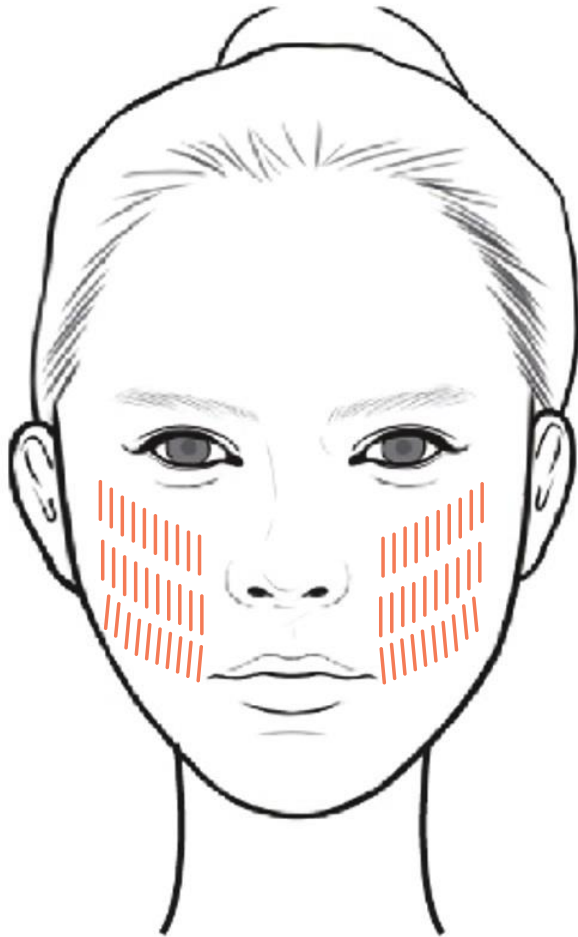

Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Cheeks	0.1-0.2J	Left: 30x3passes Right: 30x3passes	180



MF2 Treatment Guide - Neck fold

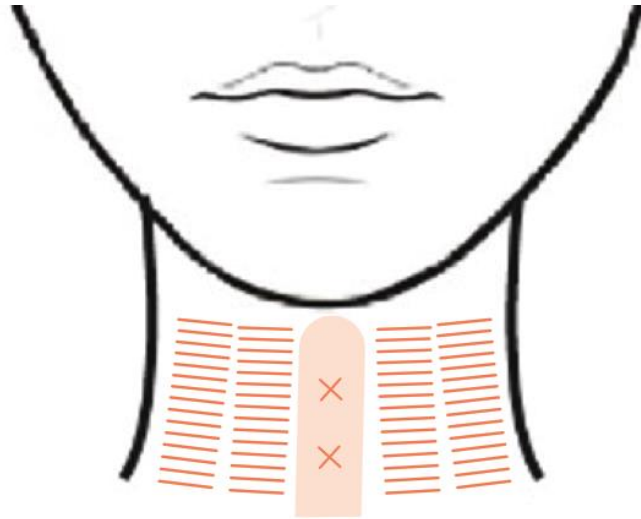


Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Neck fold	0.1-0.2J	Left: 30x2passes Right: 30x2passes	120



Baseline

Immediately
Post Procedure

MF2 Treatment Guide - Lip Wrinkle

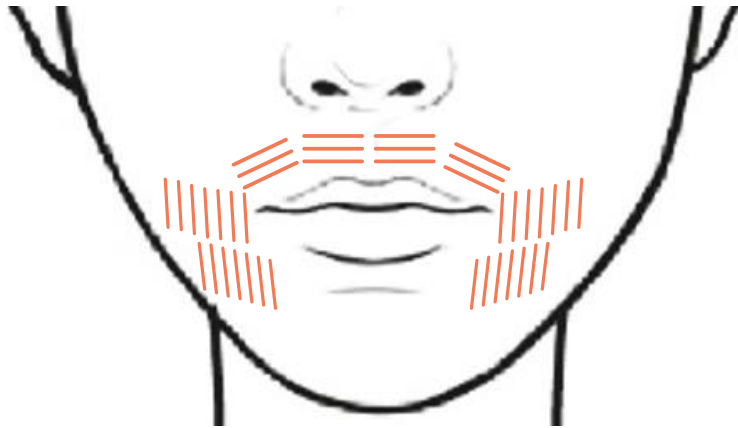


Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Lip Wrinkle	0.1-0.2J	Left: 20x2passes Right: 20x2passes	80



Baseline

Immediately Post Procedure


MF2 Treatment Guide - Eyebrow Lifting

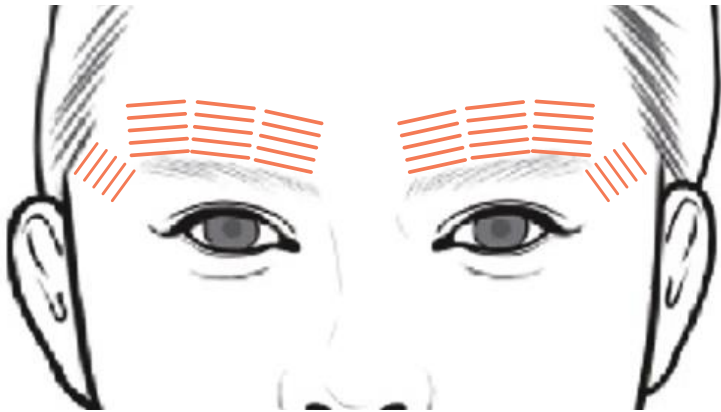

Cartridge	Treatment area	Energy	Shots / Passes	Total Shots
MF2	Eyebrow Lifting	0.1-0.2J	Left: 20x2passes Right: 20x2passes	80

MF 2 Toning Program



MF2 Treatment Guide – MF2 Toning program

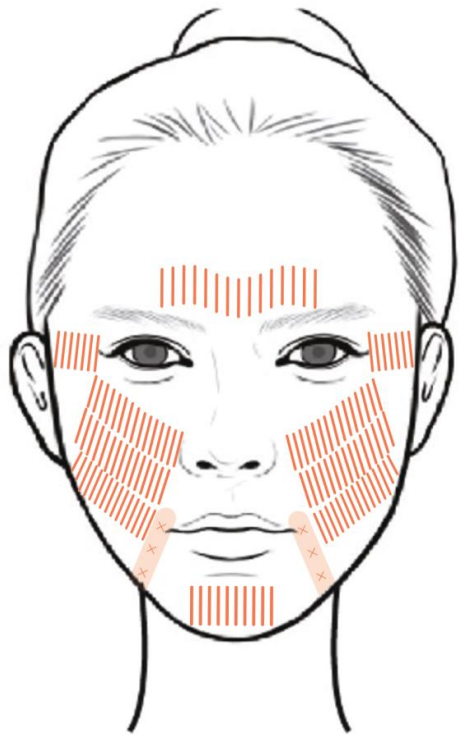

2 Weeks



2 Weeks

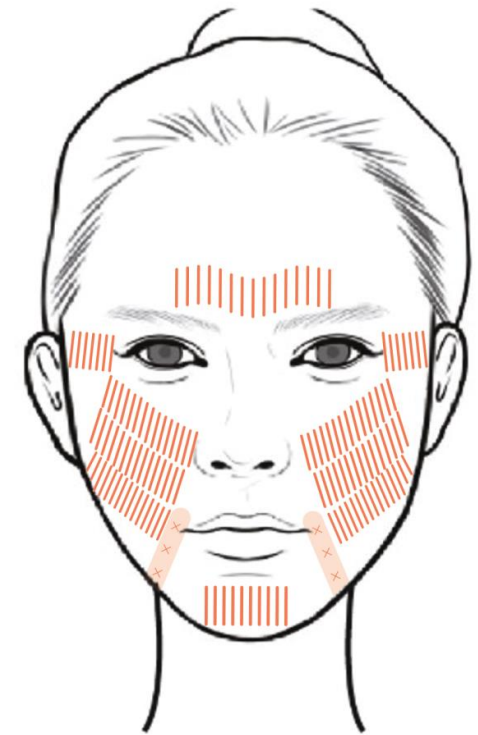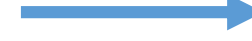


MF2 Toning
0.1-0.2J 100 shots
(2passes)

Vitamin C treatment


MF2 Toning
0.1-0.2J 100 shots
(2passes)

MF2 Treatment Guide – MF2 Toning 3 sessions program

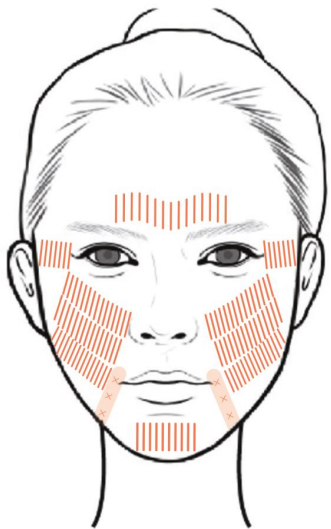


MF2 Toning
0.1-0.2J 100
shots

2 Weeks

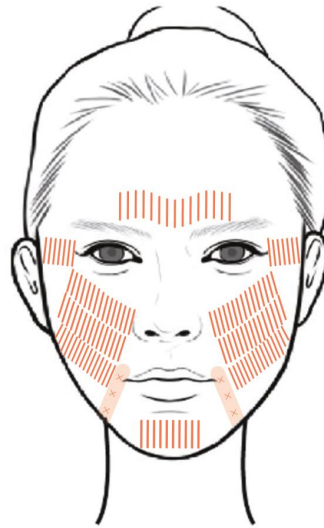


MF2 Toning
0.1-0.2J 100
shots

2 Weeks

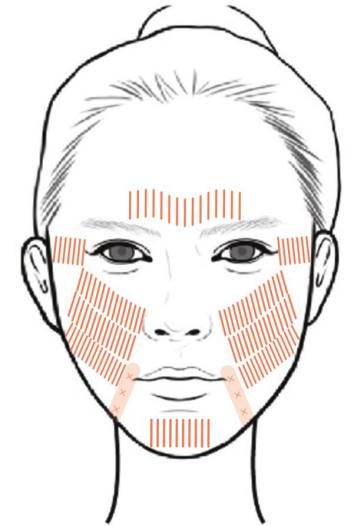


MF2 Toning
0.1-0.2J 100
shots



MF2 Treatment Guide – MF2 Toning Intensive3 sessions program

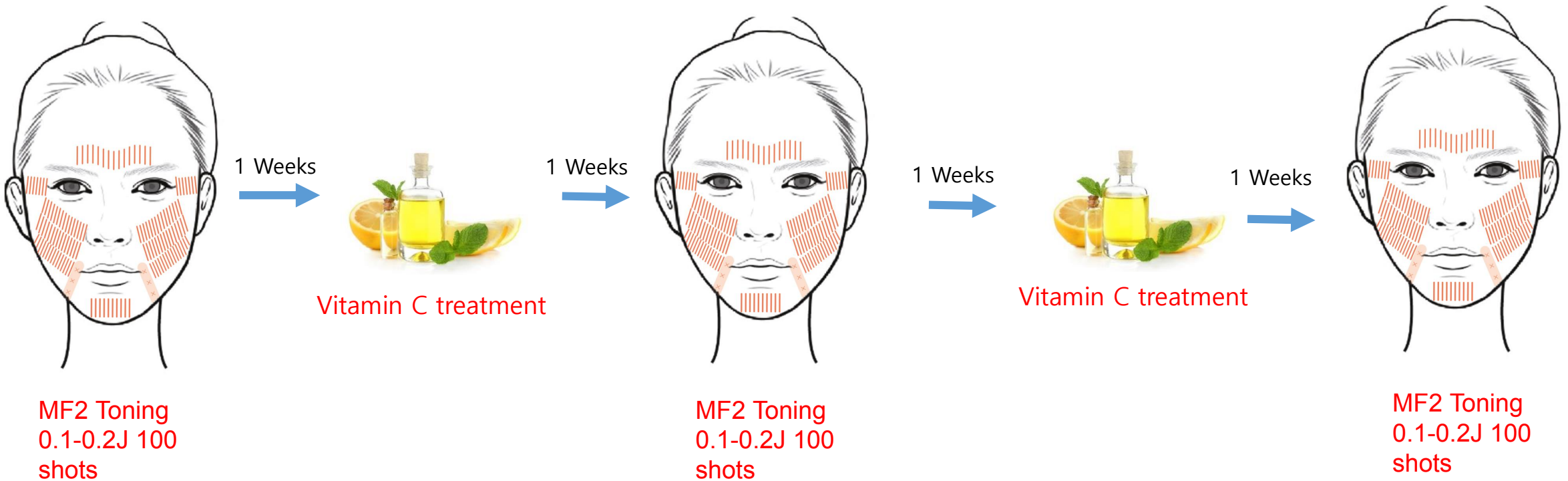

MF2 Treatment Guide – Simple Lifting & Toning

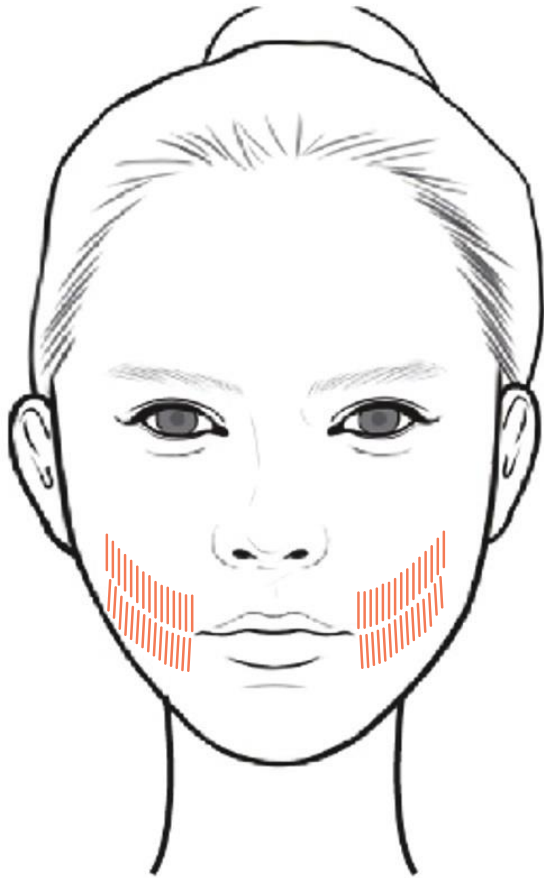


4.5mm Cartridge / 0.7J >> 60x1passes

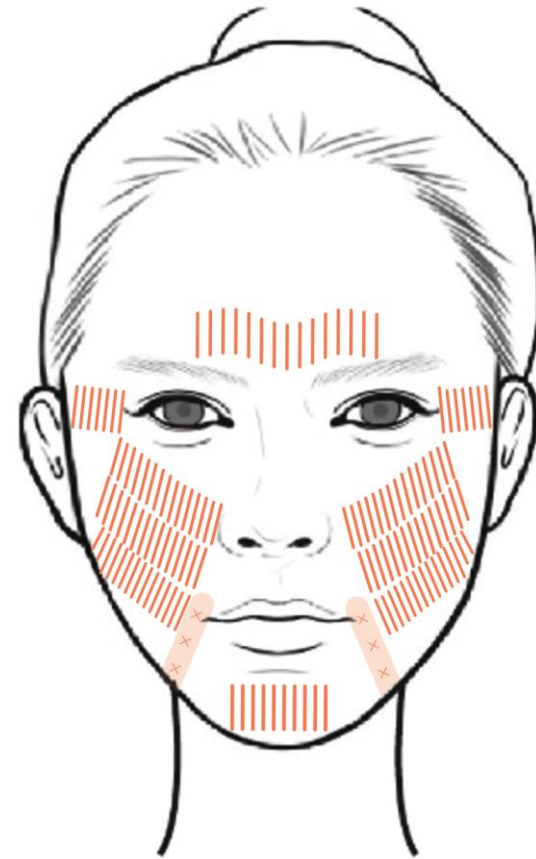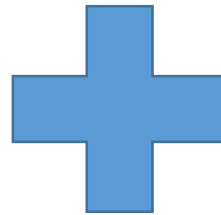


MF2 Cartridge / 0.1-0.2J >> 100x1passes


MF2 Treatment Guide - 3 layers Lifting

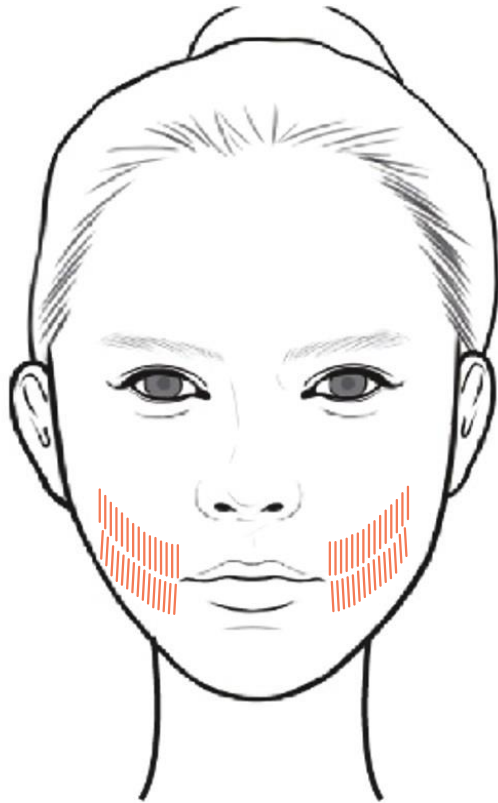

4.5mm Cartridge / 0.7J >>60x1passes

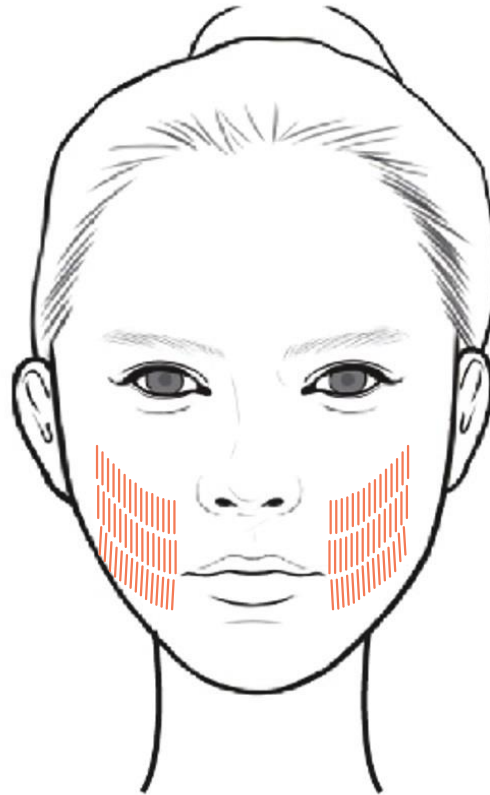

3.0mm Cartridge / 0.4J >>90x1passes

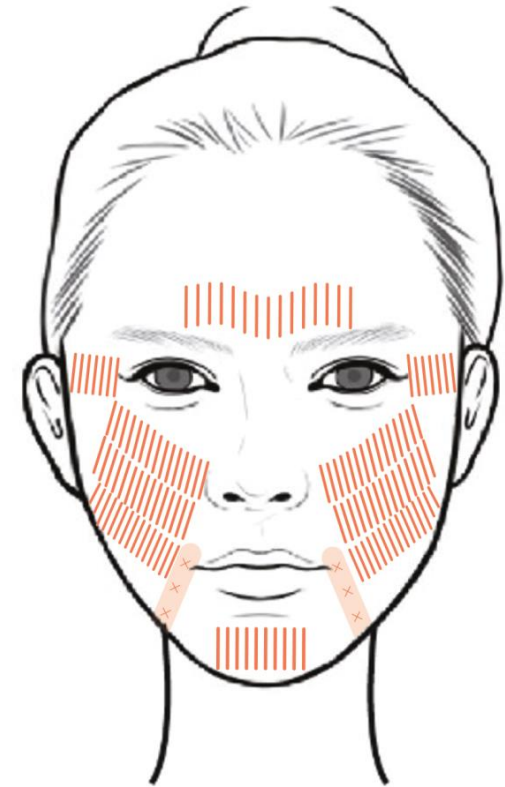

MF2 Cartridge/ 0.1-0.2J >>100x1passes