

AQUAPURE II

Smart Genuine Facial Care System



Go Pure
Be Youth

Table of Contents

Components and Features

Assembly

1. Connecting Hand-pieces
 - AQUA Hand-pieces
 - Diamond Hand-pieces
 - LIGHTPURE Hand-pieces
2. Attaching Treatment Tips and Cleaning Cap
3. Connecting the Waste Container
4. Connecting AQUASOLUTION Bottles
5. Connecting the Footswitch
6. Turning the System ON

Disassembly

1. Turning the System OFF
2. Detaching Treatment Tips and Cleaning Cap
3. Disconnecting the Footswitch
4. Disconnecting Hand-pieces
5. Disconnecting the Waste Container
6. Disconnecting AQUASOLUTION Bottles

Components and Features



Main Body



PEEL+



SEBO+



REJUVE+



CLEANSYS



AD Peel Tip



AQUA MF Needle Tip



Cleaning Cap



Air Filter



Waste Container



AC Power Cable



Footswitch



Eye Shield



AQUA Hand-Piece



Diamond Hand-Piece (option)



LIGHRPURE Hand-Piece

AQUAPURE II

ASSEMBLY

The following content details instructions to assemble the AQUAPURE II Main Body.
Refer to these instructions in respective order for a successful assembly of the AQUAPURE II and its components.



Assembly

1. Connecting Hand-pieces

Before connecting Hand-piece Cables to the Main Body, ensure that the System is turned OFF for safety purposes.

Hand-piece Cables (3x) connect on both left and right sides of the Main Body.

Diamond Hand-Piece Cable connects to outlets on the left side of the Main Body with built-in Hand-piece Holder.

AQUA Hand-piece Cable connects to outlets on the right side of the Main Body with built-in Hand-piece Holder.

LIGHTPURE Hand-piece Cable connects to outlets on the left side of the Main Body with built-in Hand-piece Holder.

*Refer to the icons above each Hand-piece Holder outlets before connecting to the Main Body.

Left Side of Main Body



Right Side of Main Body



Assembly

AQUA Hand-piece

Slide the Hand-piece into its correct Hand-piece Holder where the AQUA function icon is labeled.

Connect the AQUA Hand-piece Cable in upside position to the Main Body (with the AQUA function icon labeled just above the outlet).

Align the Hand-piece Cable with its respective outlet to ensure a successful connection.



Assembly

Diamond Hand-piece

Slide the Hand-piece into its correct Hand-piece Holder where the Diamond function icon is labeled.

Connect the Diamond Hand-piece Cable in upside position to the Main Body (with the Diamond function icon labeled just above the outlet).

Align the Hand-piece Cable with its respective outlet to ensure a successful connection.



Assembly

LIGHTPURE Hand-piece

Align the LIGHTPURE hand-piece cable with its respective outlet (with LIGHTPURE function icon labeled) to ensure a successful connection

Place LIGHTPURE Hand-piece at the bottom of the main body




Assembly

2. Attaching Treatment Tips and Cleaning Cap

The AQUAPURE Hand-piece treats the patient's skin with Diamond Tip (Dry Abrasion), AD (Aqua + Diamond) Peel Treatment Tips, and AQUA MF (Micro-Focused) Needle Treatment Tip. The Hand-piece also conducts an AQUA cleaning procedure with a Cleaning Cap.

For successful attachment, push down Treatment Tips (AD Peel Tip and AQUA MF Needle Tip) and Cleaning Cap firmly and turn diamond tip clockwise.

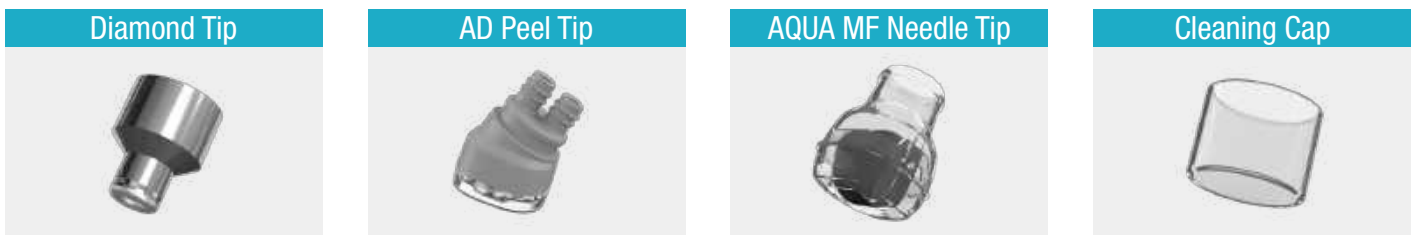

Assembly

2. Attaching Treatment Tips and Cleaning Cap

AQUA MF Needle Tip

1



2



Cleansing Cap

1



2



Assembly

3. Connecting the Waste Container

The Waste Container receives wastewater following each AQUAPURE II procedure.

For successful connection to the rear of the Main Body, adjust the Waste Container in center position with the open slot. After securing center position with the Main Body, hold the Waste Container up and turn right to fully lock into the System.

⚠ If the Waste Container is not securely locked, the suction capabilities of the AQUAPEEL Hand-piece will not work effectively.



Assembly

4. Connecting AQUASOLUTION Bottles

Receivers for attaching AQUASOLUTION bottles (4x) to the Main Body are located just below the AQUAPURE II Graphical User Interface (GUI) with a tinted, plastic cover.

Uncap each AQUASOLUTION bottle one at a time and insert the tube into each bottle. Align the bottle with the receiver of the Main Body and turn right until it is firmly inserted.

1. PEEL+
2. SEBO+
3. REJUVE+
4. CLEANSYS



Assembly

5. Connecting the Footswitch

The Footswitch is connected for both manual START and STANDBY for all AQUAPURE II procedures. The Footswitch outlet of the Main Body is located beside the AQUA cable outlet.

Connect the Footswitch Cable in upside position (with the Footswitch function icon labeled just above the outlet). Align the Hand-piece Cable with its respective outlet to ensure a successful connection.



Assembly

6. Turning the System ON

Before turning the System ON, ensure that there are no water spills or hazards present in the operating environment.

Connect the AC Power Cable to the outlet at the lower right rear of the Main Body.

Check that the AC Power Cable is firmly connected to the rear of the Main Body and then proceed to connect to a grounded power source.

To activate the System, press the 'I' symbol on the AC Power Switch.





ASSEMBLY COMPLETE

For further guidelines and enquiries on the AQUAPURE II System, refer to additional training materials for the proper and professional use of the AQUAPURE II.

- Maintenance Guide
- Treatment Guide
- Quick Operation Guide
- Operation Manual

For further enquiries contact:
info@cluederm.com

AQUAPURE II

DISASSEMBLY

The following content details instructions to disassemble the AQUAPURE II Main Body. Refer to these instructions in respective order for a successful disassembly of the AQUAPURE II and its components.



Disassembly

1. Turning the System OFF

Before turning the System OFF, ensure that there are no water spills or hazards present in the operating environment.

Before turning the System OFF, ensure that there are no water spills or hazards present in the operating environment.

To deactivate the System, press the 'O' symbol on the AC Power Switch.

Disconnect the AC Power Cable from the outlet at the lower right rear of the Main Body. Proceed to disconnect the other end of the AC Power Cable from the grounded power source.



Disassembly

2. Detaching Treatment Tips and Cleaning Cap

For successful detachment, pull Treatment Tips (Diamond Tip, AD Peel tip, AQUA MF Needle, and Cleaning Cap) from the AQUA Hand-piece. And turn Diamond Treatment Tip anticlockwise.

Store the Treatment Tips and Cleaning Cap in a safe and dust-free environment.

Diamond Tip



AD Peel Tip



AQUA MF Needle Tip



Cleaning Cap



Diamond Tip

1



2



AD Peel Tip

1



2



Disassembly

2. Detaching Treatment Tips and Cleaning Cap

AQUA MF Needle Tip

1



2



Cleansing Cap

1



2



Disassembly

3. Disconnecting the Footswitch

The Footswitch Cable is connected alongside the Hand-piece outlets.

Disconnect the Footswitch Cable from the Main Body.

Ensure to keep all cables dry and untangled in a dust-free environment for storage.



Disassembly

4. Disconnecting Hand-piece

There are 2 Hand-pieces which are attached to the Main Body.


Slide the Hand-pieces out of their respective Hand-piece Holders and proceed to disconnect the cables from the Main Body one at a time.

When disconnecting AQUA Hand-piece cable, press both sides of cable firmly while pulling down the cable as shown in figure 2 of AQUA Hand-piece.

When disconnecting Diamond Hand-piece, press the connecting part with one hand while pulling down the cable as shown in figure 2 of Diamond Hand-piece

Ensure to keep all cables dry and untangled in a dust-free environment for storage.

AQUA Hand-piece

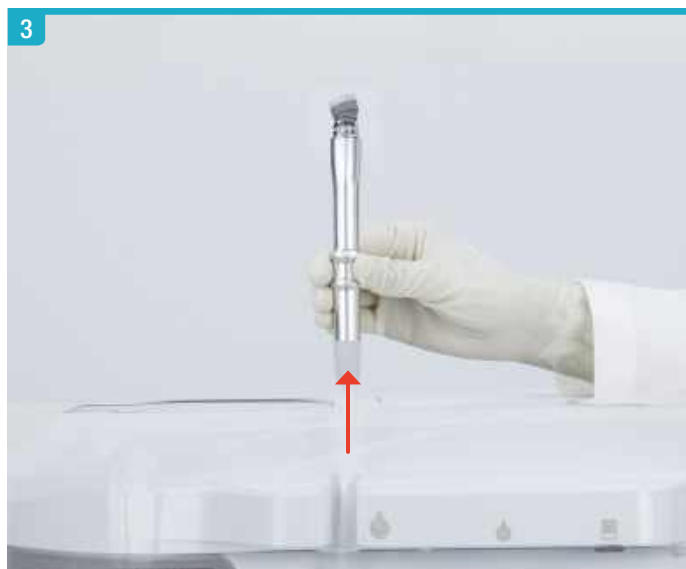

Disassembly

4. Disconnecting Hand-piece

Diamond Hand-piece



Disassembly

5. Disconnecting the Waste Container

For successful disconnection of the Waste Container from the rear of the Main Body, turn the container handle to the left until it is at center position with the Main Body.

Gently lower the Waste Container with caution to prevent wastewater from spilling.




Disassembly

6. Disconnecting AQUASOLUTION Bottles

Open the tinted, plastic cover of the Main Body. Turn each bottle to the left for proper disconnection. Slowly pull down the bottle as the tube can spatter if not removed with caution.

Wipe down each tube with a clean cloth after the bottle is safely removed.

Cover each AQUASOLUTION bottle with their respective bottle caps after safe disconnection from the Main Body.

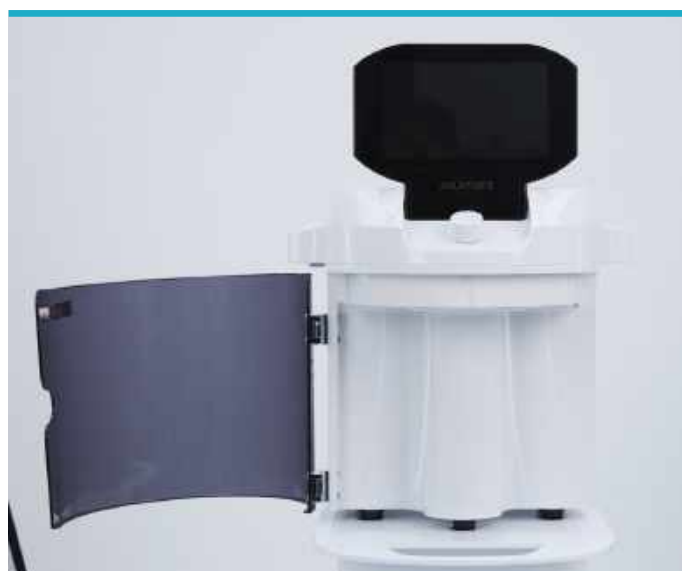

INSTALLATION GUIDE

Manufacturer CLASSYS Inc.

Tel +82-2-517-2114 **Fax** +82-2-6008-3457

Website www.cluederm.com **E-mail** info@cluederm.com