



[Step 1]

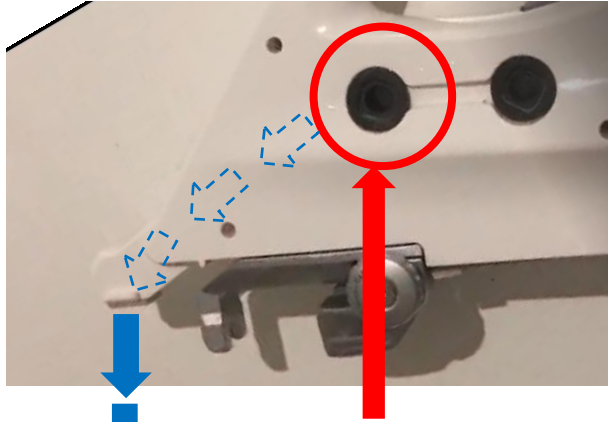

1. Fill warm water in the big syringe then connect the cleaning tool.
2. Remove the gun filter on the handpiece.
3. Connect the Cleaning tool to this hole.
4. Then, Push the syringe until the water comes out.
5. Repeat this process 2~3 times

[Step 2]

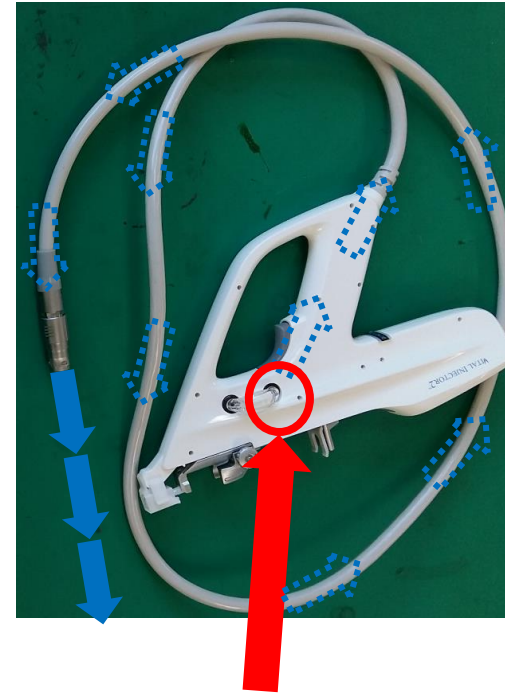

1. Fill warm water in the big syringe then connect the cleaning tool.
2. Remove the gun filter on the handpiece.
3. Connect the Cleaning tool to this hole.
4. Then, Push the syringe until the water comes out.
5. Repeat this process 2~3 times

*After the water cleaning, let it dry naturally for at least 3 hours until some water inside the vacuum tube dries.