- PRPHD PRP Tube is a vacuumed tube. The tube collects enough amount of blood and stops itself. (Apply both tubes, without removing the Blood Collection Needle from the vessel.)
- Centrifuge both tubes in a centrifuging device at 830G for 8 minutes. Buckets shall be compatible for PRPHD PRP Tubes (16 X 100 mm tube size).
- Attach Long Needle to 5ml Luer-Lok Syringe.
- Collect the PRP by sticking the Long needle into the PRPHD PRP Tube above the RBC Layer (Do not open the Lid of the tube unless it is not performed under Laminar Flow Conditions)
- Collect carefully 2ml of PRP Layer.
- Carry PRP to the PRPHD Re-suspension Tube.
- Apply again the same protocol for the other PRPHD PRP Tube.
- Carry again PRP to the PRPHD Re-suspension tube.
- You have app. 4 ml of PRP inside the PRPHD Re-suspension tube. (PRPHD re-suspension tube is 5 ml capacity sterile tube)
- Gently shake the PRPHD Re-suspension tube for 30 seconds to homogenize the settlement of the possible clumped cells in the plasma milieu. (Re-suspension process)
- Attach the application needle (Green Needle – 21G) to 1ml Luer-Lok syringe.
- Collect the re-suspended PRP inside from the PRPHD Re-suspension Tube.
- Change application needle (green needle – 21G) to injection needle (yellow needle – 30 G)
- Apply the injection method to the patient. Storage, Product Life and Handling Information
- Do not expose direct sunlight.
- Do not use if the inside sterile blister package is damaged or opened.
- Shelf life (Product Lifecycle) is the expiration.

Precautions, Warnings, Measures to be Taken and Limitations

- Use proper safety precautions to guard against needle sticks.
- Follow manufacturer instructions when using centrifuge.
- Do not use sterile component of this kit if package is opened or damaged.
- Do not open the lids of the tubes.
- Single use device, do not reuse.
- The user is to be thoroughly familiar with the equipment and the surgical procedure prior to using this device.
- The patient is to be made aware of general risks associated with treatment and possible adverse effects.
- Do not dispose of this product in the unsorted municipal waste stream.
- Long Needle is not intended to be used on human. It is just intended to be used to collect PRP Layer inside from the PRPHD PRP Tube(s)
- Patient Profile and Possible Contraindicative Situations
- Patient Profile
- Platelet rich plasma is an autologous application

technique and it is used on the same patient. Due to this information, platelet-rich plasma applications are inherently safe from infectious diseases such as HIV, Hepatitis, Creutzfeldt-Jakob disease... As an autologous technique; however Platelet-Rich Plasma can be used safely and effectively to all patients, there may have a need for further studies on pregnant and newborns. -Possible Contraindicative Situations Platelet Dysfunction Syndrome, Critical Thrombocytopaenia, Hypofibrinogenemia, Hemodynamic instability, Auto-immune diseases, Malignancy, Sepsis, Acute Chronic infections, Chronic Liver Pathology, Anti-coagulation Therapy (ASA therapy etc...), Pregnancy (for cosmetic indications), Newborns

Symbols and Explanations

	2195	Notified Body
		Catalogue Number
		Batch Code
		Manufacturing Date
		Caution, Consult Documents
		Not to be used in case package is damaged
		Sterilized using ethylene oxide
		Use by date
		Read usage instructions
		Do not resterilize
		Re-use is not allowed
		Manufacturer
		Keep away from sunlight
		Temperature Range
		İrredasyon uygulanarak sterilize edilmiştir.

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EN Instructions For Use

Table of Contents

- The name or trade name of the medical device
- The name and the address of the manufacturer
- Product Content List
- Intended use of the product
- The Performance of the Device Intended by the Manufacturer
- Clinical safety information
- Residual Risk information and Side effect information
- User Profile
- How to use
- Storage, Shelf Life and Handling information
- Precautions, Warnings, Measures to be taken and Limitations
- Patient Profile and Possible Contraindicative Situations
- Symbols and Explanations
- The Name or Trade Name of the Medical Device PRPHD PRP KIT
- The mentioned product is PRPHD PRP KIT. This product is a sterile, single use medical device.
- PRPHD PRP KIT has 2 packages.
- 1- Blister Package
- 2- Carton Box
- The Name and the Address of the Manufacturer

T-Biyoteknoloji Laboratuvar Estetik Medikal Kozmetik San. ve Tic. Ltd. Şti. Tahtalı Mah. Demirgen yolu Sok. No:10 (16280) Nilüfer / BURSA / TÜRKYE
Phone : +90 224 246 85
email : info@tbb.com.tr

Product Content List

- 2 units of PRPHD PRP Tube(s)
- 1 unit of PRPHD Re-suspension Tube
- 1 unit of Long Needle / Spinal Needle
- 1 unit of Holder
- 1 unit of 1ml Injector
- 1 unit of 5ml Injector
- 1 unit of 30G needle
- 1 unit of 21G needle
- 1 unit of Blood Collection Needle

Intended Use of the Product

The system is designed to be used for the safe and rapid preparation of autologous platelet-rich-plasma (PRP) from a small sample of blood at the patient's point of care. The PRP is prepared prior to application to a local area of soft tissue or hard tissue (including bone) site as deemed necessary by the clinical use requirements. And its intended use is for the application of PRP in order to achieve the stimulation of soft tissue and hard tissue

(Including bone) and acceleration of soft tissue and hard tissue (including bone) healing. The Performance of the Device Intended by the Manufacturer

PRP (Platelet Rich Plasma) is used for many years in the frame of regenerative medicine, which allows the cellular healing on bone and soft tissue recovery. Bone and soft tissue healing is occurred by the platelet concentration and re-injection locally to the related part of the body.

Concentration of platelets with buffy coat layer is very important for PRP treatments. Currently accepted technique for avoiding coagulation of the whole blood, referred to Sodium Citrate. The procedure uses anticoagulated blood, collected by routine phlebotomy. Buffered Citrate Solution blocks the coagulation by mixing gently the whole blood together with 3,20% concentrated 01M of Sodium Citrate. This preparation is then centrifuged to concentrate the PRP layer.

Recommended G force for the concentration is 830 G. The blood separation system takes the advantage of centrifugal force effecting on the weight of the cells. The separation occurs during centrifugation when the denser blood components move due to a gravity force. The PRPHD PRP Tube shall not be mixed after the centrifugation. 2ml of layer is harvested carefully above the erythrocytes. (Red Blood Cell Layer) The collected PRP layer is moved to PRPHD Re-suspension tube for re-suspension process. The result is a convenient tube system, collection of the whole blood and concentration of Platelet Rich Plasma. PRP Tubes included in PRPHD are sterilized using irradiation. PRPHD PRP Kit reduces the risk of air contamination eliminates the need of any other tubes or pipettes.

Clinical Safety Information
The safety and efficacy of this device has been proven by Risk Analysis Report and Clinical Evaluation Report, attached to the Technical Construction File of PRPHD PRP KIT. Residual Risk and Side Effects Information
According to the risk assessment, risk management, risk report procedures of this device, it shows no residual risks.

With respect to the requirements of Essential Requirements 6, the risk management report showed that there were no serious side effects and the performance of the product was verified by the clinical evaluation report.

User Profile
This product's intended user is Medical doctors (MDs) who shall be assisted by nurse practitioners according.

How to Use

- Apply the following instructions to obtain Platelet Rich Plasma by using PRPHD PRP KIT.
- Open the box.
 - Open the Sterile Blister Package
 - Attach Blood Collection Needle to Holder.
 - Apply the Blood Collection Needle to sanitized part of the patient's arm. (Venous)
 - Attach PRPHD PRP Tube inside the holder to Blood Collection Needle. (Routine Phlebotomy)