

Freeze ON Fat OFF



CLATUU 

No Limitations

Table of Contents

Components and Features

Installation

1. Installing the Monitor
2. Connecting Arm Stands
3. Connecting Cooling Cups
4. Inserting Water
5. Connecting the SAT Card
6. Turning the System ON

Components and Features



Main Body



Arm Stand (2x)



Monitor



Wing Type Cooling Cup
(2x : optional)




Oil Receiver



AC Power Cable



Screws (4x)

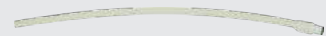


Plastic Tube

Installation

1. Installing the Monitor

The CLATUU Alpha Monitor connects on top of the Main Body facing outwards.

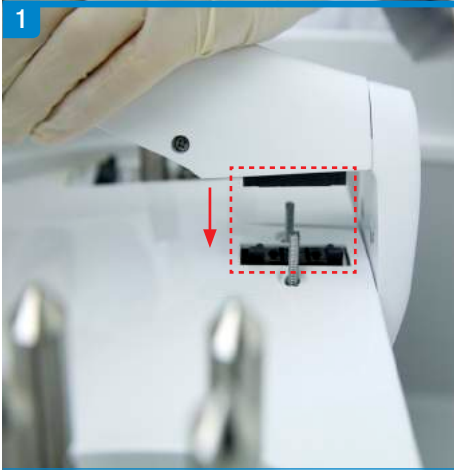

Align the connectors of both components and press down the Monitor.



Insert four screws with sockets at the base of the Monitor outlet.



Check that the Monitor is firmly connected as seen from the rear of the Main Body.

2. Connecting Arm Stands

The CLATUU Alpha Arm Stands connect to outlets located on opposite ends on top of the Main Body. Each Arm Stand should be facing outwards from the Main Body forming a “V” following successful connection.



The Hand-piece Cable connects from within the base of the Arm Stand.



Align the connectors of both components and press down firmly until a clicking sound occurs.



Press the button on top of each Arm Stand to adjust the length of the Hand-piece Cable.

Installation

3. Connecting Cooling Cups

The Hand-piece exchanges Cooling Cups for applying treatment to the skin.



The Hand-piece Cable connects from within the base of the Arm Stand.



Align the connectors of both components and press down firmly until a clicking sound occurs.




Check that the Cooling Cup is firmly connected to ensure that it does not disconnect or malfunction during treatments.

4. Inserting Water

VENT (top) and DRAIN (bottom) outlets are located at the lower rear of the Main Body.


The lid to the filter for inserting water is located at the top of the Main Body.

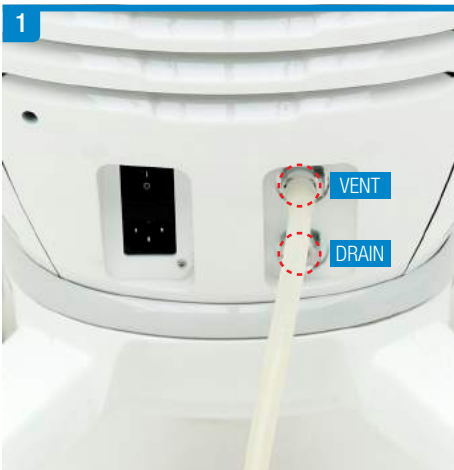

Insert the Plastic Tube into the VENT outlet with the opening at the opposite end placed in a container.



Pour approximately 2 Liters of water into the filter until water exits from the VENT outlet and into the container.

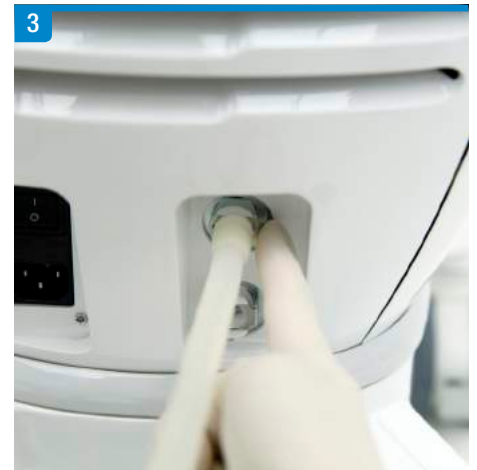


Press down the release button located next to the VENT outlet and remove the Plastic Tube.

Installation

5. Connecting the SAT Card

The SAT Card connects to the Main Body, providing memory to the System that enables all treatment functions of the CLATUU Alpha.

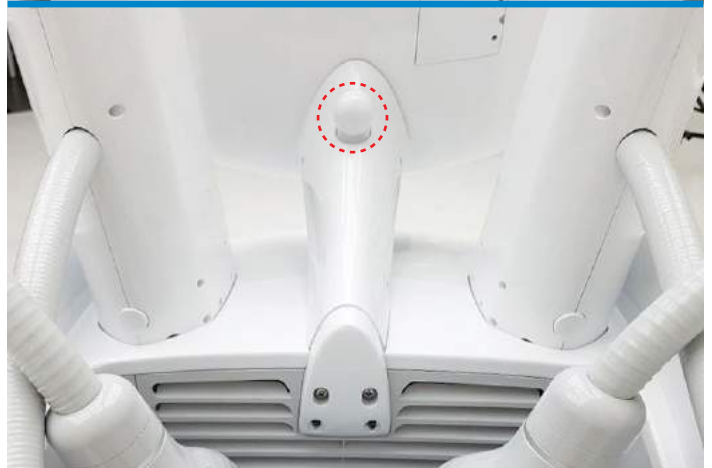

⚠ Inserting the SAT Card to any other device or inserting any other USB cable into the SAT Card USB port may cause malfunctions to occur.

Insert the SAT Card into the USB port located on the neck of the Monitor before proceeding with treatment.

6. Turning the System ON

The AC Power Switch is located at the lower rear of the Main Body with the power button located below the Monitor.



After connecting the AC Power Cable to the Main Body and grounded power source, press the 'I' symbol on the AC Power Switch.



Press the power button to turn the System ON.



Wait approx. 5 seconds for the System to load to the Main Graphical User Interface (GUI).

INSTALLATION GUIDE

Manufacturer CLASSYS Inc.

Tel +82-2-517-2114 **Fax** +82-2-6008-3457

Website www.classys.com **E-mail** info@classys.com