

— **Installation Guide**

Version 2.0AU

Reform
Your Youth



ULTRAFORMER III

MMFU

Micro and Macro Focused Ultrasound
Non-surgical Lifting, Tightening & Contouring System

Cryomed
Aesthetics

Table of Contents

Components and Features

Installation

1. Installing the Monitor to the Main Body
2. Connecting Hand-pieces to the Main Body
3. Connecting/Disconnecting Cartridges
4. Storing Cartridges in the Main Body
5. Conducting Checks
 - Connecting the AC Power Cable
 - Preventing Cables from Tangling
 - Proper Placement of Hand-pieces

Components and Features



Main Body



Cartridge



Hand-piece (2x)



System Access Key



AC Power Cable



Screws (2x)



Cartridge Storage Lid



Monitor



Monitor Cable
(Attached to Main Body)

Installation

1. Installing the Monitor to the Main Body

Before connecting the Monitor to the Main Body, ensure that the ULTRAFORMER III System is turned OFF for safety purposes.

The ULTRAFORMER III Monitor connects to the base on top of the Main Body.

The Monitor Cable comes out through the base and connects to the rear of the Monitor.

Slide the Monitor Cable through the opening at the neck of the Monitor before attaching it to the base.

Align two Screws with sockets behind the Main Body for successful connection of the Monitor to the Main Body.



Installation

Insert the Monitor Cable into the outlet behind the Monitor.



Installation

2. Connecting Hand-pieces to the Main Body

Before connecting the Hand-piece Cable to the Main Body, ensure that the System is turned OFF for safety purposes.

The ULTRAFORMER III Hand-pieces connect to Hand-piece Holders on opposite ends in front of the Main Body while the Hand-piece Cables (2x) connect behind the Main Body.

Slide the Hand-piece Cable down through the Slot Holder until the Hand-piece fits firmly into the open Hand-piece Holder.

Press the Hand-piece into the Hand-piece Holder to ensure a firm lock.

*Insert the Hand-piece into the Hand-piece Holder properly for the Automatic Hand-piece Recognition to function accordingly.



Installation

Match the arrow marked on the blue tip of the Hand-piece Cable with the square label on the blue rim of each outlet on the Main Body.

A clicking sound should occur indicating a successful connection of the Hand-piece Cables to each outlet.

To prevent the Hand-piece Cables from tangling, attach the cables from near the end of the outlets to the Slot Holder located underneath the Hand-piece Holders.



Installation

3. Connecting/Disconnecting Cartridges

Each ULTRAFORMER III Hand-piece is fitted with Cartridges for applying treatment to skin.

For successful connection of the Cartridge to the Hand-piece, align the Cartridge with the Hand-piece and proceed to connect.

A clicking sound should occur once the Cartridge and Hand-piece are connected properly.

For successful disconnection of the Cartridge from the Hand-piece, press down on the buttons located on both sides of the Hand-piece and remove the Cartridge.



Installation

4. Storing Cartridges in the Main Body

For Cartridges not used during the ULTRAFORMER III treatment, lift the Cartridge Storage Lid on the front side of the Main Body and place the Cartridges into each compartment (4x).

To successfully close the Cartridge Storage Lid, match the magnetic squares from the back end of the Cartridge Storage Lid with the top corners of the Cartridge Storage of the Main Body.



Installation


5. Conducting Checks

Connecting the AC Power Cable

Connect the AC Power Cable at the lower rear of the Main Body.

Proceed to plug in the AC Power Cable to a grounded power source.

To establish a safe power connection, press the AC Power Switch to ON of the Main Body.

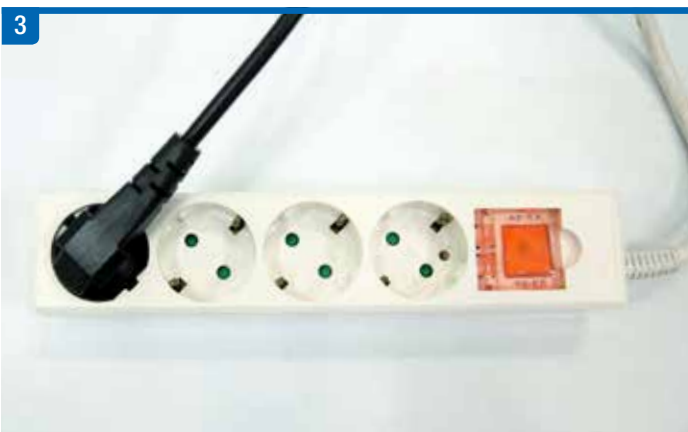

Installation

Preventing Cables from Tangling

For safety purposes, DO NOT entangle the ULTRAFORMER III Hand-piece or AC Power cable around the Main Body.

Keep each cable at a safe distance to maintain an organized and safe treatment environment for both the practitioner and patient.



Installation

Proper Placement of the Hand-pieces

To prevent improper insertion of the ULTRAFORMER III Hand-piece into the Hand-piece Holder, ensure to first slide the cable through the Slot Holder until the Hand-piece fits firmly into the open Hand-piece Holder.

Inserting the Hand-piece directly into the Hand-piece Holder may result in the Hand-piece being dropped during treatment or when moving the Main Body.



INSTALLATION GUIDE

Manufacturer CLASSYS Inc.
2F, Baekyoung Bldg., 546, Samseong-ro Gangnam-gu, Seoul, Rep. of Korea
Tel +82-2-517-2114 **Fax** +82-2-6008-3457
Website www.classys.com **E-mail** info@classys.com